



# HBV LONGITUDINAL PANEL

(Catalog No. 6501)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 27 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6501-01	3/22/01	3/23/01	5/11/74	3/15/01	JAUNDICE/ FATIGUE/ HEPATOMEGALY/ MALAISE	IVDA	TRANSFUSION	150	3/22/01	74.5	3/22/01	170	3/22/01
6501-02	4/5/01	4/5/01	5/11/74	3/15/01	JAUNDICE/ FATIGUE/ HEPATOMEGALY/ MALAISE	IVDA	TRANSFUSION	82	4/5/01	16.4	4/5/01	59	4/5/01
6501-03	4/19/01	4/20/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	38	4/19/01	11.1	4/19/01	39	4/19/01
6501-04	5/3/01	5/5/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	20	5/4/01	9.2	5/4/01	25	5/4/01
6501-05	5/17/01	5/22/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	39	5/21/01	20.0	5/21/01	40	5/21/01
6501-06	5/31/01	6/1/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	21	5/31/01	5.4	5/31/01	30	5/31/01
6501-07	6/13/01	6/14/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	13.6	6/13/01	6.8	6/13/01	27.2	6/13/01
6501-08	6/28/01	6/28/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	19.1	6/28/01	10.9	6/28/01	32.5	6/28/01
6501-09	7/12/01	7/16/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	95.0	7/13/01	8.0	7/13/01	24.3	7/13/01
6501-10	7/25/01	7/28/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	10.1	7/26/01	10.0	7/26/01	35.4	7/26/01
6501-11	8/9/01	8/13/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	13.2	8/9/01	11.4	8/9/01	29.0	8/9/01
6501-12	8/22/01	8/26/01	5/11/74	3/15/01	ASYMPTOMATIC	IVDA	TRANSFUSION	7.6	8/23/01	13.7	8/23/01	31.2	8/23/01



# HBV LONGITUDINAL PANEL

(Catalog No. 6501)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 27 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT T HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6501-01	NONE	AGUA MINERALE/ JUICE MULTIVITAMIN/SORBENT/ VITAMIN AEVIT/VITAMIN ACIDI ASCORBINICI	>40.8	0.04	>6.9	0.2	0.4	0.3	0.2	>4.9
6501-02	NONE	AGUA MINERALE/JUICE MULTIVITAMIN/ VITAMIN DUOVIT	>40.8	0.03	>6.9	0.2	0.4	0.2	N/A	N/A
6501-03	NONE	NONE	>40.8	0.06	4.8	0.3	0.5	0.3	N/A	N/A
6501-04	NONE	NONE	>40.8	0.07	3.6	0.2	0.4	0.3	N/A	N/A
6501-05	NONE	NONE	>40.8	0.07	2.6	0.2	0.4	0.2	N/A	N/A
6501-06	NONE	NONE	29.1	0.08	2.2	0.2	0.5	0.3	N/A	N/A
6501-07	NONE	NONE	0.4	0.06	1.7	0.2	0.5	0.7	N/A	N/A
6501-08	NONE	NONE	0.04	0.09	1.4	0.2	0.6	0.8	N/A	N/A
6501-09	NONE	HEPATIC TEA	0.2	0.09	1.1	0.2	0.8	0.8	N/A	N/A
6501-10	NONE	DIET N5	0.04	0.03	1.2	0.3	0.3	1.05	N/A	N/A
6501-11	NONE	DIET N5	0.09	0.03	1.12	0.3	0.4	1.09	N/A	N/A
6501-12	NONE	DIET N5	0.09	0.04	1.12	0.2	0.3	1.2	N/A	N/A