



HBV LONGITUDINAL PANEL

(Catalog No. 6502)

For Investigational Research Use Only

Application Notes: This 13 member panel was collected from a 21 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6502-01	3/22/01	3/28/01	4/30/80	3/22/01	JAUNDICE/ HEPATOMEGALY/ DARK URINE/MALAISE	IVDA	TRANSFUSION	410	3/23/01	95.9	3/23/01	170	3/23/01
6502-02	4/5/01	4/11/01	4/30/80	3/22/01	HEPATOMEGALY/ MALAISE	IVDA	TRANSFUSION	50.0	4/6/01	18.2	4/6/01	55	4/6/01
6502-03	4/19/01	4/23/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	27.0	4/20/01	14.3	4/20/01	30	4/20/01
6502-04	5/3/01	5/7/01	4/30/80	3/22/01	DARK URINE/ MALAISE	IVDA	TRANSFUSION	70	5/4/01	48.0	5/4/01	62	5/4/01
6502-05	5/17/01	5/21/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	52.0	5/18/01	26.0	5/18/01	40.0	5/18/01
6502-06	5/31/01	6/1/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	14.0	6/1/01	7.7	6/1/01	25.0	6/1/01
6502-07	6/14/01	6/19/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	22.7	6/15/01	18.1	6/15/01	26.8	6/15/01
6502-08	6/27/01	7/2/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	9.0	6/28/01	11.0	6/28/01	33.6	6/28/01
6502-09	7/11/01	7/14/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	17.3	7/12/01	16.0	7/12/01	29.0	7/12/01
6502-10	7/25/01	7/27/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	6.1	7/26/01	9.0	7/26/01	29.7	7/26/01
6502-11	8/9/01	8/13/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	2.9	8/9/01	9.3	8/9/01	25.1	8/9/01
6502-12	8/22/01	8/26/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	13.0	8/23/01	8.7	8/23/01	21	8/23/01
6502-13	11/26/01	11/29/01	4/30/80	3/22/01	ASYMPTOMATIC	IVDA	TRANSFUSION	9.3	11/28/01	11.0	11/28/01	33.0	11/28/01



HBV LONGITUDINAL PANEL

(Catalog No. 6502)

For Investigational Research Use Only

Application Notes: This 13 member panel was collected from a 21 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6502-01	NONE	AGUA MINERALE/ ENTEROSORBENT/AEVIT/ SOL. RIBOXINI	>40.8	0.06	1.3	0.2	0.8	0.3	0.2	>4.9
6502-02	NONE	AEVIT	>40.8	0.06	1.4	0.2	0.7	0.3	N/A	N/A
6502-03	NONE	NONE	>40.8	0.07	1.2	0.2	0.5	0.3	N/A	N/A
6502-04	NONE	AGUAMINERALE/DUOVIT/ HEPATIC TEA	>40.8	0.07	0.9	0.2	0.5	0.3	N/A	N/A
6502-05	NONE	NONE	20.5	0.07	0.9	0.2	0.4	0.2	N/A	N/A
6502-06	NONE	NONE	8.2	0.06	0.9	0.3	0.3	0.4	N/A	N/A
6502-07	NONE	NONE	4.0	0.10	0.8	0.2	0.4	0.3	N/A	N/A
6502-08	NONE	NONE	1.7	0.08	0.8	0.2	0.3	0.3	N/A	N/A
6502-09	NONE	NONE	1.4	0.08	0.8	0.3	0.3	0.3	N/A	N/A
6502-10	NONE	DIET N5	1.57	0.04	0.89	0.3	0.1	0.35	N/A	N/A
6502-11	NONE	DIET N5	1.74	0.05	0.81	0.2	0.1	0.29	N/A	N/A
6502-12	NONE	DIET N5	1.68	0.03	0.88	0.3	0.1	0.37	N/A	N/A
6502-13	HCV	DIET N5	0.08	0.08	0.71	0.2	0.1	0.31	N/A	N/A