



# HBV LONGITUDINAL PANEL

(Catalog No. 6504)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 24 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6504-01	3/26/01	3/28/01	5/27/77	3/26/01	JAUNDICE/ HEPATOMEGALY/ MALAISE	IVDA	TRANSFUSION	230.0	3/27/01	82.4	3/27/01	160	3/27/01
6504-02	4/6/01	4/11/01	5/27/77	3/26/01	HEPATOMEGALY	IVDA	TRANSFUSION	102	4/6/01	23.6	4/6/01	56	4/6/01
6504-03	4/19/01	4/20/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	26	4/20/01	20.5	4/20/01	22	4/20/01
6504-04	5/3/01	5/7/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	64	5/4/01	30	5/4/01	49	5/4/01
6504-05	5/22/01	5/23/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	12	5/23/01	13.2	5/23/01	19	5/23/01
6504-06	6/3/01	6/5/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	13.0	6/4/01	14.0	6/4/01	21.0	6/4/01
6504-07	6/14/01	6/19/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	18.5	6/15/01	16.1	6/15/01	27.3	6/15/01
6504-08	6/28/01	7/3/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	8.7	6/29/01	25.9	6/29/01	21.8	6/29/01
6504-09	7/12/01	7/16/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	33	7/13/01	20.0	7/13/01	30	7/13/01
6504-10	7/25/01	7/27/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	66.3	7/26/01	13.4	7/26/01	72.9	7/26/01
6504-11	8/9/01	8/15/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	13.6	8/15/01	9.7	8/15/01	11.3	8/15/01
6504-12	8/22/01	8/26/01	5/27/77	3/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	8.4	8/23/01	13.7	8/23/01	30.8	8/23/01



# HBV LONGITUDINAL PANEL

(Catalog No. 6504)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 24 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6504-01	NONE	AGUA MINERALE/ AEVIT/ ENTEROSORBENT	>40.8	0.06	>6.9	0.4	0.9	0.4	0.2	>4.9
6504-02	NONE	AEVIT	>40.8	0.06	>6.9	0.3	0.9	0.2	N/A	N/A
6504-03	NONE	NONE	>40.8	0.06	4.9	0.2	0.6	0.4	N/A	N/A
6504-04	NONE	NONE	33.2	0.08	3.7	0.1	0.5	0.3	N/A	N/A
6504-05	NONE	NONE	0.02	0.08	2.1	0.1	0.4	0.4	N/A	N/A
6504-06	NONE	NONE	0.06	0.08	2.2	0.1	0.4	0.3	N/A	N/A
6504-07	NONE	NONE	0.02	0.08	1.9	0.2	0.4	0.4	N/A	N/A
6504-08	NONE	NONE	0.02	0.06	1.9	0.1	0.4	0.2	N/A	N/A
6504-09	NONE	NONE	0.02	0.06	1.9	0.2	0.3	0.4	N/A	N/A
6504-10	NONE	DIET N5	0.02	0.04	1.43	0.3	0.1	0.57	N/A	N/A
6504-11	NONE	DIET N5	0.05	0.04	1.03	0.3	0.5	0.89	N/A	N/A
6504-12	NONE	DIET N5	0.07	0.03	0.79	0.2	0.4	0.65	N/A	N/A