



HBV LONGITUDINAL PANEL

(Catalog No. 6513)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 17 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6513-01	6/5/01	6/6/01	2/7/84	6/4/01	ANOREXIA/ FATIGUE/HEPATOMEGALY/ DARK URINE/MALaise	IVDA	TRANSFUSION	648	6/5/01	20	6/5/01	501	6/5/01
6513-02	6/19/01	6/21/01	2/7/84	6/4/01	ASYMPTOMATIC	IVDA	TRANSFUSION	107.8	6/20/01	9.3	6/20/01	134.5	6/20/01
6513-03	7/4/01	7/6/01	2/7/84	6/4/01	ASYMPTOMATIC	IVDA	TRANSFUSION	29.6	7/5/01	5.9	7/5/01	28.1	7/5/01
6513-04	7/17/01	7/19/01	2/7/84	6/4/01	ASYMPTOMATIC	IVDA	TRANSFUSION	26	7/19/01	11.0	7/19/01	24	7/19/01
6513-05	7/31/01	8/6/01	2/7/84	6/4/01	ASYMPTOMATIC	IVDA	TRANSFUSION	158	8/3/01	10.8	8/3/01	239.3	8/3/01
6513-06	8/14/01	8/15/01	2/7/84	6/4/01	ASYMPTOMATIC	IVDA	TRANSFUSION	180	8/15/01	11.3	8/15/01	243.6	8/15/01
6513-07	8/28/01	9/3/01	2/7/84	6/4/01	HEPATOMEGALY	IVDA	TRANSFUSION	82.5	8/29/01	11.2	8/29/01	223	8/29/01
6513-08	9/11/01	9/14/01	2/7/84	6/4/01	HEPATOMEGALY	IVDA	TRANSFUSION	10.7	9/12/01	11.4	9/12/01	30.2	9/12/01
6513-09	9/25/01	9/27/01	2/7/84	6/4/01	HEPATOMEGALY	IVDA	TRANSFUSION	100.7	9/26/01	13.0	9/26/01	195.9	9/26/01
6513-10	10/9/01	10/14/01	2/7/84	6/4/01	HEPATOMEGALY	IVDA	TRANSFUSION	60.0	10/10/01	18.0	10/10/01	104.0	10/10/01
6513-11	10/23/01	10/24/01	2/7/84	6/4/01	ASYMPTOMATIC	IVDA	TRANSFUSION	19.0	10/24/01	14.5	10/24/01	226	10/24/01
6513-12	11/6/01	11/9/01	2/7/84	6/4/01	HEPATOMEGALY	IVDA	TRANSFUSION	73.0	11/9/01	8.8	11/9/01	125.0	11/9/01



HBV LONGITUDINAL PANEL

(Catalog No. 6513)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 17 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6513-01	NONE	AGUA MINERALE/ SOL. RIBOXINI/DUOVIT	>40.8	0.05	0.7	0.1	0.1	0.2	4.8	>4.9
6513-02	NONE	NONE	>40.8	0.06	0.8	5.9	1.7	0.3	N/A	N/A
6513-03	NONE	NONE	>40.8	0.10	0.6	12.9	1.9	0.3	N/A	N/A
6513-04	NONE	DIET N5	>36.4	0.02	0.8	39.0	4.8	0.4	N/A	N/A
6513-05	HCV	DIET N5/ AQUA MINERALE/ AEVIT	>36.4	0.03	0.98	39.1	4.8	0.5	N/A	N/A
6513-06	HCV	DIET N5/ HEPATIC TEA/ AEVIT	>36.4	0.02	1.05	38.0	4.8	0.4	N/A	N/A
6513-07	HCV	DIET N5/ HEPATIC TEA	>36.4	0.01	1.13	33.0	4.5	0.5	N/A	N/A
6513-08	HCV	DIET N5	>36.4	0.01	1.03	26.2	3.2	0.2	N/A	N/A
6513-09	HCV	DIET N5/ ESSENCIALE/ HEPATIC TEA	>36.4	0.01	0.90	2.6	0.7	0.4	N/A	N/A
6513-10	HCV	DIET N5/ ESSENCIALE/ HEPATIC TEA	>36.4	0.03	0.84	1.0	0.4	0.3	N/A	N/A
6513-11	HCV	DIET N5/ HEPATIC TEA/ DUOVIT	>36.4	0.02	0.99	1.5	0.3	0.4	N/A	N/A
6513-12	HCV	DIET N5/ ESSENCIALE/ HEPATIC TEA	>36.4	0.03	0.9	0.2	0.01	0.3	N/A	N/A