



HBV LONGITUDINAL PANEL

(Catalog No. 6518)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 20 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6518-01	7/3/01	7/6/01	6/12/81	7/3/01	JAUNDICE/ HEPATOMEGALY/ DARK URINE/ MALAISE	IVDA	TRANSFUSION	1530	7/4/01	85.2	7/4/01	798	7/4/01
6518-02	7/17/01	7/19/01	6/12/81	7/3/01	JAUNDICE/ HEPATOMEGALY	IVDA	TRANSFUSION	1120	7/18/01	37.0	7/18/01	412	7/18/01
6518-03	7/31/01	8/6/01	6/12/81	7/3/01	ASYMPTOMATIC	IVDA	TRANSFUSION	220	8/3/01	42.6	8/3/01	192	8/3/01
6518-04	8/14/01	8/20/01	6/12/81	7/3/01	ASYMPTOMATIC	IVDA	TRANSFUSION	24.5	8/15/01	28.2	8/15/01	14.3	8/15/01
6518-05	8/28/01	9/4/01	6/12/81	7/3/01	ASYMPTOMATIC	IVDA	TRANSFUSION	23.0	8/28/01	20.0	8/28/01	19.0	8/28/01
6518-06	9/11/01	9/17/01	6/12/81	7/3/01	ASYMPTOMATIC	IVDA	TRANSFUSION	31.9	9/12/01	10.7	9/12/01	23.9	9/12/01
6518-07	9/25/01	9/28/01	6/12/81	7/3/01	HEPATOMEGALY	IVDA	TRANSFUSION	262.8	9/26/01	15.2	9/26/01	135.6	9/26/01
6518-08	10/12/01	10/17/01	6/12/81	7/3/01	HEPATOMEGALY	IVDA	TRANSFUSION	174	10/12/01	13.2	10/12/01	67	10/12/01
6518-09	10/25/01	10/26/01	6/12/81	7/3/01	HEPATOMEGALY	IVDA	TRANSFUSION	168.0	10/25/01	13.6	10/25/01	58.0	10/25/01
6518-10	11/6/01	11/10/01	6/12/81	7/3/01	HEPATOMEGALY	IVDA	TRANSFUSION	156.0	11/7/01	11.0	11/7/01	93.0	11/7/01
6518-11	11/22/01	11/24/01	6/12/81	7/3/01	HEPATOMEGALY	IVDA	TRANSFUSION	83.0	11/22/01	16.4	11/22/01	64.0	11/22/01
6518-12	12/4/01	12/6/01	6/12/81	7/3/01	ASYMPTOMATIC	IVDA	TRANSFUSION	101	12/5/01	17.5	12/5/01	55.5	12/5/01



HBV LONGITUDINAL PANEL

(Catalog No. 6518)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 20 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6518-01	NONE	AGUA MINERALE/ DUOVIT/SOL. RIBOXINI	>40.8	0.05	>6.4	0.3	0.4	0.3	0.1	0.3
6518-02	NONE	AGUA MINERALE/ HEPATIC TEA/AEVIT	>40.8	0.06	>6.4	0.3	0.3	0.3	N/A	N/A
6518-03	HCV	DIET N5/ AQUA MINERALE/ ESSENCIALE	>37.7	0.03	>6.9	0.2	0.07	0.2	N/A	N/A
6518-04	HCV	DIET N5	>37.7	0.05	6.1	0.2	0.02	0.2	N/A	N/A
6518-05	HCV	DIET N5	>37.7	0.07	4.8	0.1	0.02	0.2	N/A	N/A
6518-06	HCV	DIET N5	>37.7	0.06	3.3	0.2	0.01	0.2	N/A	N/A
6518-07	HCV	DIET N5/ ESSENCIALE/ HEPATIC TEA	>37.7	0.05	4.3	0.1	0.02	0.3	N/A	N/A
6518-08	HCV	DIET N5/ DUOVIT	0.4	0.05	3.1	0.2	0.01	0.3	N/A	N/A
6518-09	HCV	DIET N5/ DUOVIT	0.4	0.03	3.8	0.1	0.02	0.5	N/A	N/A
6518-10	HCV	DIET N5/ DUOVIT/ HEPATIC TEA	0.05	0.06	3.1	0.2	0.01	0.6	N/A	N/A
6518-11	HCV	DIET N5/ HEPATIC TEA	0.1	0.04	2.9	0.1	0.02	1.3	N/A	N/A
6518-12	HCV	DIET N5/ AQUA MINERALE	0.1	0.04	2.9	0.2	0.01	1.4	N/A	N/A