



HBV LONGITUDINAL PANEL

(Catalog No. 6528)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 20 year old Caucasian Male.

| PANEL NUMBER | DATE OF DRAW | DATE CRF COMPLETED | DATE OF BIRTH | DODX | SIGNS AND SYMPTOMS | RISK GROUP | MODE OF TRANSMISSION | ALT | ALT DATE | BILIRUBIN | BILIRUBIN DATE | SGOT | SGOT DATE |
|--------------|--------------|--------------------|---------------|---------|--------------------|------------|----------------------|-------|----------|-----------|----------------|-------|-----------|
| 6528-01 | 9/18/01 | 9/21/01 | 2/17/81 | 9/16/01 | HEPATOMEGALY | IVDA | TRANSFUSION | 367.0 | 9/19/01 | 27.4 | 9/19/01 | 162.0 | 9/19/01 |
| 6528-02 | 10/2/01 | 10/5/01 | 2/17/81 | 9/16/01 | HEPATOMEGALY | IVDA | TRANSFUSION | 146.4 | 10/3/01 | 18.0 | 10/3/01 | 76.6 | 10/3/01 |
| 6528-03 | 10/17/01 | 11/3/01 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 49.6 | 10/18/01 | 18.6 | 10/18/01 | 53.6 | 10/18/01 |
| 6528-04 | 10/30/01 | 11/3/01 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 37.5 | 10/31/01 | 9.2 | 10/31/01 | 47.0 | 10/31/01 |
| 6528-05 | 11/13/01 | 11/16/01 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 40.3 | 11/13/01 | 14.4 | 11/13/01 | 43.6 | 11/13/01 |
| 6528-06 | 11/28/01 | 11/30/01 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 49 | 11/29/01 | 13.8 | 11/29/01 | 59 | 11/29/01 |
| 6528-07 | 12/11/01 | 12/13/01 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 8.0 | 12/12/01 | 9.0 | 12/12/01 | 15.0 | 12/12/01 |
| 6528-08 | 12/26/01 | 12/27/01 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 43.0 | 12/26/01 | 16.0 | 12/26/01 | 48.0 | 12/26/01 |
| 6528-09 | 1/8/02 | 1/16/02 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 4.0 | 1/9/02 | 8.4 | 1/9/02 | 17.0 | 1/9/02 |
| 6528-10 | 1/22/02 | 1/24/02 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 22.3 | 1/23/02 | 5.5 | 1/23/02 | 35.2 | 1/23/02 |
| 6528-11 | 2/5/02 | 2/7/02 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 16.0 | 2/6/02 | 9.7 | 2/6/02 | 35.3 | 2/6/02 |
| 6528-12 | 2/19/02 | 2/22/02 | 2/17/81 | 9/16/01 | ASYMPTOMATIC | IVDA | TRANSFUSION | 28.7 | 2/19/02 | 11.0 | 2/19/02 | 23.0 | 2/19/02 |



HBV LONGITUDINAL PANEL

(Catalog No. 6528)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 20 year old Caucasian Male.

| PANEL NUMBER | CONCURRENT MEDICAL CONDITIONS | MEDICATIONS | ABBOTT HBSAG S/CO | ABBOTT HBC TOTAL S/CO | ABBOTT HBC IGM S/CO | HBEAG DIASORIN S/CO | ANTI-HBE DIASORIN S/CO | ABBOTT ANTI-HBS S/CO | ABBOTT ANTI-HIV 1/2 S/CO | ABBOTT HCV S/CO |
|--------------|-------------------------------|--------------------------------|-------------------|-----------------------|---------------------|---------------------|------------------------|----------------------|--------------------------|-----------------|
| 6528-01 | NONE | DIET N5/ HEPATIC TEA/ RIBOXINI | >37.7 | 0.06 | >6.9 | 0.75 | 0.3 | 0.2 | 0.09 | 0.3 |
| 6528-02 | NONE | DIET N5/ HEPATIC TEA | >37.7 | 0.04 | 6.6 | 0.03 | 0.1 | 0.2 | N/A | N/A |
| 6528-03 | NONE | DIET N5 | >37.7 | 0.04 | 4.7 | 0.01 | 0.06 | 0.3 | N/A | N/A |
| 6528-04 | NONE | DIET N5 | >37.7 | 0.05 | 3.8 | 0.08 | 0.03 | 0.2 | N/A | N/A |
| 6528-05 | NONE | DIET N5 | >37.7 | 0.04 | 3.4 | 0.13 | 0.01 | 0.3 | N/A | N/A |
| 6528-06 | NONE | DIET N5 | 17.2 | 0.06 | 3.1 | 0.08 | 0.02 | 3.8 | N/A | N/A |
| 6528-07 | NONE | DIET N5 | 0.7 | 0.05 | 2.6 | 0.08 | 0.01 | 4.1 | N/A | N/A |
| 6528-08 | NONE | DIET N5 | 0.02 | 0.06 | 2.5 | 0.03 | 0.02 | 3.2 | N/A | N/A |
| 6528-09 | NONE | DIET N5 | 0.02 | 0.05 | 2.2 | 0.08 | 0.01 | 3.2 | N/A | N/A |
| 6528-10 | NONE | DIET N5 | 0.02 | 0.06 | 1.9 | 0.02 | 0.01 | 3.7 | N/A | N/A |
| 6528-11 | NONE | DIET N5 | 0.02 | 0.05 | 1.8 | 0.02 | 0.01 | 5.3 | N/A | N/A |
| 6528-12 | NONE | DIET N5 | 0.03 | 0.06 | 2.2 | 0.33 | 0.03 | 5.8 | N/A | N/A |