



# HBV LONGITUDINAL PANEL

(Catalog No. 6530)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 19 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6530-01	10/18/01	10/23/01	4/22/82	10/12/01	JAUNDICE/ HEPATOMEGALY/ DARK URIN	IVDA	TRANSFUSION	136.0	10/19/01	85.2	10/19/01	983.0	10/19/02
6530-02	11/1/01	11/5/01	4/22/82	10/12/01	HEPATOMEGALY	IVDA	TRANSFUSION	239.0	11/1/01	27.4	11/1/01	66.0	11/1/01
6530-03	11/15/01	11/12/01	4/22/82	10/12/01	HEPATOMEGALY	IVDA	TRANSFUSION	259.0	11/16/01	24.4	11/16/01	204.0	11/16/01
6530-04	11/27/01	11/29/01	4/22/82	10/12/01	HEPATOMEGALY	IVDA	TRANSFUSION	220.0	11/28/01	18.7	11/28/01	93.1	11/28/01
6530-05	12/11/01	12/13/01	4/22/82	10/12/01	ASYMPTOMATIC	IVDA	TRANSFUSION	52.0	12/12/01	13.2	12/12/01	31.0	12/12/01
6530-06	12/25/01	12/27/01	4/22/82	10/12/01	ASYMPTOMATIC	IVDA	TRANSFUSION	19.0	12/26/01	12.2	12/26/01	30.0	12/26/01
6530-07	1/8/02	1/16/02	4/22/82	10/12/01	ASYMPTOMATIC	IVDA	TRANSFUSION	22.0	1/9/02	15.0	1/9/02	33.0	1/9/02
6530-08	1/22/02	1/24/02	4/22/82	10/12/01	ASYMPTOMATIC	IVDA	TRANSFUSION	8.0	1/23/02	12.2	1/23/02	20.0	1/23/02
6530-09	2/5/02	2/8/02	4/22/82	10/12/01	ASYMPTOMATIC	IVDA	TRANSFUSION	9.4	2/6/02	6.0	2/6/02	48.8	2/6/02
6530-10	2/19/02	2/22/02	4/22/82	10/12/01	ASYMPTOMATIC	IVDA	TRANSFUSION	12.9	2/20/02	7.0	2/20/02	24.2	2/20/02
6530-11	3/5/02	3/7/02	4/22/82	10/12/01	ASYMPTOMATIC	IVDA	TRANSFUSION	25.5	3/6/02	8.7	3/6/02	93.0	3/6/02
6530-12	3/18/02	3/21/02	4/22/82	10/12/01	ASYMPTOMATIC	IVDA	TRANSFUSION	12.0	3/19/02	17.5	3/19/02	34.0	3/19/02



# HBV LONGITUDINAL PANEL

(Catalog No. 6530)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 19 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6530-01	NONE	DIET N5/ ENTEROSREL/ RIBOXINI/ AEVIT	>36.4	0.02	>4.7	0.10	0.06	0.3	0.13	0.2
6530-02	NONE	DIET N5/ AEVIT/ AQUA MINERALE	>36.4	0.04	>4.7	0.22	0.01	0.3	N/A	N/A
6530-03	NONE	DIET N5/ DUOVIT/ HEPATIC TEA	>36.4	0.04	>4.7	0.09	0.01	0.4	N/A	N/A
6530-04	NONE	DIET N5/ DUOVIT/ ESSENCIALE/ HEPATIC TEA	>36.4	0.02	>4.7	0.10	0.01	0.3	N/A	N/A
6530-05	NONE	DIET N5/ HEPATIC TEA/ ESSENCIALE	>36.4	0.04	4.0	0.05	0.01	0.3	N/A	N/A
6530-06	NONE	DIET N5	>36.4	0.04	3.0	0.13	0.02	0.3	N/A	N/A
6530-07	NONE	DIET N5	0.7	0.06	3.3	0.09	0.05	0.4	N/A	N/A
6530-08	NONE	DIET N5	0.2	0.04	1.9	0.09	0.01	0.5	N/A	N/A
6530-09	NONE	DIET N5	0.05	0.04	1.9	0.05	0.01	0.5	N/A	N/A
6530-10	NONE	DIET N5	0.1	0.05	1.8	0.16	0.01	0.7	N/A	N/A
6530-11	NONE	DIET N5	13.995	0.07	1.6	0.08	0.01	0.6	N/A	N/A
6530-12	NONE	DIET N5	0.2	0.07	1.4	0.19	0.01	0.8	N/A	N/A