



# HBV LONGITUDINAL PANEL

(Catalog No. 6535)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 23 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6535-01	11/13/01	11/16/01	6/19/78	11/9/01	JAUNDICE/ HEPATOMEGALY	IVDA	TRANSFUSION	492.0	11/15/01	58.6	11/15/01	308.0	11/15/01
6535-02	11/26/01	11/29/01	6/19/78	11/9/01	HEPATOMEGALY	IVDA	TRANSFUSION	211.1	11/27/01	28.3	11/27/01	112.9	11/27/01
6535-03	12/11/01	12/14/01	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	69.0	12/12/01	16.7	12/12/01	47.0	12/12/01
6535-04	12/25/01	12/27/01	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	33.0	12/26/01	12.5	12/26/01	29.0	12/26/01
6535-05	1/8/02	1/10/02	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	160.0	1/9/02	16.4	1/9/02	136.0	1/9/02
6535-06	1/22/02	1/24/01	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	222.0	1/23/02	11.9	1/23/02	122.0	1/23/02
6535-07	2/5/02	2/7/02	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	41.9	2/6/02	6.4	2/6/02	33.2	2/6/02
6535-08	2/19/02	2/22/02	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	16.0	2/20/02	11.0	2/20/02	28.7	2/20/02
6535-09	3/5/02	3/7/02	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	10.7	3/6/02	12.9	3/6/02	33.0	3/6/02
6535-10	3/19/02	3/22/02	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	16.0	3/20/02	4.9	3/20/02	19.0	3/20/02
6535-11	4/2/02	4/5/02	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	32.0	4/4/02	9.6	4/4/02	38.0	4/4/02
6535-12	4/16/02	4/19/02	6/19/78	11/9/01	ASYMPTOMATIC	IVDA	TRANSFUSION	12.0	4/17/02	5.3	4/17/02	23.0	4/17/02



# HBV LONGITUDINAL PANEL

(Catalog No. 6535)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 23 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6535-01	HCV	DIET N5/ AQUA MINERALE/ AEVIT/ HEPATIC TEA	>36.4	0.03	>4.7	0.13	0.09	0.3	0.16	>6.2
6535-02	NONE	DIET N5/ HEPATIC TEA/ AEVIT	>36.4	0.04	>4.7	0.06	0.1	0.3	N/A	N/A
6535-03	HCV	DIET N5/ AQUA MINERALE/ HEPATIC TEA	>36.4	0.05	>4.7	0.09	0.03	0.3	N/A	N/A
6535-04	HCV	DIET N5	1.8	0.04	>4.7	0.06	0.02	0.3	N/A	N/A
6535-05	HCV	DIET N5/ AQUA MINERALE/ RIBOXINI	0.2	0.06	4.6	0.08	0.02	0.3	N/A	N/A
6535-06	HCV	DIET N5/ RIBOXINI/ ESSENCIALE	0.2	0.04	3.0	0.01	0.01	0.6	N/A	N/A
6535-07	HCV	DIET N5/ ESSENCIALE	0.1	0.06	2.0	0.05	0.01	0.7	N/A	N/A
6535-08	HCV	DIET N5	0.1	0.06	1.8	0.03	0.01	0.8	N/A	N/A
6535-09	HCV	DIET N5	0.2	0.06	2.1	0.08	0.01	1.0	N/A	N/A
6535-10	HCV	DIET N5	0.2	0.06	1.8	0.03	0.01	1.0	N/A	N/A
6535-11	HCV	DIET N5	0.2	0.05	1.4	0.14	0.03	1.3	N/A	N/A
6535-12	HCV	DIET N5	0.1	0.09	1.6	0.08	0.01	1.2	N/A	N/A