



HBV LONGITUDINAL PANEL

(Catalog No. 6537)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 17 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE	CONCURRENT MEDICAL CONDITIONS
6537-01	11/28/01	11/30/01	7/5/84	11/26/01	HEPATOMEGALY	IVDA	TRANSFUSION	474.5	11/29/01	19.5	11/29/01	84.0	11/29/01	NONE
6537-02	12/12/01	12/14/01	7/5/84	11/26/01	HEPATOMEGALY	IVDA	TRANSFUSION	58	12/13/01	13.6	12/13/01	20	12/13/01	NONE
6537-03	12/26/01	12/28/01	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	19.0	12/26/01	8.3	12/26/01	19.0	12/26/01	NONE
6537-04	1/8/02	1/10/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	13.0	1/9/02	2.5	1/9/02	14.0	1/9/02	NONE
6537-05	1/27/02	1/29/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	17.0	1/28/02	7.0	1/28/02	23.0	1/28/02	NONE
6537-06	2/5/02	2/8/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	8.0	2/7/02	6.7	2/7/02	20.6	2/7/02	NONE
6537-07	2/19/02	2/22/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	13.7	2/20/02	8.3	2/20/02	22.1	2/20/02	NONE
6537-08	3/5/02	3/7/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	7.0	3/6/02	10.0	3/6/02	21.7	3/6/02	NONE
6537-09	3/19/02	3/22/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	11.0	3/20/02	7.9	3/20/02	14.0	3/20/02	NONE
6537-10	4/2/02	4/5/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	12.0	4/4/02	3.1	4/4/02	14.0	4/4/02	NONE
6537-11	4/16/02	4/19/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	15.0	4/17/02	4.1	4/17/02	15.0	4/17/02	NONE
6537-12	4/30/02	5/4/02	7/5/84	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	10.2	5/3/02	18.5	5/3/02	25.0	5/3/02	NONE



HBV LONGITUDINAL PANEL

(Catalog No. 6537)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 17 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6537-01	NONE	DIET N5/ HEPATIC TEA/ AEVIT/ RIBOXINI	>37	0.03	>6.5	0.05	0.1	0.2	0.09	0.2
6537-02	NONE	DIET N5/ HEPATIC TEA/ AEVIT/ RIBOXINI	>37	0.1	>6.5	0.09	0.06	0.5	N/A	N/A
6537-03	NONE	DIET N5	>37	0.1	>6.5	0.04	0.05	0.3	N/A	N/A
6537-04	NONE	DIET N5	>37	0.1	>6.5	0.19	0.05	0.3	N/A	N/A
6537-05	NONE	DIET N5	>37	0.04	1.9	0.08	0.06	0.3	N/A	N/A
6537-06	NONE	DIET N5	1.1	0.1	4.0	0.14	0.06	0.3	N/A	N/A
6537-07	NONE	DIET N5	0.3	0.1	2.9	0.07	0.05	0.5	N/A	N/A
6537-08	NONE	DIET N5	0.3	0.1	2.5	0.15	0.05	0.6	N/A	N/A
6537-09	NONE	DIET N5	0.1	0.1	2.5	0.01	0.04	0.7	N/A	N/A
6537-10	NONE	DIET N5	0.1	0.1	2.4	0.11	0.05	1.8	N/A	N/A
6537-11	NONE	DIET N5	0.1	0.1	2.2	0.04	0.05	3.4	N/A	N/A
6537-12	NONE	DIET N5	0.1	0.1	2.2	0.11	0.05	3.1	N/A	N/A