



HBV LONGITUDINAL PANEL

(Catalog No. 6538)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 21 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6538-01	11/28/01	11/30/01	11/12/80	11/26/01	JAUNDICE/ HEPATOMEGALY	IVDA	TRANSFUSION	343.0	11/29/01	71.0	11/29/01	432.0	11/29/01
6538-02	12/12/01	12/14/01	11/12/80	11/26/01	HEPATOMEGALY	IVDA	TRANSFUSION	189.0	12/13/01	21.2	12/13/01	99.0	12/13/01
6538-03	12/26/01	12/28/01	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	35.0	12/26/01	16.4	12/26/01	36.0	12/26/01
6538-04	1/8/02	1/10/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	21.0	1/9/02	13.4	1/9/02	26.0	1/9/02
6538-05	1/22/02	1/24/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	6.0	1/23/02	6.9	1/23/02	19.0	1/23/02
6538-06	2/6/02	2/7/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	12.0	2/6/02	7.7	2/6/02	19.0	2/6/02
6538-07	2/19/02	2/22/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	8.2	2/20/02	9.8	2/20/02	34.8	2/20/02
6538-08	3/5/02	3/7/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	8.5	3/6/02	11.8	3/6/02	127.7	3/6/02
6538-09	3/19/02	3/22/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	18.0	3/20/02	12.7	3/20/02	22.0	3/20/02
6538-10	4/2/02	4/4/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	10.0	4/3/02	3.8	4/3/02	48.0	4/3/02
6538-11	4/16/02	4/19/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	14.0	4/17/02	3.8	4/17/02	14.0	4/17/02
6538-12	4/30/02	5/4/02	11/12/80	11/26/01	ASYMPTOMATIC	IVDA	TRANSFUSION	10.2	5/3/02	9.5	5/3/02	27.1	5/3/02



HBV LONGITUDINAL PANEL

(Catalog No. 6538)

For Investigational Research Use Only

Application Notes: This 12 member panel was collected from a 21 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6538-01	NONE	DIET N5/ HEPATIC TEA/ RIBOXIN/ AEVIT	>37	0.04	>6.5	0.08	0.04	0.3	0.10	>6.2
6538-02	HCV	DIET N5/ HEPATIC TEA/ RIBOXIN/ AEVIT	>37	0.04	5.2	0.12	0.05	0.3	N/A	N/A
6538-03	HCV	DIET N5	>37	0.1	4.2	0.04	0.05	0.3	N/A	N/A
6538-04	HCV	DIET N5	>37	0.1	3.1	0.12	0.04	0.3	N/A	N/A
6538-05	HCV	DIET N5	>37	0.1	2.4	0.24	0.05	0.4	N/A	N/A
6538-06	HCV	DIET N5	>37	0.03	1.6	0.12	0.05	0.2	N/A	N/A
6538-07	HCV	DIET N5	>37	0.1	2.1	0.04	0.03	0.3	N/A	N/A
6538-08	HCV	DIET N5	2.4	0.1	1.9	0.14	0.05	0.3	N/A	N/A
6538-09	HCV	DIET N5	1.1	0.1	1.6	0.07	0.04	0.2	N/A	N/A
6538-10	HCV	DIET N5	0.2	0.1	1.9	0.18	0.05	0.3	N/A	N/A
6538-11	HCV	DIET N5	0.2	0.1	1.5	0.08	0.05	0.4	N/A	N/A
6538-12	HCV	DIET N5	0.2	0.1	1.5	0.12	0.04	0.4	N/A	N/A