



# HBV LONGITUDINAL PANEL

(Catalog No. 6548)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 17 year old Caucasian Male.

PANEL NUMBER	DATE OF DRAW	DATE CRF COMPLETED	DATE OF BIRTH	DODX	SIGNS AND SYMPTOMS	RISK GROUP	MODE OF TRANSMISSION	ALT	ALT DATE	BILIRUBIN	BILIRUBIN DATE	SGOT	SGOT DATE
6548-01	2/8/02	2/10/02	7/9/64	2/6/02	JAUNDICE/ HEPATOMEGALY/ DARK URINE	SEXUAL CONTACT	SEXUAL CONTACT	201.6	2/9/02	212.0	2/9/02	789	2/9/02
6548-02	2/22/02	2/24/02	7/9/64	2/6/02	HEPATOMEGALY	SEXUAL CONTACT	SEXUAL CONTACT	464.0	2/22/02	29.7	2/22/02	168.0	2/22/02
6548-03	3/9/02	3/11/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	28.8	3/10/02	11.9	3/10/02	62.0	3/10/02
6548-04	3/23/02	3/25/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	38.0	3/24/02	10.7	3/24/02	26.0	3/24/02
6548-05	4/6/02	4/10/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	31.0	4/8/02	7.8	4/8/02	28.0	4/8/02
6548-06	4/20/02	4/23/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	30.0	4/22/02	14.0	4/22/02	20.0	4/22/02
6548-07	5/4/02	5/7/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	26.4	5/6/02	9.8	5/6/02	36.9	5/6/02
6548-08	5/18/02	5/21/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	16.5	5/20/02	12.7	5/20/02	25.7	5/20/02
6548-09	6/1/02	6/4/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	25.3	6/3/02	11.2	6/3/02	32.1	6/3/02
6548-10	6/15/02	6/18/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	17.3	6/17/02	4.2	6/17/02	26.6	6/17/02
6548-11	6/29/02	7/2/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	16.6	7/1/02	7.2	7/1/02	23.5	7/1/02
6548-12	7/13/02	7/16/02	7/9/64	2/6/02	ASYMPTOMATIC	SEXUAL CONTACT	SEXUAL CONTACT	17.0	7/15/02	3.1	7/15/02	28.0	7/15/02



# HBV LONGITUDINAL PANEL

(Catalog No. 6548)

For Investigational Research Use Only

**Application Notes:** This 12 member panel was collected from a 17 year old Caucasian Male.

PANEL NUMBER	CONCURRENT MEDICAL CONDITIONS	MEDICATIONS	ABBOTT HBSAG S/CO	ABBOTT HBC TOTAL S/CO	ABBOTT HBC IGM S/CO	DIASORIN HBEAG S/CO	DIASORIN ANTI-HBE S/CO	ABBOTT ANTI-HBS S/CO	ABBOTT ANTI-HIV 1/2 S/CO	ABBOTT HCV S/CO
6548-01	NONE	Diet N5/ Aqua Minerale/ Enterosgel/ No-Spa/ Mezini/ Riboxini	>36.4	0.04	3.6	0.18	0.2	0.3	0.13	0.2
6548-02	NONE	Diet N5/ Hepatic Tea/ Riboxini/ Aevit	1.0	0.05	5.1	0.16	0.3	0.3	N/A	N/A
6548-03	NONE	Diet N5	0.05	0.05	>5.5	0.10	0.2	0.4	N/A	N/A
6548-04	NONE	Diet N5	0.07	0.06	>5.5	0.12	0.1	0.5	N/A	N/A
6548-05	NONE	Diet N5	0.8	0.06	>5.5	0.04	0.08	1.04	N/A	N/A
6548-06	NONE	Diet N5	0.04	0.05	>5.5	0.12	0.08	1.5	N/A	N/A
6548-07	NONE	Diet N5	0.04	0.07	>5.5	0.04	0.07	1.4	N/A	N/A
6548-08	NONE	Diet N5	0.04	0.06	>5.5	0.08	0.07	1.9	N/A	N/A
6548-09	NONE	Diet N5	0.04	0.06	>5.5	0.04	0.07	2.5	N/A	N/A
6548-10	NONE	Diet N5	0.02	0.06	>5.5	0.10	0.05	1.4	N/A	N/A
6548-11	NONE	Diet N5	0.7	0.05	5.3	0.05	0.07	1.3	N/A	N/A
6548-12	NONE	Diet N5	0.05	0.07	>5.5	0.10	0.08	2.3	N/A	N/A