

Hepatitis B Seroconversion Panel

(Catalog No. HBV 11027)

Donor No. 71415

For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DONOR NUMBER	BLEED DATE	ALT Normal Range (10-45 u/L)	ROCHE HBV Monitor Assay (copies/mL)	ROCHE HBV Ampliscreen	VENDOR HBsAg	Genetic Systems HBsAg 2.0 Static	ABBOTT HBsAg Proc B EIA	ABBOTT HBsAg Proc C	ABBOTT PRISM HBsAg	ABBOTT Murex HBsAg Version 3	ORTHO HBsAg Proc B	ORTHO anti-HBc
11027-01	71415-01	3/16/99	3	Not Detected	NEG	NEG	NEG	0.038	0.172	0.45	0.52	0.033	0.040
11027-02	71415-02	3/21/99	14	Not Detected	NEG	NEG	NEG	0.038	0.069	0.48	0.55	0.033	0.044
11027-03	71415-03	3/28/99	11	<100	POS	NEG	NEG	0.057	0.069	0.48	0.51	0.000	0.033
11027-04	71415-04	3/30/99	12	Not Detected	NEG	NEG	NEG	0.094	0.034	0.43	0.50	0.000	0.033
11027-05	71415-05	4/4/99	7	100	POS	NEG	NEG	0.019	0.103	0.64	0.51	0.000	0.053
11027-06	71415-06	4/6/99	10	200	POS	NEG	NEG	0.019	0.241	0.65	0.53	0.033	0.035
11027-07	71415-07	4/11/99	9	200	POS	NEG	NEG	0.038	0.172	0.60	0.63	0.033	0.033
11027-08	71415-08	4/14/99	8	200	POS	NEG	NEG	0.038	0.138	0.73	0.71	0.033	0.044
11027-09	71415-09	4/18/99	12	500	POS	NEG	NEG	0.245	0.483	1.24	0.91	0.100	0.035
11027-10	71415-10	4/22/99	12	<100	POS	NEG	NEG	0.868	0.241	2.64	1.69	0.267	0.037
11027-11	71415-11	4/25/99	8	12,000	POS	NEG	NEG	4.528	1.138	6.11	3.55	0.667	0.042
11027-12	71415-12	5/4/99	44	4,600,000	POS	POS	POS	>37.736	75.862	351.69	33.44	104.333	0.026
11027-13	71415-13	5/19/99	12	45,000,000	POS	POS	POS	>37.736	75.862	415.88	33.05	93.433	0.026
11027-14	71415-14	5/23/99	4	80,000,000	POS	POS	POS	>37.736	75.862	551.58	33.35	82.767	0.048

All specimens tested for Anti-HIV1/, HIV P24, Anti HCV, HCV PCR, and found to be negative by an FDA cleared test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No preservatives have been added.