

Hepatitis B Seroconversion Panel

(Catalog No. HBV 11069)

Donor No. 73299

For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DONOR NUMBER	BLEED DATE	ALT Normal Range (10-45 u/L)	ROCHE HBV Monitor Assay (copies/mL)	ROCHE HBV Ampliscreen	Genetic Systems HBsAg 2.0 Static	ABBOTT HBsAg Proc B	ABBOTT HBsAg Proc C	ABBOTT PRISM HBsAg	ABBOTT Murex HBsAg Version 3	ORTHO anti-HBc	ORTHO HBsAg Proc B Rpt
11069-01	73299-01	7/10/99	9	Not Detected	NEG	NEG	0.019	0.222	0.73	0.58	0.094	Not Tested
11069-02	73299-02	7/12/99	5	Not Detected	NEG	NEG	0.038	0.148	0.77	0.58	0.067	Not Tested
11069-03	73299-03	7/28/99	10	Not Detected	NEG	NEG	0.096	0.185	0.57	0.58	0.053	Not Tested
11069-04	73299-04	8/11/99	16	Not Detected	NEG	NEG	0.058	0.148	0.59	0.63	0.060	Not Tested
11069-05	73299-05	8/23/99	10	200	POS	NEG	0.096	0.148	0.40	0.62	0.046	Not Tested
11069-06	73299-06	8/25/99	11	200	POS	NEG	0.038	0.185	0.56	0.65	0.007	Not Tested
11069-07	73299-07	8/30/99	15	200	POS	NEG	0.019	0.111	0.82	0.78	0.030	Not Tested
11069-08	73299-08	9/1/99	13	4,700	POS	NEG	0.096	0.148	0.92	0.76	0.053	Not Tested
11069-09	73299-09	9/6/99	11	2,600	POS	NEG	0.212	0.333	1.65	1.13	0.030	Not Tested
11069-10	73299-10	9/8/99	8	1,100	POS	NEG	0.365	0.407	2.26	1.45	0.060	Not Tested
11069-11	73299-11	9/13/99	11	7,600	POS	NEG	0.538	0.519	2.87	1.75	0.069	Not Tested
11069-12	73299-12	9/15/99	9	7,600	POS	NEG	0.635	0.667	3.12	1.95	0.085	Not Tested
11069-13	73299-13	9/29/99	19	35,000	POS	POS	3.519	3.037	18.03	9.11	0.119	2.200
11069-14	73299-14	10/2/99	13	48,000	POS	POS	5.923	4.667	27.94	16.47	0.108	Not Tested
11069-15	73299-15	10/6/99	10	36,000	POS	POS	10.577	8.000	42.67	23.95	0.138	Not Tested

PANEL MEMBER	SORIN HBeAg	SORIN HBeAb (Co/S)	SORIN HBsAg	GENETIC SYSTEMS HBsAg
11069-01	0.246	0.50	0.220	0.394
11069-02	0.130	0.50	0.283	0.472
11069-03	0.072	0.50	0.157	0.551
11069-04	0.174	0.50	0.220	0.315
11069-05	0.174	0.50	0.157	0.315
11069-06	0.174	0.52	0.220	0.315
11069-07	0.159	0.52	0.157	0.394
11069-08	0.188	0.53	0.220	0.394
11069-09	0.130	0.50	0.346	0.472
11069-10	0.087	0.50	0.377	0.709
11069-11	0.130	0.49	0.346	0.866
11069-12	0.145	0.52	0.283	0.709
11069-13	0.145	0.51	0.314	3.228
11069-14	0.159	0.55	0.314	4.252
11069-15	0.174	0.52	0.503	7.323

All specimens tested for Anti-HIV1/2, HIV P24, Anti HCV, HCV PCR, and found to be negative by an FDA cleared test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No preservatives have been added.