

Hepatitis B Seroconversion Panel

(Catalog No. HBV 9093)

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

Donor No. 1087380

For Research Use Only

PANEL NUMBER	DONOR NUMBER	BLEED NUMBER	BLEED DATE	HBsAg	DIASORIN HBsAg	DIASORIN HBsAb	DIASORIN Hbc Total	DIASORIN Hbc IgM
9093-01	1087380-01	FT04183484	6/28/2007	NEG	0.60	0.08	1.52	0.20
9093-02	1087380-02	FT04183773	7/3/2007	NEG	0.20	0.03	1.32	0.20
9093-03	1087380-03	FT04183638	7/5/2007	NEG	0.40	0.08	1.20	0.20
9093-04	1087380-04	FT04183953	7/10/2007	NEG	0.60	0.03	1.60	0.16
9093-05	1087380-05	FT04184086	7/12/2007	NEG	1.40	0.08	1.84	0.16
9093-06	1087380-06	FT04184277	7/17/2007	POS	5.60	0.02	2.52	0.16
9093-07	1087380-07	FT04184483	7/19/2007	POS	7.60	0.08	2.20	0.16
9093-08	1087380-08	FT04184654	7/24/2007	POS	31.40	0.02	1.80	0.16
9093-09	1087380-09	FT04185197	8/2/2007	POS	60.80	0.08	1.52	0.20
9093-10	1087380-10	FT04185457	8/9/2007	POS	60.40	0.08	0.68	0.20
9093-11	1087380-11	FT04185807	8/16/2007	POS	60.40	0.03	0.12	1.60
9093-12	1087380-12	FT04186144	8/23/2007	POS	60.20	0.03	0.16	1.16
9093-13	1087380-13	FT04186778	9/10/2007	POS	59.60	0.08	0.08	8.48
9093-14	1087380-14	FT04186903	9/12/2007	POS	60.20	0.03	0.12	7.88
9093-15	1087380-15	FT04188002	10/11/2007	POS	59.20	0.03	0.12	5.32
9093-16	1087380-16	FT04188078	10/15/2007	POS	59.80	0.03	0.04	5.28
9093-17	1087380-17	FT04188316	10/22/2007	POS	60.00	0.08	0.04	5.32
9093-18	1087380-18	FT04188579	10/29/2007	POS	60.40	0.08	0.04	2.24
9093-19	1087380-19	FT04188704	11/1/2007	POS	60.40	0.08	0.04	2.68
9093-20	1087380-20	FT04189171	11/14/2007	POS	61.60	0.08	0.04	1.24
9093-21	1087380-21	FT04189553	11/27/2007	POS	34.80	0.08	0.04	0.76
9093-22	1087380-22	FT04189684	11/29/2007	POS	15.00	0.08	0.04	0.76
9093-23	1087380-23	FT04189844	12/4/2007	POS	3.20	0.08	0.04	0.56
9093-24	1087380-24	FT04189984	12/6/2007	POS	2.00	0.08	0.08	0.64
9093-25	1087380-25	FT04190115	12/11/2007	NEG	14.40	0.08	0.08	0.60
9093-26	1087380-26	FT04190237	12/13/2007	NEG	0.60	0.08	0.08	0.44
9093-27	1087380-27	FT04190418	12/18/2007	NEG	0.80	0.08	0.08	0.44
9093-28	1087380-28	FT04190731	12/27/2007	NEG	0.60	0.08	0.08	0.44
9093-29	1087380-29	FT04190789	12/31/2007	NEG	0.80	0.08	0.12	0.36
9093-30	1087380-30	FT04190912	1/3/2008	NEG	0.20	0.08	0.08	0.32
9093-31	1087380-31	FT04191001	1/7/2008	NEG	0.20	0.08	0.12	0.40

Acute to Recovered

All specimens tested for Anti-HIV1/2 and Anti HCV are found to be negative by an FDA cleared test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No preservatives have been added.