



HCV Pre/Post Seroconversion Samples

(Catalog No. HCV10235)

Donor No. 66822

For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donors were un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DONOR NUMBER	BLEED DATE	BLEED NUMBER	VENDOR ALT	ZeptoMetrix HCV RT-PCR Copies/mL	VENDOR HBsAg	VENDOR Anti HIV 1/2	VENDOR HIV p24Ag	VENDOR Anti HCV	VENDOR HCV by PCR	ABBOTT Anti-HCV
10235-01	66822-01	1/9/98	N-16313-032	12	Not Detected	NEG	NEG	NEG	NEG	NEG	0.185
10235-02	66822-02	1/11/98	N-16326-032	12	Not Detected	NEG	NEG	NEG	NEG	NEG	0.222
10235-03	66822-03	4/15/98	G-23671-032	116	1,130,000	NEG	NEG	NEG	POS	POS	5.778
10235-04	66822-04	4/20/98	G-23958-032	144	1,810,000	NEG	NEG	NEG	POS	POS	7.074
10235-05	66822-05	4/22/98	G-24059-032	128	3,280,000	NEG	NEG	NEG	POS	POS	7.037

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.