

HCV Seroconversion Panel

(Catalog No. HCV 9095)

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

Donor No. 1038851

For Research Use Only

PANEL NUMBER	DONOR NUMBER	BLEED NUMBER	BLEED DATE	HCV	DIASORIN HBsAb	QUEST HCV Ab	ROCHE HCV Ampliscreen	ROCHE HCV Monitor
9095-01	1038851-01	FT12104742	6/3/2006	NEG	NOT TESTED	0.04	POS	159,000
9095-02	1038851-02	FT12104842	6/6/2006	NEG	NOT TESTED	0.04	POS	54,000
9095-03	1038851-03	FT12105177	6/13/2006	NEG	NOT TESTED	11.70	POS	141,000
9095-04	1038851-04	FT12105753	6/24/2006	POS	NOT TESTED	18.10	POS	17,500
9095-05	1038851-05	FT12105863	6/27/2006	POS	0.08	18.40	POS	42,600

All specimens tested for HBsAg and Anti HIV are found to be negative by an FDA cleared test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No preservatives have been added.