



HIV Pre/Post Seroconversion Samples

(Catalog No. HIV10234)

Donor No. 67101

For Research Use Only

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donors were un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DONOR NUMBER	BLEED DATE	BLEED NUMBER	VENDOR ALT	VENDOR HBsAg	VENDOR Anti HIV 1/2	VENDOR HIV p24Ag	VENDOR Anti HCV	VENDOR HCV by PCR	ABBOTT HIV Ab HIV ½ EIA	ABBOTT HIV Ag-1 Monoclonal
10234-01	67101-01	3/30/98	G-56125-118	9	NEG	NEG	NEG	NEG	NEG	0.129	0.102
10234-02	67101-02	4/02/98	G-56300-118	10	NEG	NEG	NEG	NEG	NEG	0.138	0.220
10234-03	67101-03	5/07/98	G-58034-118	75	NEG	NEG	REACT	NEG	NEG	13.448	11.288

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA approved test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.