

# HIV Seroconversion Panel

(Catalog No. HIV 12008)

## Donor No. 73698

For Research Use Only

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

New 3<sup>rd</sup> gen

PANEL MEMBER	BLEED DATE	GEN-PROBE PROCLEIX HIV-1 /HCV ASSAY	ROCHE COBAS AMPLICOR HIV-1 MONITOR	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ZEPTOMETRIX HIV p24 Ag	ABBOTT HIV AB 1/2 EIA AxSYM	ABBOTT HIV AB 1/2 EIA Rdna	ABBOTT HIV 1/2 III PLUS IMX	ABBOTT MUREX HIV Ab 1/2 + O	ABBOTT PRISIM ANTI HIV 1/2
12008-01	9/25/99	NEG	NEG	0.276	0.213	<3.0	0.37	0.197	0.27	0.440	0.26
12008-02	9/27/99	NEG	NEG	0.328	0.160	<3.0	0.36	0.181	0.25	0.430	0.32
12008-03	10/2/99	NEG	NEG	0.241	0.191	<3.0	0.36	0.165	0.30	0.375	0.26
12008-04	10/4/99	NEG	>400	0.293	0.181	<3.0	0.37	0.173	0.30	0.368	0.30
12008-05	10/9/99	NEG	NEG	0.259	0.191	<3.0	0.36	0.213	0.28	0.455	0.29
12008-06	10/11/99	NEG	NEG	0.293	0.181	<3.0	0.39	0.173	0.28	0.464	0.32
12008-07	10/16/99	POS POS	2,000	0.259	0.245	<3.0	0.36	0.181	0.26	0.409	0.30
12008-08	10/18/99	POS	44,700	0.517	0.511	<3.0	0.39	0.157	0.28	0.443	0.33
12008-09	10/23/99	POS	>750,000	14.293	28.723	>125	0.38	0.173	0.33	0.455	0.33
12008-10	10/28/99	POS	>750,000	34.483	34.043	>125	2.39	1.795	1.47	2.214	1.77
12008-11	10/30/99	POS	>750,000	9.655	34.043	122.3	12.66	9.528	8.42	10.700	17.93
12008-12	11/4/99	POS	143,000	0.810	2.053	<3.0	11.65	6.063	8.55	10.805	26.58
12008-13	11/6/99	POS	NEG	0.483	0.638	<3.0	12.32	4.646	7.11	10.854	23.25

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA cleared test.

**Notes on Data:** Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.

# HIV Seroconversion Panel

(Catalog No. HIV 12008)

**Donor No. 73698**

For Research Use Only

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOMERIEUX VIRONOSTIKA ANTI HIV-1	BIO-RAD ANTI HIV-1 rLAV	BIOTEST ANTI HIV 1/2	BOEHRINGER MANNHEIM ENZYMNUTEST ANTI HIV 1/2 PLUS SUBTYPE O	DADE BEHRING ENZYGNOST ANTI HIV 1/2 PLUS	MEDMIRA ANTI HIV-1 RAPID TEST	ORAQUICK ADVANCE HIV 1/2 RAPID AbAb	ORTHO ANTI HIV 1/2 ENHANCED ELISA	TRINITY BIO-TECH UNI GOLD RECOMBIGEN HIV-1	BIORAD MULTIPOT HIV 1/2 RAPID TEST R1	BIORAD MULTIPOT HIV 1/2 RAPID TEST P1	BIORAD MULTIPOT HIV 1/2 RAPID TEST P2
12008-01	0.427	0.102	0.14	0.04	0.03	NEG	NEG	0.00	NEG	NEG	NEG	NEG
12008-02	0.465	0.084	0.11	0.04	0.03	NEG	NEG	0.00	NEG	NEG	NEG	NEG
12008-03	0.344	0.084	0.22	0.04	0.03	NEG	NEG	0.01	NEG	NEG	NEG	NEG
12008-04	0.401	0.080	0.17	0.04	0.03	NEG	NEG	0.03	NEG	NEG	NEG	NEG
12008-05	0.439	0.084	0.12	0.03	0.03	NEG	NEG	0.02	NEG	NEG	NEG	NEG
12008-06	0.479	0.065	0.14	0.03	0.03	NEG	NEG	0.01	NEG	NEG	NEG	NEG
12008-07	0.382	0.065	0.12	0.04	0.03	NEG	NEG	0.00	NEG	NEG	NEG	NEG
12008-08	0.342	0.076	0.34	0.04	0.03	NEG	NEG	0.00	NEG	NEG	NEG	NEG
12008-09	0.341	0.062	0.09	0.04	<b>1.18</b>	NEG	NEG	0.00	NEG	NEG	NEG	NEG
12008-10	0.385	0.087	<b>1.59</b>	<b>1.09</b>	<b>4.10</b>	NEG	NEG	<b>2.67</b>	NEG	NEG	NEG	NEG
12008-11	0.371	0.422	<b>4.51</b>	<b>8.50</b>	<b>5.30</b>	<b>POS</b>	NEG	<b>10.75</b>	<b>POS</b>	NEG	NEG	NEG
12008-12	0.633	<b>2.025</b>	<b>3.56</b>	<b>7.75</b>	<b>6.98</b>	<b>POS</b>	<b>POS</b>	<b>12.09</b>	<b>POS</b>	<b>POS</b>	<b>POS</b>	NEG
12008-13	0.713	<b>1.713</b>	<b>5.30</b>	<b>7.45</b>	<b>7.06</b>	<b>POS</b>	<b>POS</b>	<b>11.16</b>	<b>POS</b>	<b>POS</b>	<b>POS</b>	NEG

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA cleared test.

**Notes on Data:** Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.

# HIV Seroconversion Panel

(Catalog No. HIV 12008)

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

**Donor No. 73698**

For Research Use Only

New 4<sup>th</sup> Gen

PANEL MEMBER	BIO-RAD HIV-1 WESTERN BLOT	ABBOTT PRISM HIV Ag/Ab Combo	ABBOTT MUREX MICROPLATE HIV Ag/Ab COMBO	ABBOTT ARCHITECT HIV Ag/Ab COMBO
12008-01	Not Tested	0.10	0.275	0.08
12008-02	Not Tested	0.20	0.377	0.14
12008-03	Not Tested	0.09	0.338	0.09
12008-04	Not Tested	0.43	0.270	0.11
12008-05	Not Tested	0.14	0.353	0.10
12008-06	Not Tested	0.10	0.299	0.10
12008-07	Not Tested	0.10	0.279	0.13
12008-08	NEG	0.37	0.598	0.53
12008-09	NEG	<b>29.45</b>	<b>14.417</b>	<b>50.26</b>
12008-10	NEG	<b>104.44</b>	<b>17.525</b>	<b>167.70</b>
12008-11	<b>Gp160.p55+/-,p4</b>	<b>39.76</b>	<b>17.534</b>	<b>53.66</b>
12008-12	<b>Gp160.p120+/-</b>	<b>4.23</b>	<b>17.127</b>	<b>28.35</b>
12008-13	<b>Gp160.p120+/-</b>	<b>3.29</b>	<b>16.990</b>	<b>30.20</b>

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA cleared test.

**Notes on Data:** Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No Preservatives have been added.