

HIV Seroconversion Panel

(Catalog No. HIV 9096)

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

Donor No. 1043487

For Research Use Only

PANEL NUMBER	DONOR NUMBER	BLEED NUMBER	BLEED DATE	HIV 1/2	ABBOTT HIV 1/2	DIASORIN HBsAb
9096-01	1043487-01	0460005217	11/26/2006	NEG	0.17	NOT TESTED
9096-02	1043487-02	0460005553	11/29/2006	NEG	0.25	NOT TESTED
9096-03	1043487-03	0460005778	12/3/2006	NEG	0.17	NOT TESTED
9096-04	1043487-04	0460006058	12/6/2006	NEG	0.33	NOT TESTED
9096-05	1043487-05	0460006774	12/14/2006	POS	10.50	NOT TESTED
9096-06	1043487-06	0460006943	12/17/2006	POS	5.42	0.08

PANEL NUMBER	CAM BIO TECH HIV WESTERN BLOT	TRINITY UNIGOLD	BIORAD HIV MULTISPOT	CHMBIO HIV STAT PAK	CHEMBIO HIV SURE CHECK	GENETICS SYSTEMS HIV rLAV	GENETICS SYSTEMS HIV1/2 PLUS O	ROCHE HIV MONITOR
9096-01	0	0	0	0	0	0.15	0.11	934
9096-02	0	0	0	0	0	0.08	0.15	40,500
9096-03	0	0	0	0	0	0.04	0.15	1,370,000
9096-04	0	0	0	0	0	0.08	0.15	1,660,000
9096-05	+/-	+1	+2	+1	+1	4.5	11.44	334,000
9096-06	+2	+1	+2	+2	+2	5.04	11.44	187,000

All specimens tested for HBsAg and Anti-HCV and found to be negative by an FDA cleared test.

Notes on Data: Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive. This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this longitudinal series are naturally occurring, undiluted, unprocessed samples collected from a single plasma donor in the United States. No preservatives have been added.