



# Hepatitis E Mixed Titer Panel

(Catalog No.) K-ZMC003

**LOT NUMBER: 0802-272-00032**  
For Research Use Only

PANEL MEMBER	LOT NUMBER	**ABBOTT AUZYME (HBsAg) (S/Co)	**ABBOTT AUSAB (Anti-HBs) (S/Co)	*ABBOTT CORZYME (HBc TOTAL) (S/Co)	**ABBOTT COR-M rDNA (HBc IgM) (S/Co)	**ABBOTT HIV 1/2 (S/Co)	**ABBOTT HCV 2.0 (S/Co)	FISHER SUREVUE RPR	**GENELABS HEV IgG (S/Co)	**GENELABS HEV IgM (S/Co)
1	9612-040-5652	0.10	1.8	2.6	0.05	0.2	0.3	Non Reactive	4.8	0.04
2	9612-040-5654	0.20	2.2	2.6	0.05	0.2	0.3	Non Reactive	4.1	0.40
3	9612-040-5658	0.04	4.2	2.8	0.05	0.2	0.2	Non Reactive	2.7	0.09
4	9612-040-5660	0.04	0.1	2.2	0.10	0.3	0.2	Non Reactive	2.0	0.20
5	9612-040-5661	0.04	0.5	2.2	0.05	0.2	0.2	Non Reactive	2.6	0.20
6	9612-040-5665	0.02	0.1	2.2	0.05	0.2	0.5	Non Reactive	1.4	0.40
7	9612-040-5671	0.10	0.1	2.5	0.05	0.2	0.3	Non Reactive	5.5	0.02
8	9612-040-5672	0.10	0.1	2.5	0.05	0.2	0.3	Non Reactive	5.7	0.40
9	9612-040-5675	0.05	0.3	2.4	0.05	0.4	0.2	Non Reactive	3.1	0.04
10	9612-040-5677	0.10	0.4	1.9	0.09	0.2	0.2	Non Reactive	1.3	0.09
11	9612-040-5678	0.04	0.4	2.1	0.05	0.2	0.3	Non Reactive	2.1	0.09
12	9612-040-5679	0.02	0.3	2.1	0.05	0.4	0.3	Non Reactive	1.9	0.30
13	9612-040-5680	0.02	1.7	2.4	0.05	0.2	0.4	Non Reactive	2.0	0.50
14	9612-040-5681	0.02	4.7	2.5	0.05	0.2	0.2	Non Reactive	3.1	0.04
15	9612-040-5682	0.04	0.1	1.9	0.05	0.2	0.3	Non Reactive	1.7	0.20
16	9612-040-5683	0.02	0.1	2.1	0.30	0.2	0.3	Non Reactive	2.1	0.02
17	9612-040-5684	0.04	0.1	2.4	0.05	0.2	0.2	Non Reactive	2.9	0.02
18	9612-040-5685	0.02	0.1	2.1	0.05	0.2	0.2	Non Reactive	3.9	0.20
19	9612-040-5686	0.04	0.1	2.4	0.09	0.2	0.3	Non Reactive	5.1	0.10
20	9612-040-5687	0.05	0.1	2.1	0.05	0.2	0.3	Non Reactive	2.6	0.30
21	9612-040-5688	0.07	0.3	2.2	0.05	0.2	0.3	Non Reactive	3.7	0.20
22	9612-040-5690	0.07	0.1	2.5	0.05	0.2	0.2	Non Reactive	3.5	0.09
23	9612-040-5691	0.04	0.1	2.5	0.05	0.2	0.3	Non Reactive	5.3	0.02
24	9612-040-5692	0.04	0.7	2.6	0.05	0.2	0.2	Non Reactive	1.3	0.09

\*S/Co >1.0 is considered negative    S/Co ≤ 1.0 is considered positive

\*\* S/Co ≤ 1.0 is considered negative    S/Co > 1.0 is considered positive

**Notes on Data:** This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data become available to us. Samples are in unadulterated Sodium Citrated plasma collected in US licensed blood banks from various locations around the country. No preservatives or stabilizers have been added.