



ToRCH MIXED TITER PANEL
(Catalog No.) K-ZMC004

LOT NUMBER 0803-272-00035
For Research Use Only

MEMBER #	BCP LOT NO	BLEED NO	ALT	VENDOR HBsAG	VENDOR HIV-1 p24 Ag	VENDOR ANTI HIV 1/2	VENDOR ANTI HCV	FOCUS HERPE SELECT IgG (S/CO)	FOCUS HERPE SELECT 2 IgG(S/CO)	ZEUS HSV-1+2 IgM(S/CO)	WAMPOLE FISHER RUBELLA IgG ELISA (S/CO)	WAMPOLE FISHER RUBELLA IgM ELISA (S/CO)	TRINITY CAPTIA TOXO IgG ELISA (S/CO)	TRINITY CAPTIA TOXO IgM ELISA (S/CO)
1	0011-037-01633	N-81884-075	11	NEG	NEG	NEG	NEG	0.8	0.1	0.2	0.8	0.2	0.3	0.1
2	0011-037-01664	N-93139-099	0	NEG	NEG	NEG	NEG	3.3	6.1	1.02	2.3	0.7	0.3	2.6
3	0011-037-01678	N-72712-001	3	NEG	NEG	NEG	NEG	0.1	5.5	0.4	4.2	0.3	0.2	0.5
4	0011-037-01705	G-23942-432	14	NEG	NEG	NEG	NEG	3.7	0.1	0.2	3.7	0.1	4.5	0.6
5	0011-037-01718	N-06412-198	8	NEG	NEG	NEG	NEG	0.2	3.9	1.0	0.9	0.3	0.2	1.7
6	0011-037-01724	N-67720-152	4	NEG	NEG	NEG	NEG	4.7	0.2	0.8	2.2	0.1	0.7	0.8
7	0011-037-01727	N-30251-035	12	NEG	NEG	NEG	NEG	4.9	0.1	1.6	3.3	0.3	0.6	2.7
8	0011-037-01728	N-30268-035	12	NEG	NEG	NEG	NEG	5.3	0.2	1.4	3.5	0.3	0.5	2.6
9	0011-037-01729	N-30271-035	7	NEG	NEG	NEG	NEG	4.9	5.2	0.5	2.5	1.2	0.2	0.5
10	0011-037-01733	N-53006-083	12	NEG	NEG	NEG	NEG	5.3	0.1	0.6	4.7	0.1	0.2	0.3
11	0011-037-01790	N-19158-032	12	NEG	NEG	NEG	NEG	3.7	0.1	1.5	2.7	1.3	4.2	3.9
12	0011-037-01795	N-21216-118	8	NEG	NEG	NEG	NEG	0.2	4.2	0.7	4.0	0.1	1.5	0.2
13	0011-037-01811	G-27006-083	17	NEG	NEG	NEG	NEG	4.2	0.1	0.05	4.2	0.07	3.1	0.2
14	0011-037-01818	G-94221-035	8	NEG	NEG	NEG	NEG	0.2	5.5	0.4	4.0	0.1	0.2	0.3
15	0011-037-01838	G-94499-072	0	NEG	NEG	NEG	NEG	2.3	0.6	0.3	1.09	0.05	2.5	0.95
16	0011-037-01839	G-26732-160	6	NEG	NEG	NEG	NEG	5.8	0.2	1.7	3.2	0.4	0.4	3.5
17	0011-037-01901	G-63344-039	6	NEG	NEG	NEG	NEG	0.1	0.1	0.2	0.7	0.3	0.1	0.6
18	0011-037-01929	G-34682-156	6	NEG	NEG	NEG	NEG	4.7	4.8	0.5	3.5	0.1	0.1	0.07
19	0011-037-01933	N-50903-077	14	NEG	NEG	NEG	NEG	0.1	0.1	0.1	0.6	0.1	0.1	0.8
20	0011-037-01936	G-71402-039	4	NEG	NEG	NEG	NEG	1.7	2.1	0.2	0.9	0.1	0.2	1.3
21	0011-037-01938	G-76487-076	8	NEG	NEG	NEG	NEG	0.2	0.1	0.7	4.7	0.3	0.4	1.13
22	0011-037-01960	T-17015-097	11	NEG	NEG	NEG	NEG	0.5	5.5	1.04	0.93	0.3	0.2	0.95
23	0011-037-02029	G-43703-139	4	NEG	NEG	NEG	NEG	0.3	7.3	0.5	2.7	0.3	0.5	0.4
24	0011-037-02057	M-08739-018	5	NEG	NEG	NEG	NEG	0.1	0.91	0.2	4.0	0.1	0.1	0.3

Data is presented as an S/Co ratio unless stated otherwise. Ratios ≥ 1.0 are considered to be reactive.

Notes on Data: This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. Samples are in unadulterated Sodium Citrated plasma collected in US licensed blood banks from various locations around the country. No preservatives or stabilizers have been added.



ToRCH MIXED TITER PANEL
(Catalog No.) K-ZMC004

LOT NUMBER 0803-272-00035
For Research Use Only

MEMBER #	BCP LOT NO	BLEED NO	TRINITY CAPTIA CMV IgG ELISA (S/Co)	TRINITY CAPTIA CMV IgM ELISA (S/Co)	TRINITY CAPTIA VZV IgG (S/Co)
1	0011-037-01633	N-81884-075	0.3	0.2	4.0
2	0011-037-01664	N-93139-099	2.4	0.7	3.1
3	0011-037-01678	N-72712-001	2.7	1.5	3.8
4	0011-037-01705	G-23942-432	2.4	0.4	3.1
5	0011-037-01718	N-06412-198	2.8	1.6	3.1
6	0011-037-01724	N-67720-152	2.8	0.6	3.2
7	0011-037-01727	N-30251-035	2.8	0.8	4.7
8	0011-037-01728	N-30268-035	2.8	0.6	4.9
9	0011-037-01729	N-30271-035	2.8	0.4	1.8
10	0011-037-01733	N-53006-083	2.8	0.5	1.3
11	0011-037-01790	N-19158-032	0.2	0.8	3.8
12	0011-037-01795	N-21216-118	3.2	0.02	2.3
13	0011-037-01811	G-27006-083	1.9	0.01	3.1
14	0011-037-01818	G-94221-035	2.8	0.3	1.08
15	0011-037-01838	G-94499-072	2.8	0.1	1.4
16	0011-037-01839	G-26732-160	0.3	0.5	2.7
17	0011-037-01901	G-63344-039	0.1	0.2	1.08
18	0011-037-01929	G-34682-156	2.7	0.3	1.08
19	0011-037-01933	N-50903-077	0.2	0.1	2.5
20	0011-037-01936	G-71402-039	1.9	0.4	3.4
21	0011-037-01938	G-76487-076	0.1	0.5	2.2
22	0011-037-01960	T-17015-097	3.3	0.4	3.6
23	0011-037-02029	G-43703-139	2.1	0.3	5.9
24	0011-037-02057	M-08739-018	2.1	0.3	0.3

Data is presented as an S/Co ratio unless stated otherwise. Ratios ≥ 1.0 are considered to be reactive.

Notes on Data: This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. Samples are in unadulterated Sodium Citrated plasma collected in US licensed blood banks from various locations around the country. No preservatives or stabilizers have been added.