



GLOBAL HIV SURVEILLANCE PANEL

(Catalog No. K-GSP101)

LOT NUMBER: 0611-272-00005

For Research Use Only

MEMBER ID #	ORIGIN	DONOR REACTIVITY	ZEPTOMETRIX HIV-1 p24 Ag ELISA	ABBOTT HIV 1/2 Ab	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT	BIORAD MULTISPOT HIV 1	BIORAD MULTISPOT HIV 2	CHEMBIO HIV 1/2 STAT-PAK	CHEMBIO HIV 1/2 SURE CHECK	EFOORA HIV-1/2 RAPID TEST	ORASURE ORAQUICK ADVANCE HIV 1/2	TRINITY UNI-GOLD RECOMBIGN HIV-1	SALIVA DIAGNOSTICS HEMA STRIP HIV-1/2
GSP101-01	Ghana	HIV-1	0.3	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-02	Cameroon	HIV-1	0.3	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-03	Nigeria	HIV-1/2	0.4	>18	POS	POS	POS	POS	POS	POS	POS	POS	POS
GSP101-04	Uganda	HIV-1	0.3	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-05	USA	HIV-1	0.3	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-06	USA	NORMAL	0.4	0.2	NEG	NEG	NEG	NEG	NEG	NEG	NEG	NEG	NEG
GSP101-07	Thailand	HIV-1	0.4	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-08	Uganda	HIV-1	0.6	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-09	Thailand	HIV-1	0.4	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-10	Ghana	HIV-1	0.4	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-11	Cameroon	HIV-1	0.4	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-12	HIV-2	HIV-2	0.3	6.8	POS	POS	POS	POS	POS	POS	POS	POS	POS
GSP101-13	Thailand	HIV-1	0.2	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-14	USA	NORMAL	0.4	0.2	NEG	NEG	NEG	NEG	NEG	NEG	NEG	NEG	NEG
GSP101-15	Thailand	HIV-1	0.5	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-16	Uganda	HIV-1	0.5	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-17	Cameroon	HIV-1	0.5	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-18	Ghana	HIV-1	0.3	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-19	India	HIV-1	0.8	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-20	Cameroon	HIV-1	0.4	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-21	Uganda	HIV-1	0.2	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-22	USA	HIV-1	0.2	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS
GSP101-23	USA	NORMAL	0.4	0.14	NEG	NEG	NEG	NEG	NEG	NEG	NEG	NEG	NEG
GSP101-24	USA	HIV-1	0.6	>18	POS	POS	NEG	POS	POS	POS	POS	POS	POS

Run Date	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06	16-Nov-06
Kit Lot #	302913	44136M200	B7129075	941734	941734	HIV061606	6303011	23681	806711	P17817	M6166		
Exp. Date	22 AUG 07	27-Dec-06	09 MAY 07	7-Aug-08	7-Aug-08	29-May-08	30-Nov-06	N/A	Feb 2007	22-Jun-07	Nov 2007		

Data is presented as an S/Co ratio unless stated otherwise. Ratios >1.0 are considered to be reactive.

Notes on Data: This panel is for investigational use only and the data is offered for informational purposes only. Data is generated at ZeptoMetrix Corporation and by our Lab Partners in the United States and Europe. We will endeavor to update our data sheets whenever new test data becomes available to us. All specimens collected in this series are naturally occurring, unprocessed samples collected from HIV infected donors worldwide. No preservatives have been added.