

**PRODUCT DESCRIPTION:**

Each aliquot contains 1 µg of DNA extracted from a pure culture of *Saccharomyces cerevisiae*. A gene specific to *Pneumocystis jiroveci* was inserted into the *S. cerevisiae* genome using standard recombinant techniques. *S. cerevisiae* was confirmed by rDNA sequencing. The insert was detected with a specific in-house real-time PCR assay. The purity of the culture was monitored by additional culturing and Gram staining to detect any contaminating bacteria. The DNA was extracted from the cells following a protocol based on the yeast protocol provided in the Qiagen® Genomic DNA Handbook and using Qiagen® Genomic DNA Buffers with a 500/G genomic tip. DNA concentration and A<sub>260/280</sub> ratios are determined using a NanoDrop®. The extracted DNA also tested positive on an in-house real time PCR assay.

**INTENDED USE:**

Purified Genomic DNA is designed for use as an amplification and/or detection control for nucleic acid testing of *P. jiroveci*. It can also be used to determine a limit of detection (LOD), in diagnostic assay development, cross-reactivity studies or genomic sequencing. Controls should be run using the same protocols as those used to amplify extracted clinical specimens.

**PRECAUTIONS:**

- Use Universal Precautions when handling Genomic DNA.
- The material may be re-frozen after thawing. Repetitive freezing and thawing is not recommended (aliquot material if necessary).
- To avoid cross-contamination, use separate pipette tips for all reagents.

**RECOMMENDED STORAGE:**

This control is supplied in TE Buffer and should be frozen at -20°C or below.




**DO NOT USE IN HUMANS. FOR RESEARCH USE ONLY. NOT FOR USE IN DIAGNOSTIC PROCEDURES.**

These products are intended for research, product development or manufacturing use only. These products are NOT intended for use in the manufacture or processing of injectable products subject to licensure under section 351 of the Public Health Service Act or for any other product intended for administration to humans.

© Registered trademarks are the property of their respective owners.

This control is intended to only be used with an assay from Luminex Molecular Diagnostics.

This product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

REF	Catalog Number		Temperature Limitation
LOT	Lot Number		Expiration Date
RUO	For Research Use Only		Biological Risk

**ZeptoMetrix Corporation**  
 878 Main Street  
 Buffalo, NY 14202  
 800-274-5487  
 www.zeptometrix.com