



**HBV Longitudinal Panel
Product Code: HBV6502**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

***Donor is co-infected with Hepatitis C.**

| PANEL NUMBER | DATE OF DRAW | ORTHO VITROS ECi ANTI-HIV ½ | ORTHO VITROS ECi ANTI-HCV | ABBOTT HBsAg | ORTHO VITROS ECi HBsAg | ABBOTT ANTI-HBc TOTAL | ORTHO VITROS ECi ANTI-HBc TOTAL | ABBOTT ANTI-HBc IgM |
|--------------|--------------|-----------------------------|---------------------------|----------------|------------------------|-----------------------|---------------------------------|---------------------|
| | | TEST DATE: 8/4/2014 | TEST DATE: 8/4/2014 | TEST DATE: N/A | TEST DATE: 8/4/2014 | TEST DATE: N/A | TEST DATE: 8/4/2014 | TEST DATE: N/A |
| 6502-01 | 03/22/01 | 0.07 | 35.5 | >40.8 | 4380 | 0.06 | 0.05 | 1.3 |
| 6502-02 | 04/05/01 | 0.08 | 35.2 | >40.8 | 1540 | 0.06 | 0.04 | 1.4 |
| 6502-03 | 04/19/01 | 0.09 | 35.2 | >40.8 | 441 | 0.07 | 0.05 | 1.2 |
| 6502-04 | 05/03/01 | 0.09 | 35.0 | >40.8 | 75.0 | 0.07 | 0.05 | 0.9 |
| 6502-05 | 05/17/01 | 0.09 | 34.9 | 20.5 | 21.3 | 0.07 | 0.05 | 0.9 |
| 6502-06 | 05/31/01 | 0.09 | 34.6 | 8.2 | 8.10 | 0.06 | 0.06 | 0.9 |
| 6502-07 | 06/14/01 | 0.09 | 34.4 | 4.0 | 2.10 | 0.10 | 0.05 | 0.8 |
| 6502-08 | 06/27/01 | 0.09 | 34.3 | 1.7 | 0.85 | 0.08 | 0.06 | 0.8 |
| 6502-09 | 07/11/01 | 0.09 | 33.7 | 1.4 | 0.43 | 0.08 | 0.06 | 0.8 |
| 6502-10 | 07/25/01 | 0.08 | 33.7 | 1.57 | 0.25 | 0.04 | 0.04 | 0.89 |
| 6502-11 | 08/09/01 | 0.09 | 33.0 | 1.74 | 0.17 | 0.05 | 0.04 | 0.81 |
| 6502-12 | 08/22/01 | 0.08 | 32.6 | 1.68 | 0.19 | 0.03 | 0.06 | 0.88 |
| 6502-13 | 11/26/01 | 0.09 | 30.7 | 0.08 | 0.09 | 0.08 | 0.04 | 0.71 |

| PANEL NUMBER | ORTHO VITROS ECi ANTI-HBc IgM | DIASORIN ANTI-HBe | ORTHO VITROS ECi ANTI-HBe | ABBOTT ANTI-HBs QUANT | ORTHO VITROS ECi ANTI-HBs QUANT | ORTHO VITROS ECi HBeAg | DIASORIN HBeAg |
|--------------|-------------------------------|-------------------|---------------------------|-----------------------|---------------------------------|------------------------|----------------|
| | TEST DATE: 8/4/2014 | TEST DATE: N/A | TEST DATE: 8/4/2014 | TEST DATE: N/A | TEST DATE: 8/4/2014 | TEST DATE: 8/4/2014 | TEST DATE: N/A |
| 6502-01 | 0.71 | 0.8 | 3.82 | 0.3 | 2.8 | 0.12 | 0.2 |
| 6502-02 | 0.71 | 0.7 | 4.72 | 0.3 | 1.7 | 0.15 | 0.2 |
| 6502-03 | 0.68 | 0.5 | 9.08 | 0.3 | 1.2 | 0.13 | 0.2 |
| 6502-04 | 0.51 | 0.5 | 9.68 | 0.3 | 1.8 | 0.12 | 0.2 |
| 6502-05 | 0.46 | 0.4 | 11.2 | 0.2 | 1.4 | 0.14 | 0.2 |
| 6502-06 | 0.44 | 0.3 | 14.0 | 0.4 | 3.2 | 0.14 | 0.3 |
| 6502-07 | 0.37 | 0.4 | 12.9 | 0.3 | 2.1 | 0.12 | 0.2 |
| 6502-08 | 0.36 | 0.3 | 14.0 | 0.3 | 2.0 | 0.12 | 0.2 |
| 6502-09 | 0.32 | 0.3 | 11.5 | 0.3 | 1.9 | 0.13 | 0.3 |
| 6502-10 | 0.30 | 0.1 | 11.2 | 0.35 | 1.6 | 0.13 | 0.3 |
| 6502-11 | 0.30 | 0.1 | 7.77 | 0.29 | 2.3 | 0.14 | 0.2 |
| 6502-12 | 0.30 | 0.1 | 9.37 | 0.37 | 1.6 | 0.12 | 0.3 |
| 6502-13 | 0.25 | 0.1 | 3.56 | 0.31 | 1.1 | 0.13 | 0.2 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com