



**HBV Longitudinal Panel
Part Number: HBV6513**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

***Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ABBOTT ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM	DIASORIN ANTI-HBe	ABBOTT ANTI-HBs QUANT	DIASORIN HBeAg
		TEST DATE: 8/8/2014	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6513-01	06/05/01	0.09	25.3	>40.8	0.05	0.70	0.1	0.2	0.1
6513-02	06/19/01	0.07	24.6	>40.8	0.06	0.80	1.7	0.3	5.9
6513-03	07/04/01	0.08	22.9	>40.8	0.10	0.60	1.9	0.3	12.9
6513-04	07/17/01	0.08	22.9	>36.4	0.02	0.80	4.8	0.4	39.0
6513-05	07/31/01	0.10	21.6	>36.4	0.03	0.98	4.8	0.5	39.1
6513-06	08/14/01	0.08	20.5	>36.4	0.02	1.05	4.8	0.4	38.0
6513-07	08/28/01	0.09	19.1	>36.4	0.01	1.13	4.5	0.5	33.0
6513-08	09/11/01	0.09	17.2	>36.4	0.01	1.03	3.2	0.2	26.2
6513-09	09/25/01	0.10	17.3	>36.4	0.01	0.90	0.7	0.4	2.6
6513-10	10/09/01	0.08	16.0	>36.4	0.03	0.84	0.4	0.3	1.0
6513-11	10/23/01	0.08	17.3	>36.4	0.02	0.99	0.3	0.4	1.5
6513-12	11/06/01	0.08	19.6	>36.4	0.03	0.90	0.01	0.3	0.2

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6513 Rev03
Effective Date: 09/01/2021