



**HBV Longitudinal Panel
Part Number: HBV6546**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures .*

***Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECI ANTI-HIV ½	ORTHO VITROS ECI ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECI HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECI ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK
6546-01	01/28/02	0.11	20.9	>36.4	7790	0.1	0.15	>5.5
6546-02	02/11/02	0.12	22.3	>36.4	7490	0.1	0.03	>5.5
6546-03	02/25/02	0.1	24.9	>36.4	2090	0.1	0.03	4.1
6546-04	03/11/02	0.11	27.7	>36.4	177	0.1	0.03	2.7
6546-05	03/25/02	0.10	28.5	1.9	0.27	0.1	0.03	1.9
6546-06	04/08/02	0.17	27.3	0.3	0.07	0.1	0.03	1.4
6546-07	04/22/02	0.14	26.2	0.2	0.06	0.1	0.04	1.04
6546-08	05/06/02	0.13	24.9	0.1	0.05	0.1	0.02	1.2
6546-09	05/20/02	0.12	23.3	0.04	0.08	0.1	0.03	0.8
6546-10	06/03/02	0.12	23.5	0.07	0.08	0.1	0.02	0.9
6546-11	06/17/02	0.13	21.9	0.02	0.07	0.1	0.04	0.6
6546-12	07/01/02	0.12	20.7	0.04	0.06	0.1	0.02	0.5

PANEL NUMBER	ORTHO VITROS ECI ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECI ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECI ANTI-HBs QUANT	ORTHO VITROS ECI HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK
6546-01	3.64	0.2	0.90	0.3	1.6	0.15	0.07
6546-02	3.19	0.09	3.86	0.4	2.1	0.14	0.10
6546-03	2.23	0.05	5.86	0.3	1.9	0.14	0.04
6546-04	1.56	0.05	QNS	0.3	2.7	0.13	0.15
6546-05	0.98	0.04	19.7	0.5	3.1	0.13	0.07
6546-06	0.77	0.05	25.7	0.6	1.8	0.12	0.12
6546-07	0.56	0.02	28.1	0.8	3.0	0.12	0.08
6546-08	0.49	0.03	30.4	1.12	6.3	0.13	0.11
6546-09	0.37	0.01	27.8	1.11	7.6	0.16	0.01
6546-10	0.37	0.01	30.6	1.5	11.5	0.15	0.11
6546-11	0.28	0.01	25.8	1.4	12.0	0.17	0.05
6546-12	0.28	0.02	28.3	1.0	13.6	0.14	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6546 Rev02
Effective Date: 09/22/2021