



HBV Seroconversion Panel Donor No.64090
Product Code: HBV10231

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

Panel Member	Blood Date	VENDOR ALT	ZEPTOMETRIX HBV RT-PCR Copies/mL	VENDOR HBsAg	VENDOR Anti HIV 1/2	VENDOR HIV p24Ag	VENDOR Anti HCV	GENETICS SYSTEMS HBsAg
10231-01	04/26/97	7	Not Detected	Negative	Negative	Negative	Negative	Negative
10231-02	05/13/97	15	102	Negative	Negative	Negative	Negative	Negative
10231-03	06/02/97	11	905	Negative	Negative	Negative	Negative	Negative
10231-04	06/08/97	10	33.3	Positive	Negative	Negative	Negative	Positive
10231-05	06/17/97	10	2,960	Positive	Negative	Negative	Negative	Positive

Panel Member	ABBOTT Auszyme Monoclonal HBsAg	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi Anti-HBc Total	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi Anti-HBe	ORTHO VITROS ECi Anti-HBc IgM	DIASORIN Anti-HBs	DIASORIN Hbc IgM	ABBOTT Core-Total
10231-01	0.100	Indeterminate	3.49	0.18	0.16	0.00	0.03	0.17	0.495
10231-02	0.117	0.15	3.73	0.16	0.18	0.00	0.02	0.17	0.521
10231-03	0.267	0.42	3.74	0.16	0.20	0.01	0.05	0.17	0.485
10231-04	1.512	Indeterminate	3.69	0.18	0.18	0.01	0.01	0.17	0.496
10231-05	4.667	18.8	3.67	0.22	0.19	0.01	0.04	0.17	0.581

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com