



**HBV Seroconversion Panel Donor No. 65524**  
**Product Code: HBV11001**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 3/15/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL)  TEST DATE: 6/2/2005	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: UNK	ROCHE Cobas Core HBsAg II EIA  TEST DATE: UNK	DIASORIN HBsAg  TEST DATE: UNK	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: UNK
11001-01	10-12-97	2	NOT DETECTED	NOT TESTED	0.300	0.200	0.206	0.513
11001-02	10-18-97	4	NOT DETECTED	NOT TESTED	0.190	0.180	0.209	0.577
11001-03	11-14-97	7	73	NEG	0.190	0.190	0.163	0.449
11001-04	11-16-97	4	156	3280	0.080	0.230	0.246	0.577
11001-05	11-25-97	2	1358	5330	2.780	1.490	0.270	0.577
11001-06	11-28-97	3	2709	11700	2.320	2.510	0.401	1.667
11001-07	12-10-97	1	107834	NOT TESTED	59.140	69.600	4.888	9.167
11001-08	12-14-97	4	303917	832000	OVER	OVER	5.168	10.449

PANEL MEMBER	DADE BEHRING Enzygnost HBsAg 5.0  TEST DATE: 1/24/2000	ORTHO VITROS HBsAg Reagent Pack  TEST DATE: 2/24/2000	ROCHE Elecys HBsAg  TEST DATE: 2/9/2000	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 3/22/2000	SORIN ETI-MAK 4  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3  TEST DATE: 8/27/2009	ABBOTT AxSYM HBsAg (V2) Reagent Pack  TEST DATE: 2/28/2000
11001-01	0.340	0.090	0.520	0.190	0.150	0.77	0.51	0.370
11001-02	0.290	0.090	0.540	0.190	0.150	0.52	0.50	0.340
11001-03	0.260	0.090	0.580	0.190	0.250	0.74	0.61	0.390
11001-04	0.350	0.090	0.610	0.320	0.280	0.63	0.64	0.420
11001-05	1.600	1.840	1.800	2.080	1.830	4.50	2.53	0.920
11001-06	2.810	3.340	2.950	2.750	3.330	8.36	4.37	1.540
11001-07	35.030	144.000	72.490	OVER	70.150	141.70	31.27	33.340
11001-08	OVER	521.000	258.500	OVER	87.250	44.92	32.01	60.870

PANEL MEMBER	ABBOTT Auszyme Monoclonal HBsAg  TEST DATE: UNK	ORTHO VITROS ECi HBsAg  TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL (Co/S)  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc IgM  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL  TEST DATE: UNK	ORTHO VITROS ECi HBeAg  TEST DATE: UNK	ABBOTT HBeAg (EIA)  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBe  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBs QUANT  TEST DATE: UNK
11001-01	0.075	0.14	NOT TESTED	0.01	3.06	0.22	NOT TESTED	0.17	1.40
11001-02	0.057	0.13	NOT TESTED	0.01	3.07	0.23	NOT TESTED	0.15	0.50
11001-03	0.113	0.14	NOT TESTED	0.01	3.21	0.22	NOT TESTED	0.14	1.70
11001-04	0.113	0.17	NOT TESTED	0.01	3.03	0.21	NOT TESTED	0.17	0.80
11001-05	1.642	1.66	NOT TESTED	0.01	3.01	0.23	NOT TESTED	0.15	1.80
11001-06	3.226	3.45	NOT TESTED	0.01	3.15	0.23	NOT TESTED	0.15	0.50
11001-07	>37.736	118	NOT TESTED	0.02	3.08	0.55	NOT TESTED	0.15	1.40
11007-08	>37.736	448	0.507	0.01	3.19	2.26	0.608	0.14	2.30

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)