



**HBV Seroconversion Panel Donor No. 65556
Part Number: HBV11002**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 5/17/1999	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: UNK	ROCHE Cobas Core HBsAg II EIA TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	DIASORIN HBsAg TEST DATE: UNK
11002-01	11/06/97	4	183	0.270	0.220	NEG	0.072
11002-02	11/08/97	2	183	0.490	0.250	NEG	0.104
11002-03	11/13/97	12	1030	0.580	0.590	1390	0.102
11002-04	11/15/97	5	1389	0.680	0.880	2750	0.168
11002-05	12/11/97	9	1029804	OVER	OVER	1150000	4.324
11002-06	12/15/97	11	373899	72.550	OVER	537000	5.655

PANEL MEMBER	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 1/24/2000	ORTHO VITROS HBsAg Reagent Pack TEST DATE: 2/24/2000	ROCHE Elecsys HBsAg TEST DATE: 2/9/2000	ABBOTT ANTI-HBc TOTAL (Co/S) TEST DATE: UNK	SORIN ETI-MAK 4 TEST DATE: UNK
11002-01	0.513	0.360	0.140	0.680	NOT TESTED	0.330
11002-02	0.577	0.420	0.130	0.680	NOT TESTED	0.300
11002-03	0.577	1.120	0.530	1.070	NOT TESTED	0.700
11002-04	0.513	1.120	0.820	1.340	NOT TESTED	1.000
11002-05	11.538	35.170	253.000	176.700	NOT TESTED	86.680
11002-06	7.692	OVER	1010.000	628.000	0.513	69.650

PANEL MEMBER	ABBOTT HBeAg TEST DATE: 9/1/1999	ABBOTT PRISM HBsAg TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3 TEST DATE: 8/27/2009	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 2/28/2000	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/22/2000	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK
11002-01	0.288	0.46	0.63	0.420	0.250	0.132
11002-02	0.288	0.95	0.67	0.410	0.250	0.170
11002-03	0.288	2.18	1.23	0.570	0.690	0.660
11002-04	0.481	2.91	1.56	0.690	1.480	0.849
11002-05	2.019	364.79	31.70	89.950	OVER	>37.736
11002-06	4.135	227.22	30.85	56.660	OVER	>37.736

Notes on Data:

- This panel is for **Research Use Only. Not for use in diagnostic procedures.**
- Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
- All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
- No preservatives have been added.
- All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com