



**HBV Seroconversion Panel Donor No. 66201**  
**Part Number: HBV11006**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 3/3/1998	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL)  TEST DATE: 6/2/2005	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: 7/14/2000	DADE BEHRING Enzygnost HBsAg 5.0  TEST DATE: 3/16/2000	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: 1/4/12000	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK	ABBOTT AxSYM HBsAg (V2) Reagent Pack  TEST DATE: UNK
11006-01	01/03/98	4	NOT DETECTED	NEGATIVE	0.256	0.280	0.170	0.46	0.500
11006-02	01/06/98	5	NOT DETECTED	NEGATIVE	0.256	0.190	0.280	0.55	0.510
11006-03	01/10/98	3	NOT DETECTED	NEGATIVE	0.577	0.200	0.310	0.51	0.440
11006-04	01/13/98	1	NOT DETECTED	NEGATIVE	0.385	0.220	0.090	0.50	0.450
11006-05	01/17/98	5	NOT DETECTED	NEGATIVE	0.513	0.150	0.330	0.48	0.470
11006-06	01/22/98	5	357	NEGATIVE	0.385	0.150	0.230	0.50	0.430
11006-07	01/24/98	6	<52	NEGATIVE	0.577	0.150	0.250	0.49	0.480
11006-08	01/29/98	12	<52	NEGATIVE	0.449	0.190	0.280	0.47	0.440
11006-09	01/31/98	12	<52	NEGATIVE	0.321	0.250	0.250	0.49	0.390
11006-10	02/07/98	13	149	369	0.321	0.370	0.420	0.68	0.490
11006-11	02/09/98	6	259	282	0.321	0.280	0.390	0.59	0.460
11006-12	02/14/98	10	1238	4150	0.385	0.740	0.580	1.09	0.740
11006-13	02/16/98	5	1918	5680	1.026	1.160	1.270	1.46	0.900
11006-14	02/21/98	12	6962	22500	2.756	4.270	4.320	4.57	2.770
11006-15	02/23/98	9	22530	76300	1.859	5.450	5.500	7.22	3.120
11006-16	02/28/98	9	54045	184000	4.679	24.530	22.670	25.07	10.260
11006-17	03/02/98	12	85655	349000	6.282	40.540	41.080	27.79	20.830

PANEL MEMBER	ORTHO VITROS HBsAg Reagent Pack  TEST DATE: UNK	ROCHE ELECSYS HBsAg  TEST DATE: UNK	DIASORIN HBsAg  TEST DATE: 10/24/1999	ABBOTT PRISM HBsAg  TEST DATE: UNK	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 12/20/1999	ABBOTT Auszyme Monoclonal HBsAg  TEST DATE: UNK	SORIN ETI-MAK 4  TEST DATE: UNK	ORTHO VITROS HBsAg  TEST DATE: 8/30/2018
11006-01	0.230	0.510	0.269	0.58	0.170	0.127	0.330	0.04
11006-02	0.090	0.460	0.225	0.48	0.260	0.200	0.240	0.04
11006-03	0.130	0.460	0.201	0.60	0.150	0.345	0.260	0.03
11006-04	0.100	0.420	0.192	0.54	0.170	0.073	0.210	0.05
11006-05	0.080	0.530	0.154	0.39	0.200	0.273	0.190	0.04
11006-06	0.090	0.440	0.183	0.57	0.200	0.164	0.210	0.04
11006-07	0.130	0.500	0.160	0.35	0.290	0.073	0.290	0.03
11006-08	0.080	0.490	0.186	0.42	0.170	0.073	0.190	0.04
11006-09	0.070	0.480	0.186	0.63	0.150	0.218	0.290	0.04
11006-10	0.130	0.580	0.198	0.58	0.260	0.182	0.430	0.07
11006-11	0.120	0.490	0.225	0.54	0.260	0.345	0.380	0.09
11006-12	0.430	0.940	0.249	1.64	0.610	0.455	0.950	0.26
11006-13	0.980	1.330	0.246	2.77	1.140	0.927	1.590	0.64
11006-14	5.580	4.070	0.527	9.04	3.350	3.818	5.520	4.48
11006-15	8.070	5.690	0.642	18.12	4.750	4.727	7.520	7.98
11006-16	37.400	23.040	1.908	56.16	25.050	23.818	26.260	36.7
11006-17	66.880	44.570	3.334	80.17	41.130	30.000	48.410	54.5

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HBV Seroconversion Panel Donor No. 66201  
Part Number: HBV11006**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi ANTI-HCV TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018	ABBOTT ANTI-HBc TOTAL (Co/S) TEST DATE: 9/1/1999	ABBOTT HBeAg TEST DATE: 9/1/1999	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI HBe TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs TEST DATE: 8/30/2018
11006-01	0.01	0.12	3.11	NOT TESTED	NOT TESTED	0.13	0.18	0.01	0.0
11006-02	0.01	0.12	3.14	NOT TESTED	NOT TESTED	0.17	0.18	0.01	0.0
11006-03	0.01	0.12	3.00	NOT TESTED	NOT TESTED	0.14	0.17	0.01	0.0
11006-04	0.01	0.12	3.27	NOT TESTED	NOT TESTED	0.13	0.17	0.01	0.0
11006-05	0.01	0.12	3.33	NOT TESTED	NOT TESTED	0.14	0.18	0.01	0.0
11006-06	0.01	0.12	3.22	NOT TESTED	NOT TESTED	0.13	0.18	0.01	0.3
11006-07	0.01	0.13	3.35	NOT TESTED	NOT TESTED	0.14	0.16	0.01	0.0
11006-08	0.01	0.12	3.26	NOT TESTED	NOT TESTED	0.13	0.19	0.01	0.0
11006-09	0.01	0.13	3.10	NOT TESTED	NOT TESTED	0.13	0.19	0.01	0.1
11006-10	0.01	0.13	3.19	NOT TESTED	NOT TESTED	0.13	0.18	0.01	0.0
11006-11	0.01	0.12	3.14	NOT TESTED	NOT TESTED	0.15	0.18	0.01	0.2
11006-12	0.01	0.12	3.20	NOT TESTED	NOT TESTED	0.14	0.20	0.01	0.0
11006-13	0.01	0.12	3.10	NOT TESTED	NOT TESTED	0.14	0.18	0.01	0.8
11006-14	0.01	0.11	3.13	NOT TESTED	NOT TESTED	0.16	0.18	0.01	0.0
11006-15	0.01	0.13	3.16	NOT TESTED	NOT TESTED	0.17	0.19	0.01	0.1
11006-16	0.01	0.13	3.17	NOT TESTED	NOT TESTED	0.27	0.18	0.01	0.0
11006-17	0.01	0.13	3.11	0.565	0.405	0.33	0.18	0.01	0.2

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)