



HBV Seroconversion Panel Donor No. 66537
Part Number: HBV11007

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: UNK	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: 7/14/2000	DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 2/21/2000	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK
11007-01	01/27/98	2	NOT DETECTED	0.192	0.150	0.000	0.51	0.094
11007-02	01/29/98	2	NOT DETECTED	0.256	0.200	0.000	0.50	0.038
11007-03	02/03/98	5	NOT DETECTED	0.385	0.190	0.000	0.49	0.094
11007-04	02/09/98	5	NOT DETECTED	0.513	0.230	0.000	0.49	0.094
11007-05	02/11/98	2	<52	0.256	0.250	0.080	0.50	0.038
11007-06	02/16/98	11	<52	0.449	0.190	0.140	0.52	0.038
11007-07	02/18/98	3	<52	0.577	0.250	0.000	0.50	0.075
11007-08	02/23/98	13	76	0.321	0.250	0.000	0.56	0.132
11007-09	02/25/98	15	297	0.321	0.220	0.080	0.64	0.170
11007-10	03/02/98	16	566	0.577	0.630	0.200	1.28	0.547
11007-11	03/04/98	10	1296	0.513	0.890	0.360	1.82	0.792
11007-12	03/09/98	8	3571	2.692	2.950	1.640	4.04	2.642
11007-13	03/11/98	10	6804	2.051	4.550	3.200	7.10	4.736
11007-14	03/16/98	11	18313	2.038	9.250	6.830	12.61	8.698

PANEL MEMBER	BIORAD MONALISA HBsAg PLUS TEST DATE: 2/2/2000	ABBOTT PRISM HBsAg TEST DATE: UNK	DIASORIN HBsAg TEST DATE: UNK	ROCHE ELECSYS HBsAg TEST DATE: 2/9/2000	ABBOTT Auszyme MC Dynamic TEST DATE: UNK	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 1/24/2000	ABBOTT IMx HBsAg (V2) Reagent Pack TEST DATE: 2/9/2000	ORTHO VITROS HBsAg Reagent Pack TEST DATE: 2/24/2000	SORIN ETI-MAK 4 TEST DATE: UNK
11007-01	0.230	0.96	0.148	0.330	0.040	0.420	0.410	0.080	0.210
11007-02	0.260	0.81	0.219	0.450	0.000	0.440	0.380	0.070	0.110
11007-03	0.280	0.46	0.166	0.380	0.080	0.460	0.400	0.090	0.240
11007-04	0.300	0.49	0.154	0.350	0.000	0.470	0.380	0.080	0.080
11007-05	0.300	0.90	0.163	0.390	0.000	0.380	0.450	0.070	0.130
11007-06	0.210	0.60	0.213	0.430	0.200	0.440	0.470	0.070	0.130
11007-07	0.230	0.42	0.204	0.410	0.120	0.470	0.450	0.080	0.160
11007-08	0.260	0.49	0.207	0.440	0.080	0.470	0.450	0.080	0.180
11007-09	0.280	0.58	0.148	0.500	0.160	0.510	0.480	0.100	0.370
11007-10	0.420	1.70	0.213	0.810	0.920	0.710	0.540	0.280	0.92
11007-11	0.490	1.85	0.219	1.100	1.440	0.740	0.580	0.600	1.290
11007-12	1.110	6.48	0.373	2.830	5.000	1.780	1.120	3.540	4.240
11007-13	1.530	10.32	0.553	4.590	6.960	2.940	1.640	7.050	6.450
11007-14	2.260	19.29	0.675	8.710	14.920	5.220	2.500	13.900	14.900

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HBV Seroconversion Panel Donor No. 66537
Part Number: HBV11007

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SORIN ETI-MAK 3	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi ANTI-HCV	ORTHO VITROS ECi ANTI-HIV 1/2	ABBOTT ANTI-HBc TOTAL (Co/S)	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT HBeAg	ORTHO VITROS ECi HBeAg
	TEST DATE: 12/9/1999	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 9/1/1999	TEST DATE: 8/30/2018	TEST DATE: 9/1/1999	TEST DATE: 8/30/2018
11007-01	0.070	0.04	0.05	0.11	NOT TESTED	3.49	NOT TESTED	0.14
11007-02	0.120	0.08	0.03	0.11	NOT TESTED	3.55	NOT TESTED	0.12
11007-03	0.120	0.05	0.03	0.12	NOT TESTED	3.52	NOT TESTED	0.12
11007-04	0.140	0.05	0.03	0.11	NOT TESTED	3.66	NOT TESTED	0.12
11007-05	0.070	0.04	0.02	0.11	NOT TESTED	3.39	NOT TESTED	0.14
11007-06	0.070	0.04	0.02	0.11	NOT TESTED	3.42	NOT TESTED	0.13
11007-07	0.090	0.05	0.02	0.12	NOT TESTED	3.54	NOT TESTED	4.69
11007-08	0.070	0.04	0.02	0.11	NOT TESTED	3.69	NOT TESTED	0.13
11007-09	0.140	0.06	0.02	0.11	NOT TESTED	3.78	NOT TESTED	0.13
11007-10	0.480	0.22	0.02	0.12	NOT TESTED	3.47	NOT TESTED	0.13
11007-11	0.590	0.49	0.02	0.11	NOT TESTED	3.56	NOT TESTED	0.16
11007-12	1.920	2.60	0.01	0.13	NOT TESTED	3.53	NOT TESTED	0.15
11007-13	3.950	5.53	0.02	0.12	NOT TESTED	3.40	NOT TESTED	0.18
11007-14	6.740	20.3	0.02	0.13	0.450	3.51	0.338	0.23

PANEL MEMBER	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs
	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11007-01	0.19	0.01	4.0
11007-02	0.18	0.03	1.9
11007-03	0.19	0.03	0.9
11007-04	0.18	0.03	2.0
11007-05	0.18	0.02	1.1
11007-06	0.18	0.02	1.9
11007-07	0.17	0.02	1.0
11007-08	0.16	0.02	0.9
11007-09	0.17	0.02	1.1
11007-10	0.18	0.02	3.0
11007-11	0.19	0.02	0.4
11007-12	0.17	0.02	1.8
11007-13	0.18	0.02	0.3
11007-14	0.19	0.03	0.4

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com