



HBV Seroconversion Panel Donor No. 67303
Part Number: HBV11008

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: UNK	ABBOTT HBV REALTIME CP/ML TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: 7/14/2000	DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 1/24/2000	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 4/18/2000	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	DIASORIN HBsAg TEST DATE: 10/24/1999
11008-01	03/09/98	11	NOT DETECTED	NEGATIVE	0.643	0.250	0.000	0.53	0.216
11008-02	03/16/98	10	NOT DETECTED	NEGATIVE	0.643	0.250	0.070	0.50	0.172
11008-03	04/06/98	6	NOT DETECTED	NEGATIVE	0.643	0.300	0.040	0.50	0.195
11008-04	04/09/98	7	NOT DETECTED	NEGATIVE	0.571	0.330	0.400	0.47	0.180
11008-05	04/14/98	10	NOT DETECTED	NEGATIVE	0.714	0.360	0.070	0.52	0.169
11008-06	04/16/98	14	NOT DETECTED	NEGATIVE	0.500	0.290	0.070	0.55	0.166
11008-07	04/21/98	10	NOT DETECTED	NEGATIVE	0.500	0.210	0.040	0.56	0.166
11008-08	04/26/98	3	NOT DETECTED	NEGATIVE	0.714	0.260	0.130	0.52	0.175
11008-09	04/29/98	6	<52	NEGATIVE	0.571	0.250	0.160	0.51	0.219
11008-10	05/03/98	7	<52	NEGATIVE	0.714	0.240	0.070	0.50	0.175
11008-11	05/06/98	7	58	NEGATIVE	0.571	0.240	0.070	0.50	0.166
11008-12	05/10/98	8	297	70.7	0.714	0.320	0.430	0.60	0.183
11008-13	05/13/98	4	290	844	0.857	0.470	0.190	0.78	0.231
11008-14	05/17/98	7	781	2020	0.786	0.730	0.700	1.50	0.243
11008-15	05/20/98	4	2055	6730	1.071	0.730	0.870	2.23	0.260
11008-16	05/27/98	6	8180	32600	5.286	8.680	9.880	14.10	0.769
11008-17	06/01/98	10	31824	83300	9.429	20.280	26.180	27.46	1.680
11008-18	06/03/98	7	91781	169000	14.214	31.350	41.410	29.33	2.083

PANEL MEMBER	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: UNK	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: UNK	ABBOTT IMx HBsAg (V2) Reagent Pack TEST DATE: 2/10/2000	ORTHO VITROS HBsAg Reagent Pack TEST DATE: 2/24/2000	ROCHE ELECSYS HBsAg TEST DATE: 2/9/2000	ABBOTT PRISM HBsAg TEST DATE: UNK	SORIN ETI-MAK 3 TEST DATE: UNK	SORIN ETI-MAK 4 TEST DATE: UNK
11008-01	0.100	0.073	0.570	0.500	0.080	0.470	0.37	0.080	0.200
11008-02	0.100	0.055	0.470	0.480	0.080	0.440	0.67	0.080	0.180
11008-03	0.100	0.073	0.480	0.3940	0.070	0.450	0.69	0.140	0.180
11008-04	0.100	0.073	0.430	0.390	0.070	0.530	0.66	0.120	0.280
11008-05	0.100	0.073	0.440	0.440	0.080	0.480	0.44	0.000	0.200
11008-06	0.140	0.073	0.550	0.420	0.070	0.560	0.50	0.250	0.180
11008-07	0.310	0.055	0.460	0.440	0.070	0.490	0.71	0.210	0.200
11008-08	0.100	0.055	0.470	0.420	0.060	0.510	0.62	0.190	0.230
11008-09	0.100	0.055	0.500	0.420	0.060	0.520	0.46	0.210	0.180
11008-10	0.100	0.091	0.500	0.430	0.070	0.580	0.59	0.190	0.230
11008-11	0.170	0.036	0.500	0.440	0.070	0.560	0.42	0.190	0.350
11008-12	0.140	0.091	0.520	0.460	0.080	0.580	0.79	0.190	0.300
11008-13	0.280	0.200	0.560	0.470	0.150	0.650	0.80	0.280	0.350
11008-14	0.830	0.455	0.670	0.590	0.310	0.920	1.64	0.410	0.800
11008-15	1.030	0.818	0.940	0.670	0.730	1.090	2.24	0.470	1.280
11008-16	11.520	7.455	4.520	2.960	12.800	6.690	17.14	3.740	12.230
11008-17	25.900	15.636	12.010	7.460	41.100	18.560	46.64	8.880	24.250
11008-18	35.380	33.273	16.930	10.640	69.00	30.810	63.85	15.480	37.180

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HBV Seroconversion Panel Donor No. 67303
Part Number: HBV11008**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS Eci HBsAg TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HCV TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: 8/30/2018	ABBOTT ANTI-HBc TOTAL (Co/S) TEST DATE: 9/1/1999	ORTHO VITROS Eci ANTI-HBc TOTAL TEST DATE: 8/30/2018	ABBOTT HBeAg TEST DATE: 9/1/1999	ORTHO VITROS Eci HBeAg TEST DATE: 8/30/2018
11008-01	0.05	0.02	0.11	NOT TESTED	2.35	NOT TESTED	0.12
11008-02	0.05	0.01	0.11	NOT TESTED	2.14	NOT TESTED	0.13
11008-03	0.04	0.01	0.12	NOT TESTED	2.48	NOT TESTED	0.13
11008-04	0.04	0.01	0.11	NOT TESTED	2.34	NOT TESTED	0.12
11008-05	0.04	0.01	0.11	NOT TESTED	2.31	NOT TESTED	0.13
11008-06	0.04	0.01	0.11	NOT TESTED	2.23	NOT TESTED	0.13
11008-07	0.04	0.01	0.12	NOT TESTED	2.28	NOT TESTED	0.12
11008-08	0.04	0.01	0.11	NOT TESTED	2.24	NOT TESTED	0.13
11008-09	0.29	0.01	0.12	NOT TESTED	2.29	NOT TESTED	0.24
11008-10	0.04	0.01	0.10	NOT TESTED	2.25	NOT TESTED	0.12
11008-11	0.04	0.01	0.10	NOT TESTED	2.53	NOT TESTED	0.12
11008-12	0.06	0.01	0.12	NOT TESTED	2.03	NOT TESTED	0.13
11008-13	0.08	0.01	0.12	NOT TESTED	2.42	NOT TESTED	0.13
11008-14	0.38	0.01	0.12	NOT TESTED	2.44	NOT TESTED	0.14
11008-15	0.84	0.01	0.12	NOT TESTED	2.58	NOT TESTED	0.13
11008-16	9.35	0.01	0.11	NOT TESTED	2.41	NOT TESTED	0.17
11008-17	27.5	0.01	0.11	NOT TESTED	2.26	NOT TESTED	0.33
11008-18	44.1	0.01	0.12	0.469	2.48	0.608	0.93

PANEL MEMBER	ORTHO VITROS Eci ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBs TEST DATE: 8/30/2018
11008-01	0.18	0.03	1.3
11008-02	0.20	0.03	0.7
11008-03	0.19	0.03	0.4
11008-04	0.19	0.02	0.6
11008-05	0.19	0.03	1.6
11008-06	0.19	0.02	0.8
11008-07	0.19	0.03	0.5
11008-08	0.19	0.03	1.3
11008-09	0.18	0.03	0.7
11008-10	0.19	0.03	0.4
11008-11	0.19	0.02	0.3
11008-12	0.21	0.03	0.3
11008-13	0.19	0.02	0.6
11008-14	0.19	0.02	0.2
11008-15	0.20	0.02	0.1
11008-16	0.19	0.02	0.6
11008-17	0.19	0.03	0.2
11008-18	0.17	0.02	0.4

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com