



HBV Seroconversion Panel Donor No. 67457
Part Number: HBV11010

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	GENETIC SYSTEMS HBsAg (2.0)	DADE BEHRING Enzygnost HBsAg 5.0	ORTHO HBsAg ELISA Testsystem 3	ABBOTT MUREX HBsAg Version 3	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 1/24/2000	TEST DATE: 4/18/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11010-01	03/27/98	7	NOT DETECTED	0.429	0.360	0.100	0.53	0.070	0.055
11010-02	04/01/98	1	NOT DETECTED	0.643	0.280	0.190	0.54	0.100	0.055
11010-03	04/18/98	8	NOT DETECTED	0.429	0.260	0.000	0.57	0.170	0.073
11010-04	04/21/98	4	NOT DETECTED	0.500	0.290	0.100	0.63	0.100	0.018
11010-05	04/25/98	0	NOT DETECTED	0.429	0.290	0.070	0.63	0.100	0.055
11010-06	04/30/98	6	NOT DETECTED	0.429	0.400	0.490	0.57	0.140	0.036
11010-07	05/02/98	9	NOT DETECTED	0.714	0.370	0.130	0.52	0.030	0.036
11010-08	05/07/98	8	NOT DETECTED	0.643	0.470	0.130	0.55	0.070	0.036
11010-09	05/09/98	0	NOT DETECTED	0.571	0.300	0.100	0.54	0.140	0.055
11010-10	05/14/98	0	NOT DETECTED	0.571	0.280	0.160	0.54	0.100	0.073
11010-11	05/16/98	2	NOT DETECTED	0.357	0.280	0.000	0.58	0.100	0.036
11010-12	05/21/98	0	105	0.357	0.280	0.100	0.61	0.070	0.055
11010-13	05/23/98	6	103	0.500	0.330	0.070	0.65	0.140	0.055
11010-14	05/28/98	4	460	0.571	0.550	0.700	0.77	0.310	0.145
11010-15	05/30/98	8	1182	0.643	0.750	0.490	1.02	0.480	0.364
11010-16	06/08/98	4	8969	3.857	5.300	5.820	6.93	4.860	4.545
11010-17	06/12/98	3	39152	6.786	14.840	16.250	21.45	15.760	11.091
11010-18	06/15/98	3	98345	13.571	36.620	46.710	28.21	39.170	26.909

PANEL MEMBER	ABBOTT PRISM HBsAg	DIASORIN HBsAg	ROCHE ELECSYS HBsAg	SORIN ETI-MAK 3	SORIN ETI-MAK 4	ABBOTT IMx HBsAg (V2) Reagent Pack	ABBOTT AxSYM HBsAg (V2) Reagent Pack	ORTHO VITROS HBsAg Reagent Pack	ORTHO VITROS ECI HBsAg
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 2/9/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 2/10/2000	TEST DATE: UNK	TEST DATE: 2/24/2000	TEST DATE: 8/30/2018
11010-01	0.33	0.157	0.400	0.230	0.250	0.380	0.400	0.100	0.05
11010-02	0.33	0.178	0.500	0.260	0.150	0.400	0.490	0.090	0.07
11010-03	0.60	0.180	0.420	0.690	0.230	0.380	0.460	0.100	0.06
11010-04	0.46	0.192	0.450	0.230	0.250	0.430	0.420	0.100	0.05
11010-05	0.72	0.180	0.500	0.390	0.230	0.380	0.460	0.100	0.06
11010-06	0.26	0.198	0.430	0.490	0.230	0.390	0.460	0.100	0.07
11010-07	0.32	0.166	0.470	0.330	0.200	0.420	0.410	0.110	0.12
11010-08	0.35	0.154	0.410	0.390	0.200	0.390	0.460	0.090	0.06
11010-09	0.53	0.160	0.460	0.310	0.250	0.400	0.420	0.110	0.05
11010-10	0.47	0.154	0.470	0.3220	0.230	0.430	0.490	0.090	0.13
11010-11	0.31	0.148	0.490	0.240	0.200	0.460	0.470	0.090	0.06
11010-12	0.52	0.142	0.560	0.260	0.200	0.400	0.440	0.110	0.06
11010-13	0.56	0.151	0.480	0.580	0.200	0.430	0.480	0.120	0.08
11010-14	1.03	0.180	0.630	0.310	0.450	0.490	0.490	0.160	0.16
11010-15	2.98	0.186	0.770	0.490	0.630	0.520	0.590	0.260	0.21
11010-16	5.15	0.385	4.460	2.950	4.280	1.560	2.610	7.620	6.43
11010-17	46.01	0.929	12.970	6.370	15.130	4.200	7.350	23.000	27.1
11010-18	94.00	2.201	35.640	19.310	36.700	11.310	18.890	66.400	51.7

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HBV Seroconversion Panel Donor No. 67457
Part Number: HBV11010**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT ANTI-HBc TOTAL (Co/S) TEST DATE: 9/1/1999	ORTHO VITROS ECi ANTI-HCV TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HIV 1/2 TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018	ABBOTT HBeAg TEST DATE: 9/1/1999	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI HBe TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs TEST DATE: 8/30/2018
11010-01	NOT TESTED	0.04	0.16	3.51	NOT TESTED	0.14	0.15	0.01	0.8
11010-02	NOT TESTED	0.03	0.16	3.34	NOT TESTED	0.14	0.14	0.01	0.9
11010-03	NOT TESTED	0.04	0.14	3.37	NOT TESTED	0.14	0.16	0.01	1.9
11010-04	NOT TESTED	0.04	0.16	3.32	NOT TESTED	0.14	0.15	0.01	0.9
11010-05	NOT TESTED	0.04	0.16	3.46	NOT TESTED	0.14	0.15	0.01	2.5
11010-06	NOT TESTED	0.04	0.15	3.34	NOT TESTED	0.14	0.17	0.01	1.2
11010-07	NOT TESTED	0.03	0.17	3.52	NOT TESTED	0.13	0.17	0.01	0.5
11010-08	NOT TESTED	0.03	0.15	3.37	NOT TESTED	0.14	0.16	0.01	1.4
11010-09	NOT TESTED	0.03	0.16	3.22	NOT TESTED	0.14	0.16	0.01	0.7
11010-10	NOT TESTED	0.03	0.15	3.57	NOT TESTED	0.13	0.16	0.01	1.0
11010-11	NOT TESTED	0.02	0.16	3.61	NOT TESTED	0.13	0.16	0.01	1.1
11010-12	NOT TESTED	0.04	0.15	3.31	NOT TESTED	0.14	0.17	0.01	1.9
11010-13	NOT TESTED	0.03	0.16	3.38	NOT TESTED	0.17	0.17	0.01	2.3
11010-14	NOT TESTED	0.03	0.18	3.26	NOT TESTED	0.14	0.16	0.01	0.4
11010-15	NOT TESTED	0.04	0.15	3.26	NOT TESTED	0.15	0.18	0.01	0.2
11010-16	NOT TESTED	0.03	0.16	3.41	NOT TESTED	0.17	0.16	0.01	0.9
11010-17	NOT TESTED	0.05	0.15	3.46	NOT TESTED	0.25	0.18	0.01	1.2
11010-18	0.421	0.04	0.15	3.18	0.405	0.75	0.16	0.01	0.4

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com