



**HBV Seroconversion Panel Donor No. 67694  
Part Number: HBV11011**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: UNK	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL)  TEST DATE: 6/2/2005	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0  TEST DATE: 1/24/2000
11011-01	03/14/98	9	NOT DETECTED	NEGATIVE	0.286	0.360
11011-02	04/30/98	9	NOT DETECTED	NEGATIVE	0.357	0.270
11011-03	05/02/98	7	NOT DETECTED	NEGATIVE	0.357	0.220
11011-04	05/07/98	3	NOT DETECTED	NEGATIVE	0.571	0.290
11011-05	05/09/98	1	NOT DETECTED	NEGATIVE	0.429	0.290
11011-06	05/16/98	5	NOT DETECTED	NEGATIVE	0.429	0.310
11011-07	05/21/98	6	NOT DETECTED	NEGATIVE	0.786	0.500
11011-08	05/23/98	7	NOT DETECTED	NEGATIVE	0.643	0.610
11011-09	06/25/98	8	1327	6610	0.929	0.810
11011-10	06/27/98	8	1559	5860	1.000	1.130
11011-11	07/02/98	4	9178	41100	2.786	4.200
11011-12	07/04/98	4	18313	86800	8.786	6.390
11011-13	07/09/98	6	66490	228000	15.000	24.830
11011-14	07/11/98	6	112916	366000	17.143	34.460

PANEL MEMBER	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK	DIASORIN HBsAg  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: UNK	ROCHE ELECSYS HBsAg  TEST DATE: 2/9/2000	ROCHE Cobas Core HBsAg II EIA  TEST DATE: UNK	SORIN ETI-MAK 4  TEST DATE: UNK
11011-01	NOT REACTIVE	0.51	0.136	0.46	0.510	0.130	0.230
11011-02	0.550	0.47	0.104	0.42	0.550	0.120	0.200
11011-03	0.330	0.54	0.142	0.52	0.560	0.110	0.180
11011-04	0.190	0.55	0.157	0.60	0.520	0.110	0.180
11011-05	0.460	0.50	0.157	0.49	0.480	0.120	0.150
11011-06	0.190	0.49	0.151	0.56	0.560	0.130	0.180
11011-07	0.190	0.48	0.127	0.42	0.490	0.120	0.250
11011-08	0.380	0.48	0.154	0.36	0.540	0.140	0.280
11011-09	0.930	1.04	0.219	1.80	0.820	0.820	0.750
11011-10	0.980	1.76	0.166	3.35	1.050	1.080	1.230
11011-11	3.900	4.90	0.269	10.93	2.650	4.200	11.230
11011-12	7.560	8.76	0.550	18.51	4.430	8.500	8.250
11011-13	30.360	27.72	1.876	65.36	17.400	37.100	32.580
11011-14	50.400	29.29	2.633	94.81	27.310	56.900	47.080

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HBV Seroconversion Panel Donor No. 67694  
Part Number: HBV11011**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT HBeAg  TEST DATE: 9/1/1999	ABBOTT ANTI-HBc TOTAL (Co/S)  TEST DATE: 9/1/1999	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 3/22/2000	ABBOTT Auszyme Monoclonal HBsAg  TEST DATE: UNK	ORTHO VITROS HBsAg Reagent Pack  TEST DATE: 2/24/2000	ABBOTT AxSYM HBsAg (V2) Reagent Pack  TEST DATE: 2/28/2000
11011-01	NOT TESTED	NOT TESTED	0.130	0.019	0.140	0.320
11011-02	NOT TESTED	NOT TESTED	0.090	0.038	0.130	0.380
11011-03	NOT TESTED	NOT TESTED	0.130	0.038	0.130	0.370
11011-04	NOT TESTED	NOT TESTED	0.160	0.057	0.120	0.370
11011-05	NOT TESTED	NOT TESTED	0.220	0.057	0.130	0.390
11011-06	NOT TESTED	NOT TESTED	0.000	0.019	0.130	0.380
11011-07	NOT TESTED	NOT TESTED	0.060	0.094	0.140	0.370
11011-08	NOT TESTED	NOT TESTED	0.060	0.019	0.130	0.500
11011-09	NOT TESTED	NOT TESTED	0.470	0.604	0.900	0.630
11011-10	NOT TESTED	NOT TESTED	0.510	0.774	1.240	0.670
11011-11	NOT TESTED	NOT TESTED	4.230	3.660	8.140	1.740
11011-12	NOT TESTED	NOT TESTED	8.340	6.566	15.700	3.170
11011-13	NOT TESTED	NOT TESTED	26.120	26.038	71.400	9.500
11011-14	0.540	0.531	54.000	36.792	117.000	15.880

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)