



**HBV Seroconversion Panel Donor No. 67773
Part Number: HBV11012**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 7/31/1998	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 1/24/2000	ROCHE Cobas Core HBsAg II EIA TEST DATE: UNK
11012-01	06/27/98	3	<52	NEGATIVE	0.385	0.390	0.160
11012-02	06/29/98	4	<52	NEGATIVE	0.321	0.300	0.170
11012-03	07/05/98	12	149	NEGATIVE	0.256	0.400	0.190
11012-04	07/15/98	4	2305	5200	0.769	1.040	0.490
11012-05	07/17/98	8	2709	10400	0.833	1.530	0.930
11012-06	07/22/98	5	5404	24500	2.051	4.710	3.210

PANEL MEMBER	DIASORIN HBsAg TEST DATE: UNK	ROCHE ELECSYS HBsAg TEST DATE: 2/9/2000	ABBOTT PRISM HBsAg TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: UNK	SORIN ETI-MAK 4 TEST DATE: UNK
11012-01	0.112	0.450	0.68	0.48	0.190	0.180
11012-02	0.160	0.440	0.56	0.50	0.270	0.330
11012-03	0.183	0.530	0.72	0.56	0.270	0.200
11012-04	0.183	0.930	2.23	1.19	0.790	1.030
11012-05	0.225	1.490	3.69	1.98	1.670	2.330
11012-06	0.589	4.420	11.86	5.79	3.850	7.230

PANEL MEMBER	ABBOTT HBeAg TEST DATE: 9/8/1999	ABBOTT ANTI-HBc TOTAL (Co/S) TEST DATE: 9/8/1999	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/22/2000	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 2/28/2000	ORTHO VITROS HBsAg Reagent Pack TEST DATE: 2/24/2000
11012-01	NOT TESTED	NOT TESTED	0.190	0.038	0.390	0.120
11012-02	NOT TESTED	NOT TESTED	0.160	0.019	0.380	0.110
11012-03	NOT TESTED	NOT TESTED	0.250	0.096	0.370	0.120
11012-04	NOT TESTED	NOT TESTED	0.850	0.577	0.750	0.630
11012-05	NOT TESTED	NOT TESTED	1.770	1.192	0.980	1.690
11012-06	0.364	0.522	4.990	3.154	2.380	6.920

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com