



HBV Seroconversion Panel Donor No. 68029
Part Number: HBV11014

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ROCHE CAP-G/CTM HBV (cp/mL)	GENETIC SYSTEMS HBsAg (2.0)	DADE BEHRING Enzygnost HBsAg 5.0	ORTHO HBsAg ELISA Testsystem 3	ABBOTT MUREX HBsAg Version 3	BIORAD MONALISA HBsAg PLUS
		TEST DATE: 11/2/1998	TEST DATE: UNK	TEST DATE: 6/2/2005	TEST DATE: UNK	TEST DATE: 12/15/1999	TEST DATE: 1/4/2000	TEST DATE: UNK	TEST DATE: UNK
11014-01	06/26/98	3	NOT DETECTED	NEGATIVE	0.571	0.230	0.000	0.52	0.470
11014-02	06/28/98	7	NOT DETECTED	NEGATIVE	0.357	0.170	0.000	0.55	0.350
11014-03	07/03/98	6	NOT DETECTED	NEGATIVE	0.357	0.300	0.000	0.50	0.420
11014-04	07/05/98	5	NOT DETECTED	NEGATIVE	0.500	0.280	0.000	0.50	0.400
11014-05	07/12/98	20	NOT DETECTED	NEGATIVE	0.571	0.250	0.000	0.54	0.350
11014-06	07/17/98	17	NOT DETECTED	NEGATIVE	0.643	0.290	0.000	0.57	0.350
11014-07	07/19/98	12	NOT DETECTED	NEGATIVE	0.714	0.230	0.000	0.55	0.200
11014-08	07/31/98	0	71	NEGATIVE	0.571	0.340	0.000	0.69	0.270
11014-09	08/02/98	7	139	263	1.357	0.320	0.000	0.64	0.400
11014-10	08/16/98	2	3654	21400	3.214	5.170	3.980	4.88	2.000
11014-11	08/20/98	2	13892	85200	7.143	15.310	13.360	13.96	11.240
11014-12	08/23/98	4	41952	136000	13.571	33.160	32.260	24.33	20.620

PANEL MEMBER	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	ABBOTT PRISM HBsAg	BIOTEST ELISA HBsAg	DIASORIN HBsAg	ORTHO VITROS HBsAg Reagent Pack	ABBOTT AxSYM HBsAg (V2) Reagent Pack	ROCHE Cobas Core HBsAg II EIA	SORIN ETI-MAK 3
	TEST DATE: 12/15/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 4/18/2000	TEST DATE: UNK	TEST DATE: 3/23/2000	TEST DATE: 1/10/2000	TEST DATE: 4/18/2000	TEST DATE: 12/8/1999
11014-01	0.110	0.038	0.97	0.140	0.142	0.100	0.750	0.160	0.020
11014-02	0.030	0.019	0.26	0.140	0.109	0.100	0.580	0.140	0.020
11014-03	0.170	0.019	0.72	0.050	0.095	0.100	0.520	0.160	0.020
11014-04	0.110	NOT REACTIVE	0.74	0.220	0.151	0.100	0.500	0.180	0.000
11014-05	0.230	0.096	0.78	0.140	0.189	0.100	0.550	0.220	0.060
11014-06	0.110	0.038	1.46	0.180	0.142	0.100	0.420	0.160	0.040
11014-07	0.090	0.058	1.80	0.120	0.166	0.100	0.470	0.160	0.060
11014-08	0.140	0.058	1.18	0.180	0.198	0.200	0.600	0.220	0.170
11014-09	0.370	0.058	1.88	0.140	0.186	0.100	0.510	0.260	0.070
11014-10	5.140	3.077	11.51	1.010	0.716	6.090	2.720	3.180	2.900
11014-11	16.620	8.731	25.18	3.390	1.284	25.090	7.060	10.630	9.320
11014-12	36.030	22.692	74.26	9.540	2.843	60.560	15.860	25.150	22.880

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HBV Seroconversion Panel Donor No. 68029
Part Number: HBV11014**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SORIN ETI-MAK 4	ABBOTT HBeAg	ABBOTT ANTI-HBc TOTAL (Co/S)
	TEST DATE: UNK	TEST DATE: 9/4/1999	TEST DATE: 9/9/1999
11014-01	0.200	NOT TESTED	NOT TESTED
11014-02	0.150	NOT TESTED	NOT TESTED
11014-03	0.130	NOT TESTED	NOT TESTED
11014-04	0.200	NOT TESTED	NOT TESTED
11014-05	0.150	NOT TESTED	NOT TESTED
11014-06	0.230	NOT TESTED	NOT TESTED
11014-07	0.230	NOT TESTED	NOT TESTED
11014-08	0.350	NOT TESTED	NOT TESTED
11014-09	0.330	NOT TESTED	NOT TESTED
11014-10	6.100	NOT TESTED	NOT TESTED
11014-11	17.200	NOT TESTED	NOT TESTED
11014-12	39.900	0.455	0.517

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com