



**HBV Seroconversion Panel Donor No. 68105**  
**Part Number: HBV11015**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ROCHE CAP-G/CTM HBV (cp/mL)	GENETIC SYSTEMS HBsAg (2.0)	ORTHO HBsAg ELISA Testsystem 3	ABBOTT MUREX HBsAg Version 3
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11015-01	06/15/98	24	NOT DETECTED	NEGATIVE	0.571	0.000	0.55
11015-02	06/17/98	14	NOT DETECTED	NEGATIVE	0.500	0.000	0.69
11015-03	06/23/98	14	NOT DETECTED	NEGATIVE	0.714	0.000	0.54
11015-04	06/26/98	9	NOT DETECTED	NEGATIVE	0.714	0.000	0.55
11015-05	06/30/98	7	NOT DETECTED	NEGATIVE	0.643	0.000	0.53
11015-06	07/03/98	11	NOT DETECTED	NEGATIVE	0.500	0.000	0.52
11015-07	07/11/98	10	NOT DETECTED	NEGATIVE	0.500	0.000	0.54
11015-08	07/13/98	10	NOT DETECTED	NEGATIVE	0.571	0.000	0.51
11015-09	07/19/98	12	<52	NEGATIVE	0.500	0.020	0.53
11015-10	07/21/98	8	<52	NEGATIVE	0.714	0.000	0.63
11015-11	07/26/98	2	54	NEGATIVE	0.571	0.020	0.63
11015-12	07/28/98	NOT TESTED	152	NEGATIVE	0.571	0.310	0.73
11015-13	08/24/98	9	634972	1510000	20.000	OVER	28.61
11015-14	08/26/98	8	1238096	NOT TESTED	21.429	OVER	28.72

PANEL MEMBER	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	ROCHE Cobas Core HBsAg II EIA	BIORAD MONALISA HBsAg PLUS	DADE BEHRING Enzygnost HBsAg 5.0	BIOTEST ELISA HBsAg	DIASORIN HBsAg	ABBOTT PRISM HBsAg
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11015-01	0.060	0.091	0.140	0.290	0.410	0.180	0.112	0.71
11015-02	0.200	0.091	0.140	0.240	0.280	0.180	0.128	0.34
11015-03	0.520	0.036	0.140	0.150	0.260	0.100	0.134	0.48
11015-04	0.200	0.073	0.140	0.220	0.250	0.140	0.139	0.73
11015-05	0.520	0.073	0.140	0.460	0.230	0.120	0.136	0.71
11015-06	0.630	0.055	0.120	0.310	0.260	0.100	0.393	0.54
11015-07	0.370	0.073	0.120	0.350	0.260	0.120	0.249	0.76
11015-08	0.550	0.055	0.140	0.510	0.300	0.180	0.123	0.60
11015-09	0.200	0.073	0.160	0.500	0.280	0.180	0.144	0.34
11015-10	1.030	0.073	0.160	0.370	0.260	0.200	0.152	0.41
11015-11	0.430	0.236	0.200	0.380	0.330	0.140	0.179	0.79
11015-12	1.060	0.109	0.240	0.430	0.400	0.120	0.142	1.62
11015-13	OVER	OVER	70.470	58.480	OVER	56.120	4.570	353.39
11015-14	OVER	OVER	70.470	59.420	OVER	56.390	4.307	440.31

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HBV Seroconversion Panel Donor No. 68105  
Part Number: HBV11015**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: UNK	ORTHO VITROS HBsAg Reagent Pack TEST DATE: UNK	ABBOTT HBeAg TEST DATE: UNK	SORIN ETI-MAK 4 TEST DATE: UNK	SORIN ETI-MAK 3 TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL (Co/S) TEST DATE: UNK
11015-01	0.510	0.210	0.385	0.180	0.060	NOT TESTED
11015-02	0.530	0.170	0.385	0.200	0.020	NOT TESTED
11015-03	0.820	0.130	0.288	0.200	0.040	NOT TESTED
11015-04	0.450	0.120	0.288	0.230	0.040	NOT TESTED
11015-05	0.400	0.110	0.385	0.200	0.000	NOT TESTED
11015-06	0.440	0.120	0.288	0.200	0.040	NOT TESTED
11015-07	0.490	0.180	0.385	0.180	0.060	NOT TESTED
11015-08	0.520	0.150	0.288	0.200	0.020	NOT TESTED
11015-09	0.510	0.160	0.288	0.230	0.110	NOT TESTED
11015-10	0.700	0.160	0.288	0.230	0.070	NOT TESTED
11015-11	0.530	0.170	0.288	0.250	0.110	NOT TESTED
11015-12	0.560	0.170	0.288	0.330	0.110	NOT TESTED
11015-13	101.590	855.130	2.404	84.180	OVER	NOT TESTED
11015-14	116.410	1479.910	3.462	84.530	OVER	0.603

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)