



**HBV Seroconversion Panel Donor No. 68541  
Part Number: HBV11016**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ABBOTT ANTI-HBc TOTAL (Co/S)	GENETIC SYSTEMS HBsAg (2.0)	DADE BEHRING Enzygnost HBsAg 5.0	ABBOTT PRISM HBsAg	BIOTEST ELISA HBsAg
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/8/1999	TEST DATE: UNK	TEST DATE: 12/15/1999	TEST DATE: UNK	TEST DATE: 4/18/2000
11016-01	08/23/98	6	NOT DETECTED	NOT TESTED	0.625	0.950	0.36	0.160
11016-02	08/25/98	3	NOT DETECTED	NOT TESTED	0.563	0.230	0.39	0.180
11016-03	08/31/98	6	NOT DETECTED	NOT TESTED	0.813	0.230	0.33	0.220
11016-04	09/07/98	5	103	NOT TESTED	0.563	0.220	0.36	0.240
11016-05	09/10/98	7	156	NOT TESTED	0.375	0.280	0.65	0.160
11016-06	09/19/98	8	2709	NOT TESTED	0.688	1.240	2.93	0.390
11016-07	09/22/98	7	6804	NOT TESTED	2.000	3.060	5.97	0.540
11016-08	09/28/98	8	NOT TESTED	NOT TESTED	2.813	16.110	5.04	4.050
11016-09	10/01/98	2	220168	NOT TESTED	9.813	37.190	71.83	10.950
11016-10	10/06/98	12	856553	0.484	14.375	OVER	175.62	40.220

PANEL MEMBER	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	ORTHO HBsAg ELISA Testsystem 3	ABBOTT MUREX HBsAg Version 3	ROCHE Cobas Core HBsAg II EIA	BIORAD MONALISA HBsAg PLUS
	TEST DATE: 12/15/1999	TEST DATE: UNK	TEST DATE: 1/4/2000	TEST DATE: UNK	TEST DATE: 4/18/2000	TEST DATE: UNK
11016-01	0.140	0.038	0.310	0.52	0.180	0.280
11016-02	0.200	0.019	0.050	0.52	0.160	0.480
11016-03	0.340	0.038	0.150	0.51	0.160	0.320
11016-04	0.860	0.038	0.120	0.63	0.180	0.300
11016-05	0.110	0.077	0.120	0.64	0.180	0.420
11016-06	1.380	0.846	1.180	1.93	1.050	1.000
11016-07	2.010	2.500	2.370	3.89	2.400	1.930
11016-08	48.110	11.731	15.060	17.68	13.830	8.520
11016-09	51.880	37.942	41.470	28.41	38.960	18.200
11016-10	57.420	>38.462	OVER	28.86	70.470	52.970

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HBV Seroconversion Panel Donor No. 68541  
Part Number: HBV11016**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS HBsAg Reagent Pack TEST DATE: 3/23/2000	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 1/10/2000	ABBOTT HBeAg TEST DATE: 9/8/1999	SORIN ETI-MAK 3 TEST DATE: 12/8/1999	SORIN ETI-MAK 4 TEST DATE: UNK
11016-01	0.130	0.590	NOT TESTED	0.020	0.130
11016-02	0.120	0.500	NOT TESTED	0.000	0.180
11016-03	0.130	0.530	NOT TESTED	0.070	0.150
11016-04	0.120	0.500	NOT TESTED	0.070	0.200
11016-05	0.150	0.560	NOT TESTED	0.090	0.230
11016-06	1.360	1.010	NOT TESTED	0.780	1.380
11016-07	3.980	1.970	NOT TESTED	1.860	3.600
11016-08	29.780	8.510	NOT TESTED	40.180	21.100
11016-09	76.890	20.400	NOT TESTED	28.710	44.980
11016-10	408.660	57.530	0.818	OVER	82.930

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)