



HBV Seroconversion Panel Donor No. 68739
Part Number: HBV11017

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT	ROCHE	GENETIC	DADE	BIORAD	ROCHE
		Normal Range (10-45u/L)	HBV REAL TIME CP/ML	CAP-G/CTM HBV (cp/mL)	SYSTEMS HBsAg (2.0)	BEHRING Enzygnost HBsAg 5.0	MONALISA HBsAg PLUS	Cobas Core HBsAg II EIA
		TEST DATE: 10/27/1998	TEST DATE: UNK	TEST DATE: 6/2/2005	TEST DATE: UNK	TEST DATE: 12/15/1999	TEST DATE: UNK	TEST DATE: 4/18/2000
11017-01	08/26/98	7	NOT DETECTED	NEGATIVE	0.643	0.770	0.430	0.180
11017-02	08/31/98	6	NOT DETECTED	NEGATIVE	0.571	0.210	0.380	0.160
11017-03	09/02/98	9	NOT DETECTED	NEGATIVE	0.643	0.320	0.500	0.160
11017-04	09/11/98	8	NOT DETECTED	NEGATIVE	0.714	0.190	0.570	0.180
11017-05	09/18/98	6	NOT DETECTED	NEGATIVE	0.500	0.290	0.330	0.180
11017-06	09/20/98	4	<52	NEGATIVE	0.500	0.330	0.430	0.180
11017-07	09/27/98	8	66	NEGATIVE	0.571	0.300	0.280	0.220
11017-08	09/29/98	6	96	224	0.714	0.390	0.380	0.220
11017-09	10/05/98	13	318	619	0.643	0.610	0.600	0.480
11017-10	10/07/98	8	799	2230	0.643	0.870	0.820	0.660
11017-11	10/12/98	9	4100	21500	1.429	2.400	1.870	1.810
11017-12	10/15/98	3	5791	32000	2.357	4.760	2.570	3.890
11017-13	10/19/98	3	17896	61100	5.071	11.880	5.180	9.640
11017-14	10/22/98	9	41952	89300	7.857	17.700	11.700	15.380

PANEL MEMBER	ORTHO	ABBOTT	ABBOTT	BIOTEST	ABBOTT	ABBOTT
	VITROS HBsAg Reagent Pack	AxSYM HBsAg (V2) Reagent Pack	PRISM HBsAg	ELISA HBsAg	Auszyme MC Dynamic HBsAg	Auszyme Monoclonal HBsAg
	TEST DATE: 3/23/2000	TEST DATE: 1/10/2000	TEST DATE: UNK	TEST DATE: 4/18/2000	TEST DATE: 12/15/1999	TEST DATE: UNK
11017-01	0.150	0.570	0.39	0.220	0.110	0.055
11017-02	0.190	0.600	0.36	0.240	0.290	0.073
11017-03	0.200	0.420	0.33	0.180	0.260	0.073
11017-04	0.140	0.470	0.39	0.140	0.110	0.109
11017-05	0.130	0.480	0.35	0.180	0.140	0.073
11017-06	0.120	0.430	0.40	0.140	0.140	0.127
11017-07	0.160	0.590	0.45	0.140	0.260	0.182
11017-08	0.190	0.530	0.53	0.220	0.400	0.145
11017-09	0.320	0.650	1.08	0.240	0.690	0.327
11017-10	0.540	0.820	2.62	0.290	0.800	0.673
11017-11	2.180	1.700	2.02	1.140	3.070	1.655
11017-12	5.500	2.920	14.85	1.460	5.250	4.727
11017-13	14.940	6.280	24.83	4.200	13.380	12.182
11017-14	27.520	12.330	46.94	4.970	18.230	14.909

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HBV Seroconversion Panel Donor No. 68739
Part Number: HBV11017**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 4/18/2000	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ABBOTT HBeAg TEST DATE: 9/1/1999	SORIN ETI-MAK 3 TEST DATE: 12/8/1999	SORIN ETI-MAK 4 TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL (Co/S) TEST DATE: 9/1/1999
11017-01	0.040	0.47	NOT TESTED	0.070	0.230	NOT TESTED
11017-02	0.190	0.47	NOT TESTED	0.070	0.250	NOT TESTED
11017-03	0.130	0.63	NOT TESTED	0.090	0.330	NOT TESTED
11017-04	0.160	0.48	NOT TESTED	0.090	0.300	NOT TESTED
11017-05	0.070	0.45	NOT TESTED	0.060	0.250	NOT TESTED
11017-06	0.130	0.50	NOT TESTED	0.060	0.200	NOT TESTED
11017-07	0.190	0.55	NOT TESTED	0.090	0.400	NOT TESTED
11017-08	0.310	0.59	NOT TESTED	0.130	0.300	NOT TESTED
11017-09	0.430	0.96/1.095	NOT TESTED	0.350	0.730	NOT TESTED
11017-10	0.810	1.31	NOT TESTED	0.630	1.030	NOT TESTED
11017-11	2.300	3.55	NOT TESTED	1.560	2.850	NOT TESTED
11017-12	4.810	7.07	NOT TESTED	3.500	5.700	NOT TESTED
11017-13	13.560	19.02	NOT TESTED	8.780	9.850	NOT TESTED
11017-14	23.130	25.80	0.405	11.910	25.570	0.464

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com