



**HBV Seroconversion Panel Donor No. 70793
Part Number: HBV11024**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 6/21/1999	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ROCHE HBV Monitor Assay (copies/mL) TEST DATE: UNK	ROCHE HBV Ampliscreen TEST DATE: UNK	GENETIC SYSTEMS HBsAg 2.0 Static TEST DATE: UNK	ORTHO TM HBsAg ELISA Testsystem 3 TEST DATE: UNK
11024-01	02/03/99	15	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.29
11024-02	02/06/99	6	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.27
11024-03	02/10/99	17	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.21
11024-04	02/14/99	10	NOT DETECTED	NEGATIVE/POSITIVE	NEGATIVE/POSITIVE	NEGATIVE	0.29
11024-05	02/18/99	10	NOT DETECTED	200	POSITIVE	NEGATIVE	0.18
11024-06	02/21/99	7	<52	200	POSITIVE	NEGATIVE	0.21
11024-07	02/25/99	11	70	200	POSITIVE	NEGATIVE	0.21
11024-08	02/28/99	16	259	700	POSITIVE	NEGATIVE	0.27
11024-09	03/04/99	12	450	500	POSITIVE	NEGATIVE	0.24
11024-10	03/07/99	12	763	7,300	POSITIVE	NEGATIVE	0.32
11024-11	03/15/99	15	4929	4,400	POSITIVE	NEGATIVE	0.77
11024-12	03/18/99	9	8371	24,000	POSITIVE	NEGATIVE	0.59
11024-13	03/24/99	16	44953	110,000	POSITIVE	NEGATIVE	2.15
11024-14	03/29/99	13	183128	350,000	POSITIVE	POSITIVE	15.18

PANEL MEMBER	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	BIORAD MONALISA HBsAg PLUS TEST DATE: UNK	VENDOR HBsAg TEST DATE: UNK	BIOTEST ELISA HBsAg TEST DATE: UNK	ABBOTT ARCHITECT HBsAg TEST DATE: UNK	ROCHE ELECSYS HBsAg TEST DATE: UNK	ABBOTT PRISM HBsAg TEST DATE: UNK	ABBOTT MUREX TEST DATE: UNK	DADE BEHRING Enzygnost TEST DATE: UNK
11024-01	0.48	0.28	NEGATIVE	0.15	0.00	0.36	0.68	0.52	0.27
11024-02	0.50	0.26	NEGATIVE	0.13	0.00	0.38	0.56	0.49	0.23
11024-03	0.49	0.30	NEGATIVE	0.13	0.00	0.40	0.30	0.48	0.43
11024-04	0.47	0.28	NEGATIVE	0.21	0.00	0.33	0.45	0.53	0.23
11024-05	0.62	0.30	NEGATIVE	0.19	0.00	0.39	0.34	0.49	0.22
11024-06	0.53	0.34	NEGATIVE	0.17	0.00	0.31	0.37	0.49	0.21
11024-07	0.53	0.26	NEGATIVE	0.15	0.00	0.34	0.69	0.51	0.28
11024-08	0.54	0.36	NEGATIVE	0.15	0.00	0.35	0.60	0.51	0.21
11024-09	0.53	0.26	NEGATIVE	0.19	0.00	0.39	0.73	0.44	0.27
11024-10	0.53	0.30	NEGATIVE	0.19	0.00	0.45	0.74	0.52	0.27
11024-11	1.30	0.47	NEGATIVE	0.36	0.03	1.29	2.21	0.81	0.94
11024-12	1.61	0.42	NEGATIVE	0.34	0.05	1.28	2.27	0.99	0.93
11024-13	3.99	0.74	NEGATIVE	0.87	0.14	3.27	6.31	1.97	2.04
11024-14	26.18	5.43	POSITIVE	9.13	1.12	27.71	48.14	13.57	15.85

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HBV Seroconversion Panel Donor No. 70793
Part Number: HBV11024**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO ANTI-HBc TOTAL S/CO TEST DATE: UNK	ABBOTT Auszyme MC Dynamic TEST DATE: UNK	ORTHO VITROS HBsAg Reagent Pack TEST DATE: 7/8/2002	ABBOTT AxSYM HBsAg V2 Reagent Pack TEST DATE: 5/14/2002	ABBOTT IMx HBsAg V2 Reagent Pack TEST DATE: 7/9/2002	ABBOTT AxSYM HBe 2.0 Reagent Pack S/CO TEST DATE: 5/10/2002	ABBOTT AxSYM CORE S/CO TEST DATE: 6/19/2002	ABBOTT HBsAg Proc B TEST DATE: UNK	ABBOTT HBsAg Proc C TEST DATE: UNK
11024-01	0.989	0.11	0.08	0.45	0.33	0.20	0.21	0.052	0.103
11024-02	0.919	0.07	0.08	0.45	0.37	0.13	0.19	0.086	0.207
11024-03	0.905	0.07	0.08	0.41	0.37	0.10	0.18	0.069	0.172
11024-04	0.932	0.11	0.08	0.43	0.35	0.22	0.18	0.138	0.138
11024-05	0.947	0.07	0.08	0.37	0.25	0.20	0.17	0.086	0.103
11024-06	0.793	0.11	0.09	0.49	0.44	0.23	0.16	0.052	0.207
11024-07	0.923	0.11	0.09	0.37	0.33	0.14	0.17	0.138	0.310
11024-08	0.793	0.07	0.09	0.37	0.26	0.13	0.17	0.052	0.241
11024-09	0.749	0.11	0.09	0.38	0.40	0.16	0.16	0.086	0.069
11024-10	0.684	0.11	0.10	0.50	0.38	0.15	0.14	0.121	0.034
11024-11	0.844	0.86	0.46	0.73	1.10	0.17	0.17	0.569	0.586
11024-12	0.554	1.00	0.51	0.79	1.03	0.28	0.13	0.655	0.552
11024-13	0.613	2.79	2.01	1.50	2.85	0.81	0.14	1.948	1.276
11024-14	0.613	23.11	24.10	9.77	21.68	2.16	0.13	12.241	9.931

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com