



**HBV Seroconversion Panel Donor No.61248
Part Number: HBV6271**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 11/15/1996	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	BIOMERIEUX VIDAS HBsAg TEST DATE: 1/16/1997	BOEHR. MANNHEIM ELECSYS HBsAg TEST DATE: 10/9/1997	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 4/28/1998	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 7/2/1997
6271-01	03/16/1996	9	215	0.16	0.53	0.57	0.20
6271-02	03/19/1996	14	763	0.25	0.72	0.49	0.43
6271-03	03/23/1996	11	2359	0.69	1.43	0.60	1.36
6271-04	03/28/1996	10	19623	3.96	7.11	1.35	8.67
6271-05	04/03/1996	13	167015	32.68	50.94	7.59	58.70

PANEL MEMBER	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	SORIN ETI-MAK 3 HBsAg TEST DATE: 1/16/1997	ABBOTT PRISM HBsAg TEST DATE: 2/5/1997	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ORTHO AMERLITE HBsAg II TEST DATE: 7/14/1997	ABBOTT AxSYM HBsAg TEST DATE: 1/6/1997	ABBOTT Austria II-125 TEST DATE: 12/3/1996
6271-01	119	0.35	0.39	0.57	0.09	0.50	0.32
6271-02	2150	0.43	0.70	0.80	0.15	0.64	0.38
6271-03	8660	0.82	1.81	1.69	0.39	0.91	0.51
6271-04	78100	3.38	9.49	9.06	3.98	3.11	1.67
6271-05	372000	19.38	56.7	32.61	38.55	19.24	12.67

PANEL MEMBER	ABBOTT ANTI-HBe (EIA) (Co/S) TEST DATE: 12/17/1996	ABBOTT ANTI-HBc TOTAL (EIA) (Co/S) TEST DATE: 12/17/1996	ABBOTT Auszyme MC Dynamic TEST DATE: 1/21/1997	DIASORIN HBc IgM TEST DATE: 10/5/2011	MUREX HBsAg TEST DATE: 1/9/1997	ABBOTT Auszyme Monoclonal TEST DATE: 9/30/1997	ABBOTT HBsAg (EIA) TEST DATE: 11/15/1996
6271-01	0.526	0.426	0.45	0.21	0.68	0.19	0.158
6271-02	0.560	0.448	0.64	0.17	0.60	0.28	0.263
6271-03	0.593	0.451	1.70	0.17	0.73	0.82	0.895
6271-04	0.556	0.429	9.85	0.17	1.95	5.81	6.491
6271-05	0.549	0.417	47.61	0.17	10.35	36.35	>35.088

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HBV Seroconversion Panel Donor No.61248
Part Number: HBV6271**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT HBeAg (EIA) TEST DATE: 12/17/1996	ABBOTT IMx HBsAg Reagent Pack TEST DATE: 3/13/1997	BIOTEST ELISA HBsAg TEST DATE: 3/3/1997	SANOFI Monolisa HBsAg 2 nd Generation TEST DATE: 3/6/1997	ROCHE Cobas Core HBsAg II (EIA) TEST DATE: 12/17/1996	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 12/19/1996
6271-01	0.339	0.47	0.20	0.49	0.28	0.49
6271-02	0.339	0.53	0.28	0.46	0.47	0.46
6271-03	0.421	0.63	0.67	0.89	0.95	0.69
6271-04	0.314	2.01	4.18	2.40	6.10	2.40
6271-05	0.438	9.50	23.76	14.99	41.99	14.99

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com