



**HBV Seroconversion Panel Donor No.61291
Part Number: HBV6272**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 11/18/1996	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	BIOMERIEUX VIDAS HBsAg TEST DATE: 1/16/1997	BOEHR. MANNHEIM ELECSYS HBsAg TEST DATE: 10/9/1997	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 12/18/1996	ORTHO VITROS ECI HBsAg ELISA Testsystem 3 TEST DATE: 12/16/1996	MUREX HBsAg TEST DATE: 1/9/1997	SORIN ETI- MAK 3 HBsAg TEST DATE: 2/17/1997
6272-01	12/17/95	13	70	0.13	0.48	0.44	0.03	0.54	0.09
6272-02	12/19/95	15	55	0.08	0.40	0.53	0.06	0.53	0.05
6272-03	12/24/95	13	59	0.08	0.44	0.50	0.12	0.50	0.04
6272-04	01/09/96	9	68	0.11	0.39	0.45	0.09	0.56	0.04
6272-05	01/11/96	8	113	0.17	0.42	0.49	0.09	0.52	0.05
6272-06	01/16/96	3	70	0.10	0.45	0.54	0.09	0.58	0.09
6272-07	01/18/96	10	105	0.10	0.48	0.48	0.09	0.57	0.05
6272-08	01/23/96	13	113	0.11	0.39	0.45	0.01	0.65	0.11
6272-09	01/25/96	13	88	0.10	0.44	0.41	0.03	0.54	0.07
6272-10	01/30/96	12	82	0.08	0.45	0.47	0.01	0.52	0.11
6272-11	02/01/96	11	124	0.12	0.49	0.42	0.20	0.49	0.04
6272-12	02/06/96	7	118	0.12	0.45	0.45	0.03	0.55	0.02
6272-13	02/08/96	5	80	0.13	0.43	0.47	0.17	0.71	0.13
6272-14	02/13/96	21	142	0.10	0.45	0.46	0.12	0.62	0.05
6272-15	02/15/96	14	52	0.10	0.49	0.45	0.20	0.53	0.07
6272-16	02/20/96	15	84	0.15	0.49	0.45	0.06	0.55	0.09
6272-17	02/22/96	6	NOT TESTED	0.10	0.47	0.42	0.04	0.57	0.07
6272-18	02/27/96	14	133	0.13	0.52	0.48	0.01	0.57	0.15
6272-19	02/29/96	10	142	0.15	0.55	0.49	0.13	0.52	0.05
6272-20	03/20/96	12	341	0.24	0.98	0.57	0.12	0.57	0.09
6272-21	03/23/96	9	696	0.27	1.46	0.50	0.20	0.63	0.09
6272-22	03/27/96	10	961	0.35	2.07	0.54	0.41	0.68	0.11
6272-23	03/30/96	10	781	0.52	2.83	0.61	0.99	0.69	0.07
6272-24	04/03/96	3	746	0.97	4.25	0.66	1.10	0.92	0.22
6272-25	04/06/96	6	1358	1.71	7.46	0.87	2.84	1.08	0.35
6272-26	04/10/96	6	2253	3.47	13.64	1.33	4.32	1.49	0.75

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HBV Seroconversion Panel Donor No.61291
Part Number: HBV6272**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HBsAg TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ORTHO VITROS ECI AMERLITE HBsAG II TEST DATE: 1/13/1997	ABBOTT AxSYM HBsAg Reagent Pack TEST DATE: 1/6/1997	ABBOTT Austria II-125 TEST DATE: 12/3/1996	ABBOTT ANTI-HBc TOTAL (EIA) (Co/S) TEST DATE: 12/17/1996	ABBOTT PRISM HBsAg TEST DATE: 2/5/1997	ABBOTT Auszyme MC Dynamic TEST DATE: 1/21/1997	DIASORIN ANTI-HBc IgM TEST DATE: UNK
6272-01	1.09	0.49	0.14	0.46	0.41	0.527	0.49	0.18	NOT TESTED
6272-02	0.74	0.50	0.13	0.45	0.44	0.485	0.33	0.24	NOT TESTED
6272-03	0.80	0.50	0.14	0.41	0.42	0.385	0.28	0.18	NOT TESTED
6272-04	0.78	0.50	0.12	0.45	0.32	0.486	0.52	0.21	NOT TESTED
6272-05	0.69	0.50	0.12	0.48	0.41	0.480	0.38	0.18	NOT TESTED
6272-06	0.37	0.53	0.13	0.40	0.41	0.497	0.37	0.21	NOT TESTED
6272-07	0.99	0.53	0.13	0.45	0.33	0.534	0.37	0.21	NOT TESTED
6272-08	0.72	0.50	0.11	0.46	0.40	0.520	0.43	0.18	NOT TESTED
6272-09	0.68	0.51	0.13	0.41	0.29	0.434	0.42	0.21	0.2
6272-10	0.79	0.49	0.13	0.48	0.43	0.437	0.36	0.21	0.2
6272-11	0.48	0.50	0.10	0.56	0.36	0.525	0.47	0.21	0.2
6272-12	0.94	0.53	0.14	0.48	0.38	0.366	0.46	0.21	0.2
6272-13	0.84	0.54	0.12	0.49	0.29	0.483	0.47	0.21	0.2
6272-14	0.76	0.53	0.13	0.49	0.31	0.339	0.41	0.24	0.2
6272-15	0.90	0.56	0.12	0.45	0.30	0.516	0.39	0.30	0.2
6272-16	0.48	0.64	0.12	0.53	0.34	0.424	0.73	0.30	0.2
6272-17	0.93	0.62	0.10	0.54	0.34	0.510	0.56	0.30	0.2
6272-18	1.23	0.64	0.09	0.54	0.37	0.432	0.64	0.30	0.2
6272-19	1.80	0.69	0.09	0.56	0.33	0.526	0.61	0.33	0.2
6272-20	2.97	1.31	0.13	0.90	0.47	0.337	1.60	1.39	0.2
6272-21	4.17	1.78	0.15	1.21	0.39	0.448	2.23	2.30	0.2
6272-22	5.31	2.48	0.17	1.58	0.39	0.437	2.77	3.33	0.1
6272-23	6.17	3.24	0.20	1.82	0.49	0.331	3.57	5.15	0.2
6272-24	9.38	5.31	0.31	2.79	0.59	0.492	6.03	8.30	0.2
6272-25	14.48	7.26	0.49	4.30	0.84	0.469	10.35	15.24	0.2
6272-26	22.06	11.50	0.79	6.23	1.40	0.437	11.38	22.58	0.2

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



**HBV Seroconversion Panel Donor No.61291
Part Number: HBV6272**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DIASORIN HBc TOTAL TEST DATE: UNK	ABBOTT Auszyme Monoclonal TEST DATE: 9/30/1997	ABBOTT HBsAg (EIA) TEST DATE: 11/19/1996	DIASORIN ANTI-HBs QUANT TEST DATE: UNK	ABBOTT IMx HBsAg Reagent Pack TEST DATE: 3/13/1997	BIOTEST ELISA HBsAg TEST DATE: 3/3/1997	SANOFI Monolisa HBsAg 2 nd Generation TEST DATE: 3/6/1997	ROCHE Cobas Core HBsAg II (EIA) TEST DATE: 12/17/1996	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 12/19/1996
6272-01	4.13	0.07	0.050	3.4	0.45	0.18	0.38	0.15	0.69
6272-02	3.80	0.06	0.050	NOT TESTED	0.44	0.16	0.35	0.19	0.31
6272-03	4.20	0.07	0.067	NOT TESTED	0.41	0.16	0.40	0.19	0.55
6272-04	2.86	0.06	0.067	NOT TESTED	0.46	0.18	0.49	0.28	0.54
6272-05	3.53	0.09	0.067	NOT TESTED	0.49	0.22	0.42	0.30	0.59
6272-06	3.60	0.07	0.083	NOT TESTED	0.49	0.16	0.42	0.19	0.56
6272-07	3.53	0.07	0.083	NOT TESTED	0.41	0.28	0.40	0.19	0.55
6272-08	3.86	0.15	0.067	NOT TESTED	0.48	0.28	0.40	0.21	0.77
6272-09	4.07	0.07	0.083	NOT TESTED	0.47	0.26	0.35	0.19	0.55
6272-10	4.13	0.06	0.100	NOT TESTED	0.48	0.24	0.36	0.19	0.64
6272-11	3.60	0.11	0.133	NOT TESTED	0.55	0.20	0.36	0.30	0.67
6272-12	3.73	0.11	0.133	NOT TESTED	0.46	0.24	0.69	0.15	0.87
6272-13	3.67	0.09	0.183	NOT TESTED	0.44	0.20	0.42	0.15	0.47
6272-14	4.13	0.11	0.150	NOT TESTED	0.52	0.22	0.53	0.15	0.42
6272-15	3.67	0.11	0.133	NOT TESTED	0.49	0.30	0.42	0.28	0.49
6272-16	3.80	0.17	0.183	NOT TESTED	0.52	0.30	0.42	0.21	0.58
6272-17	3.53	0.13	0.183	NOT TESTED	0.48	0.24	0.33	0.19	0.44
6272-18	3.46	0.13	0.333	NOT TESTED	0.47	0.28	0.35	0.21	0.33
6272-19	3.80	0.22	0.350	NOT TESTED	0.53	0.26	0.38	0.19	0.35
6272-20	3.80	1.08	1.767	NOT TESTED	0.69	0.32	0.47	0.42	0.71
6272-21	3.73	1.77	2.567	NOT TESTED	0.75	0.38	0.62	0.70	0.74
6272-22	3.86	2.87	3.967	NOT TESTED	1.52	0.34	0.87	0.89	1.22
6272-23	4.13	2.81	5.000	NOT TESTED	1.16	0.63	0.60	1.42	1.56
6272-24	4.40	5.25	8.767	NOT TESTED	1.50	1.05	0.76	2.94	1.15
6272-25	4.13	9.52	10.283	NOT TESTED	2.30	1.76	1.40	5.46	1.83
6272-26	4.20	13.86	17.467	NOT TESTED	3.19	2.89	3.05	9.42	3.08

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com