



**HBV Seroconversion Panel Donor No.61042**  
**Part Number: HBV6273**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 3/01/1996	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	BIOMERIEUX VIDAS HBsAg  TEST DATE: 1/16/1997	BOEHR. MANNHEIM ELECSYS HBsAg  TEST DATE: 10/9/1997	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II  TEST DATE: 12/18/1996	ORTHO VITROS Eci HBsAg ELISA Testsystem 3  TEST DATE: 12/16/1996	MUREX HBsAg  TEST DATE: 1/9/1997
6273-01	01/29/96	12	NOT DETECTED	0.21	0.40	0.50	0.26	0.52
6273-02	02/01/96	8	<52	0.12	0.39	0.59	0.06	0.51
6273-03	02/05/96	12	136	0.16	0.39	0.48	0.23	0.65
6273-04	02/12/96	15	1422	0.42	0.64	0.54	0.38	0.62
6273-05	02/23/96	16	20080	6.75	7.33	1.91	7.28	2.36
6273-06	02/28/96	10	215156	39.17	61.29	10.48	55.07	12.11

PANEL MEMBER	SORIN ETI- MAK 3 HBsAg  TEST DATE: 1/16/1997	ABBOTT PRISM HBsAg  TEST DATE: 8/27/2009	ABBOTT Murex HBsAg Version 3  TEST DATE: 8/27/2009	ORTHO VITROS Eci AMERLITE HBsAg II  TEST DATE: 1/13/1997	ABBOTT AxSYM HBsAg Reagent  TEST DATE: 1/16/1997	ABBOTT Austria II-125  TEST DATE: 12/3/1996	ABBOTT Anti-HBc TOTAL (EIA) (Co/S)  TEST DATE: 12/17/1996	ABBOTT Auszyme MC Dynamic  TEST DATE: 1/21/1997
6273-01	0.15	0.57	0.48	0.07	0.67	0.44	0.490	0.21
6273-02	0.18	1.15	0.55	0.07	0.49	0.32	0.463	0.30
6273-03	0.18	1.62	0.53	0.08	0.53	0.47	0.447	0.27
6273-04	0.42	1.66	0.92	0.17	1.10	0.57	0.486	0.76
6273-05	7.08	23.06	7.67	5.81	5.19	3.46	0.426	13.00
6273-06	34.85	118.26	34.13	42.48	26.47	17.55	0.534	OVER

PANEL MEMBER	DIASORIN HBc IgM  TEST DATE: 10/5/2011	ABBOTT Auszyme Monoclonal  TEST DATE: 9/30/1997	ABBOTT HBsAg (EIA)  TEST DATE: 11/19/1996	ABBOTT IMx HBsAg Reagent Pack  TEST DATE: 3/13/1997	BIOTEST ELISA HBsAg  TEST DATE: 3/3/1997	SANOFI Monalisa HBsAg 2 <sup>nd</sup> Generation  TEST DATE: 3/6/1997	ROCHE CAP-G/CTM HBV (cp/mL)  TEST DATE: 6/2/2005	BEHRINGWERKE Enzygnost HBsAg Monoclonal II  TEST DATE: 12/19/1996
6273-01	0.17	0.15	0.067	0.52	0.32	0.38	NEGATIVE	0.74
6273-02	0.17	0.13	0.117	0.51	0.20	0.47	NEGATIVE	0.45
6273-03	0.21	0.15	0.117	0.47	0.22	0.51	NOT TESTED	0.36
6273-04	0.17	0.34	0.300	0.93	0.34	0.58	5090	0.87
6273-05	0.17	6.47	5.800	3.21	4.57	3.87	69200	3.13
6273-06	0.17	36.73	>33.333	12.49	26.95	24.8	416000	16.88

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)