



HBV Seroconversion Panel Donor No.61066
Part Number: HBV6275

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 11/15/1996	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	BIOMERIEUX VIDAS HBsAg TEST DATE: 1/16/1997	BOEHR. MANNHEIM ELECSYS HBsAg TEST DATE: 10/9/1997	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 12/18/1996	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 12/16/1996	MUREX HBsAg TEST DATE: 1/9/1997
6275-01	02/06/96	8	98	0.15	0.41	0.44	0.06	0.55
6275-02	02/08/96	5	75	0.14	0.50	0.46	0.17	0.54
6275-03	02/13/96	4	<52	0.33	0.90	0.55	0.43	0.65
6275-04	02/15/96	5	163	0.40	0.90	0.51	0.70	0.57
6275-05	02/28/96	8	1670	3.15	6.48	1.29	5.68	1.53
6275-06	03/04/96	7	5926	8.54	17.21	2.78	16.41	3.38
6275-07	03/06/96	1	10064	11.15	28.41	3.53	23.36	4.09

PANEL MEMBER	SORIN ETI- MAK 3 HBsAg TEST DATE: 1/16/1997	ABBOTT PRISM HBsAg TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3 TEST DATE: 8/27/2009	ORTHO AMERLITE HBsAg II TEST DATE: 1/13/1997	ABBOTT AxSYM HBsAg TEST DATE: 1/6/1997	ABBOTT Austria II-125 TEST DATE: 12/3/1996	ABBOTT ANTI-HBc TOTAL (EIA) (Co/S) TEST DATE: 12/17/1996	ABBOTT HBeAg (EIA) TEST DATE: 12/17/1996
6275-01	0.12	1.26	0.53	0.47	0.55	0.32	0.523	0.314
6275-02	0.25	1.07	0.58	0.09	0.54	0.32	0.418	0.298
6275-03	0.46	2.25	0.90	0.16	0.90	0.41	0.550	0.264
6275-04	0.63	2.15	0.99	0.20	0.79	0.39	0.531	0.322
6275-05	4.52	12.83	5.72	2.70	3.44	1.85	0.499	0.355
6275-06	12.4	31.50	15.59	9.94	8.88	4.64	0.459	0.322
6275-07	18.03	43.59	22.01	13.01	11.58	5.69	0.515	0.380

PANEL MEMBER	ABBOTT Auszyme MC Dynamic TEST DATE: 1/21/1997	ABBOTT Auszyme Monoclonal TEST DATE: 10/21/1997	ROCHE Cobas Core HBsAg II (EIA) TEST DATE: 12/17/1996	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 12/19/1996	DIASORIN HbC IgM TEST DATE: 10/5/2011	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005
6275-01	0.36	0.15	0.25	0.45	0.17	407
6275-02	0.36	0.15	0.21	0.41	0.17	NOT TESTED
6275-03	0.79	0.37	0.57	0.40	0.17	NEGATIVE
6275-04	0.97	0.42	0.63	0.42	0.17	NEGATIVE
6275-05	8.61	4.75	5.27	1.99	0.17	NOT TESTED
6275-06	24.91	12.04	15.2	5.31	0.17	25500
6275-07	31.55	21.26	22.12	7.28	0.17	20500

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com