



**HBV Seroconversion Panel Donor No.63701**  
**Product Code: HBV6279**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	RESEARCH USE PCR	ROCHE CAP-G/CTM HBV (cp/mL)	BIOMERIEUX VIDAS HBsAg (>0.13 reactive)	BOEHR. MANNHEIM HBsAg Enzymum-test ES Systems	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3
		TEST DATE: 3/20/1997	TEST DATE: UNK	TEST DATE: 6/14/1998	TEST DATE: 6/2/2005	TEST DATE: 10/21/1997	TEST DATE: 10/9/1997	TEST DATE: 10/7/1997	TEST DATE: 10/30/1997
6279-01	03-20-97	2	IND	<100	278	0.02	QNS	0.46	0.25
6279-02	04-01-97	1	76	<100	NEG	0.01	0.31	0.42	0.22
6279-03	04-03-97	5	<52	<100	ONS	0.01	QNS	0.50	0.19
6279-04	04-08-97	5	304	100	75.3	0.02	0.36	0.49	0.27
6279-05	04-10-97	4	439	100	529	0.03	0.34	0.55	0.30
6279-06	04-15-97	3	1632	200	5110	0.08	0.72	0.60	0.77
6279-07	04-17-97	7	3332	2000	10800	0.20	0.97	0.80	1.91

PANEL MEMBER	ABBOTT MUREX HBsAg	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	BIOTEST ELISA HBsAg	ABBOTT AxSYM HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT HBsAg (EIA)	ABBOTT IMx HBsAg Reagent Pack	BOEHR. MANNHEIM ELECSYS HBsAg
	TEST DATE: 10/29/1997	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 11/5/1997	TEST DATE: 10/22/1997	TEST DATE: UNK	TEST DATE: 12/5/1997	TEST DATE: 10/30/1997	TEST DATE: 10/8/1997
6279-01	0.51	0.82	0.49	0.14	0.48	0.14	0.048	0.40	0.54
6279-02	0.46	0.75	0.49	0.16	0.43	0.13	0.048	0.41	0.48
6279-03	0.37	0.59	0.54	0.16	0.43	0.13	0.071	0.43	0.43
6279-04	0.39	0.73	0.66	0.19	0.49	0.15	0.071	0.44	0.50
6279-05	0.39	0.84	0.70	0.17	0.51	0.21	0.024	0.50	0.61
6279-06	0.55	2.33	1.31	0.62	0.74	0.62	0.119	0.60	1.21
6279-07	0.86	4.23	2.38	0.93	1.31	1.91	0.286	0.95	2.23

PANEL MEMBER	BEHRINGWE RKE Enzygnost HBsAg Monoclonal II	DIASORIN ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBc IgM	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT
	TEST DATE: 10/16/1997	TEST DATE: 10/5/2011	TEST DATE: UNK	TEST DATE: 12/5/1997	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 12/5/1997	TEST DATE: UNK
6279-01	0.82	0.17	0.01	0.067	2.19	0.21	0.16	0.143	0.20
6279-02	0.30	0.17	0.01	0.044	2.5	0.24	0.16	0.143	1.60
6279-03	0.30	0.17	0.01	0.040	2.58	0.24	0.16	0.111	0.10
6279-04	0.33	0.17	0.01	0.042	2.49	0.22	0.16	0.143	0.40
6279-05	0.36	0.17	0.01	0.040	2.7	0.21	0.16	0.111	0.50
6279-06	0.47	0.17	0.01	0.057	2.61	0.22	0.16	0.175	0.80
6279-07	0.78	0.21	0.01	0.044	2.56	0.20	0.16	0.127	2.30

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)