



**HBV Seroconversion Panel Donor No. 61832**  
**Product Code: HBV6280**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	RESEARCH USE PCR	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3	ROCHE Cobas Core HBsAg II EIA
		TEST DATE: 1/21/1998	TEST DATE: UNK	TEST DATE: 6/2/1998	TEST DATE: 4/22/1998	TEST DATE: 4/23/1998	TEST DATE: 4/23/1998	TEST DATE: 3/18/1998	TEST DATE: 4/23/1998
6280-01	07-03-96	2	429	4000	0.12	0.36	0.29	0.13	0.24
6280-02	07-16-96	9	15232	21000	1.10	1.25	1.32	2.73	2.00
6280-03	07-18-96	2	20080	19000	2.14	2.10	2.17	5.13	3.22
6280-04	07-24-96	4	126694	280000	9.10	12.06	8.55	27.71	13.17
6280-05	07-30-96	*	1053791	470000	45.01	51.97	OVER	OVER	62.63

PANEL MEMBER	GENETIC SYSTEMS HBsAg (2.0)	ABBOTT AxSYM Core-M	ABBOTT AxSYM Core (Co/S)	ABBOTT ANTI-HBc IgM (Co/S)	ABBOTT HBeAg (Co/S)	BEHRINGWERKE Enzygnost HBsAg Monoclonal II	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	BIOTEST ELISA HBsAg	ABBOTT AxSYM HBsAg	ORTHO VITROS ECi HBsAg
	TEST DATE: 25/4/1998	TEST DATE: 4/7/1998	TEST DATE: 4/7/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/4/1998	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 4/6/1998	TEST DATE: 3/19/1998	TEST DATE: UNK
6280-01	0.236	0.04	0.49	0.045	0.254	0.35	1.52	0.54	0.26	0.49	0.14
6280-02	0.966	0.03	0.48	0.050	0.808	1.04	4.73	2.89	1.32	1.31	2.42
6280-03	1.315	0.03	0.49	0.047	0.338	1.51	9.21	5.31	2.17	1.94	5.1
6280-04	4.685	0.04	0.52	0.049	0.338	6.10	47.43	24.70	10.37	9.01	32.5
6280-05	17.674	0.04	0.51	0.050	1.015	29.45	127.67	OVER	32.70	30.29	142

PANEL MEMBER	ABBOTT Auszyme MC Dynamic	ABBOTT Auszyme Monoclonal HBsAg	SORIN ETI-MAK 3 HBsAg	SORIN (INCSTAR) HBsAg	SORIN ETI- Core IGM2	SORIN ETI-AB CoreK2 (Co/S)	ABBOTT ANTI-HBc TOTAL (Co/S)	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc IgM
	TEST DATE: 3/31/1998	TEST DATE: 4/30/1998	TEST DATE: 5/5/1998	TEST DATE: UNK	TEST DATE: 5/7/1998	TEST DATE: 5/7/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6280-01	0.16	0.07	0.13	0.235	0.05	0.18	<0.23	3.19	0.01
6280-02	2.24	1.04	1.27	0.549	0.03	0.16	<0.23	3.08	0.01
6280-03	5.77	1.69	2.69	0.725	0.01	0.16	0.32	3.07	0.01
6280-04	26.74	8.87	11.45	2.435	0.01	0.15	0.34	3.31	0.01
6280-05	OVER	41.73	OVER	8.470	0.02	0.18	<0.23	3.17	0.01

PANEL MEMBER	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBs QUANT
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6280-01	0.21	0.17	2.2
6280-02	0.26	0.16	2.8
6280-03	0.29	0.17	1.6
6280-04	0.76	0.15	1.4
6280-05	3.54	0.13	2.9

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)