



HBV Seroconversion Panel Donor No. 62675
Part Number: HBV6282

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT	BIOMERIEUX	BOEHR.	ORGANON	ORTHO	ORTHO	ROCHE
		Normal Range (10-45u/L)	HBV REAL TIME CP/ML	VIDAS HBsAg	MANNHEIM HBsAg Enzymun-test ES Systems	TEKNIKA Hepanostika HBsAg Uni-Form II	HBsAg ELISA Testsystem 3	VITROS ECi HBsAg	Cobas Core HBsAg II EIA
		TEST DATE: 1/27/1998	TEST DATE: UNK	TEST DATE: 4/22/1998	TEST DATE: 4/23/1998	TEST DATE: 4/23/1998	TEST DATE: 3/18/1998	TEST DATE: 11/3/2015	TEST DATE: 4/23/1998
6282-01	10/23/96	7	96	0.11	0.40	0.45	0.01	0.06	0.15
6282-02	10/25/96	6	163	0.07	0.38	0.33	0.07	0.05	0.19
6282-03	10/30/96	6	553	0.07	0.36	0.36	0.07	0.06	0.26
6282-04	11/04/96	7	3257	0.20	0.47	0.42	0.10	0.07	0.35
6282-05	11/06/96	10	4929	0.26	0.47	0.49	0.27	0.15	0.67
6282-06	11/11/96	7	17896	0.96	0.85	0.97	1.27	1.19	1.41
6282-07	11/13/96	8	40064	1.57	1.23	1.33	2.48	2.4	2.56
6282-08	11/18/96	15	220168	7.80	5.38	5.96	12.69	17.9	11.28
6282-09	11/20/96	5	400640	10.67	7.64	9.03	17.83	24.8	15.46
6282-10	11/25/96	9	2151565	37.76	24.91	23.01	51.65	NOT TESTED	44.63
6282-11	11/27/96	8	4816753	65.34	58.47	MAX.ABS.	MAX.ABS.	179	MAX.ABS.
6282-12	12/02/96	12	26470027	102.73	MAX.ABS.	MAX.ABS.	MAX.ABS.	870	MAX.ABS.
6282-13	12/05/96	11	100636235	108.80	MAX.ABS.	MAX.ABS.	MAX.ABS.	1910	MAX.ABS.
6282-14	12/09/96	14	837055740	99.54	MAX.ABS.	MAX.ABS.	MAX.ABS.	5770	MAX.ABS.

PANEL MEMBER	GENETIC SYSTEMS	ABBOTT	ABBOTT	ORTHO	ABBOTT	ABBOTT	ORTHO	ABBOTT
	HBsAg (2.0)	AxSYM Core-M	ANTI-HBc TOTAL (Co/S)	VITROS ECi ANTI-HBc TOTAL	ANTI-HBc IgM (Co/S)	HBeAg (Co/S)	VITROS ECi HBeAg	PRISM HBsAg
	TEST DATE: 2/4/1998	TEST DATE: 4/7/1998	TEST DATE: UNK	TEST DATE: 11/3/2015	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/3/2015	TEST DATE: 8/27/2009
6282-01	0.117	0.05	0.26	2.76	0.094	0.277	0.22	1.13
6282-02	0.170	0.04	0.24	2.76	0.094	0.246	0.23	1.08
6282-03	0.191	0.06	0.40	2.67	0.094	0.238	0.23	1.16
6282-04	0.202	0.04	0.43	2.57	0.075	0.246	0.24	1.41
6282-05	0.234	0.04	0.52	2.76	0.094	0.323	0.26	1.66
6282-06	0.670	0.04	0.51	2.62	0.075	0.292	0.55	3.92
6282-07	1.170	0.04	0.42	2.52	0.075	0.346	0.61	6.95
6282-08	5.362	0.04	0.26	2.80	0.094	1.892	5.90	30.14
6282-09	6.915	0.04	0.38	2.73	0.083	1.523	6.71	40.44
6282-10	15.777	0.04	0.33	NOT TESTED	0.075	4.069	NOT TESTED	32.14
6282-11	20.872	0.04	0.36	2.51	0.075	5.608	107	18.49
6282-12	25.766	0.04	0.32	2.55	0.075	10.885	706	371.88
6282-13	28.277	0.04	0.39	2.69	0.075	12.077	2070	400.85
6282-14	28.957	0.06	0.53	2.16	0.075	14.615	3370	498.56

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HBV Seroconversion Panel Donor No. 62675
Part Number: HBV6282

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT MUREX HBsAg Version 3 TEST DATE: 8/27/2009	BIOTEST ELISA HBsAg TEST DATE: 4/6/1998	RESEARCH USE PCR PCR (copies/ml) TEST DATE: 6/2/1998	ABBOTT AxSYM HBsAg Reagent Pack TEST DATE: 3/19/1998	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/31/1998	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: 4/30/1998	ORTHO VITROS Eci ANTI-HBe TEST DATE: 11/3/2015	ORTHO VITROS Eci ANTI- HBs QUANT TEST DATE: 11/3/2015
6282-01	0.52	0.22	500	0.54	0.00	0.07	0.16	1.8
6282-02	0.90	0.22	500	0.53	0.10	0.27	QNS	2.2
6282-03	0.59	0.22	500	0.56	0.00	0.24	0.15	2.3
6282-04	0.68	0.25	7100	0.58	0.07	0.24	0.15	1.7
9282-05	0.85	0.31	8200	0.65	0.36	0.20	0.15	1.1
6282-06	1.89	0.63	65000	1.10	1.12	0.51	0.15	1.6
6282-07	3.28	1.17	19000	1.35	1.71	1.01	QNS	2.2
6282-08	13.77	5.23	200000	5.18	11.20	3.84	0.12	QNS
6282-09	18.27	6.90	750000	6.03	14.93	4.52	QNS	QNS
6282-10	32.51	17.17	2700000	19.05	61.95	29.94	NOT TESTED	NOT TESTED
6282-11	33.40	24.40	6700000	28.95	MAX.ABS.	43.82	0.03	1.9
6282-12	33.15	28.82	31000000	58.77	MAX.ABS.	MAX.ABS.	0.01	1.1
6282-13	32.63	30.16	43000000	75.18	MAX.ABS.	MAX.ABS.	0.00	1.5
6282-14	32.85	29.24	>50000000	107.22	MAX.ABS.	MAX.ABS.	0.00	1.7

PANEL MEMBER	ORTHO VITROS Eci ANTI-HBc IgM TEST DATE: 11/3/2015	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 5/4/1998	SORIN ETI-MAK 3 HBsAg TEST DATE: 5/5/1998	SORIN (INCSTAR) HBsAg TEST DATE: UNK	SORIN ETI-Core IGMK2 TEST DATE: 5/7/1998	SORIN ETI-AB CoreK2 (Co/S) TEST DATE: 5/7/1998
6282-01	0.03	0.49	0.50	0.235	0.12	0.27
6282-02	0.02	0.41	0.17	0.235	0.01	0.27
6282-03	0.03	0.39	0.34	0.118	0.02	0.28
6282-04	0.03	0.38	0.50	0.118	0.06	0.28
9282-05	0.03	0.42	0.31	0.157	0.06	0.24
6282-06	0.02	0.85	0.59	0.275	0.06	0.22
6282-07	0.03	1.29	1.75	0.353	0.06	0.22
6282-08	QNS	5.22	8.90	1.686	0.06	0.28
6282-09	QNS	6.57	10.71	2.431	0.02	0.26
6282-10	NOT TESTED	19.49	23.56	6.000	0.00	0.26
6282-11	0.04	33.88	MAX.ABS.	8.080	0.00	0.27
6282-12	0.03	31.97	MAX.ABS.	4.750	0.00	0.26
6282-13	0.03	MAX.ABS.	MAX.ABS.	4.157	0.00	0.25
6282-14	0.04	31.71	MAX.ABS.	4.666	0.01	0.30

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com