



HBV Seroconversion Panel Donor No. 62825
Product Code: HBV6283

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	RESEARCH USE PCR (copies/ml)	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3	ROCHE Cobas Core HBsAg II EIA
		TEST DATE: 1/27/1998	TEST DATE: UNK	TEST DATE: 6/2/1998	TEST DATE: 4/22/1998	TEST DATE: 4/27/1998	TEST DATE: 4/23/1998	TEST DATE: 3/18/1998	TEST DATE: 4/23/1998
6283-01	11/05/96	5	NOT DETECTED	100	0.16	0.13	0.38	0.15	0.24
6283-02	11/07/96	0	NOT DETECTED	<100	0.13	0.36	0.38	0.15	0.20
6283-03	11/13/96	3	<52	<100	0.17	0.33	0.39	0.10	0.17
6283-04	11/16/96	5	NOT DETECTED	100	0.16	0.38	0.39	0.07	0.15
6283-05	11/21/96	8	70	100	0.13	0.38	0.35	0.21	0.22
6283-06	11/24/96	6	82	500	0.18	0.29	0.32	0.15	0.15
6283-07	12/01/96	5	1182	2000	0.32	0.58	0.64	0.60	1.02/0.55
6283-08	12/04/96	10	2836	6000	0.68	0.80	1.15	1.30	1.15/0.84
6283-09	12/12/96	8	22017	100000	7.57	6.32	9.15	14.14	11.81
6283-10	12/15/96	12	71245	160000	18.19	13.71	17.57	28.99	23.22
6283-11	12/20/96	15	449526	600000	56.76	54.65	MAX.ABS.	80.91	MAX.ABS.

PANEL MEMBER	GENETIC SYSTEMS HBsAg (2.0)	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	BEHRING WERKE Enzygnost HBsAg Monoclonal II	BIOTEST ELISA HBsAg	ABBOTT AxSYM HBsAg	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	ORTHO VITROS ECi HBsAg
	TEST DATE: 2/4/1998	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 5/4/1998	TEST DATE: 4/6/1998	TEST DATE: 3/19/1998	TEST DATE: 3/31/1998	TEST DATE: 4/30/1998	TEST DATE: 8/10/2010
6283-01	0.170	0.69	0.65	0.51	0.47	0.54	0.07	0.28	0.24
6283-02	0.159	0.71	0.70	0.51	0.32	0.52	0.07	0.24	0.19
6283-03	0.205	0.57	0.67	0.53	0.32	0.57	0.03	0.41	0.17
6283-04	0.227	0.26	0.53	0.46	0.28	0.53	0.07	0.14	0.18
6283-05	0.148	0.82	0.58	0.40	0.27	0.48	0.03	0.10	0.20
9283-06	0.170	0.88	0.60	0.35	0.25	0.49	0.10	0.13	0.21
6283-07	0.284	1.62	0.99	0.46	0.39	0.70	0.53	0.31	0.59
6283-08	0.420	4.34	2.02	0.60	0.64	0.97	1.25	0.47	1.92
6283-09	4.670	34.66	14.97	2.78	5.11	5.37	13.62	6.40	25.00
6283-10	10.216	62.33	27.79	5.35	9.53	10.70	29.74	7.82	51.60
6283-11	24.545	180.84	32.67	21.28	24.09	35.35	over	35.35	216.00

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com



HBV Seroconversion Panel Donor No. 62825
Product Code: HBV6283

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT AxSYM Core-M	ABBOTT Core-M	ABBOTT HBcAg	ABBOTT Core-TOTAL (Co/S)	ABBOTT AxSYM Core (Co/S)	SORIN ETI-MAK 3	SORIN (INCSTAR) HBsAg	SORIN ETI-Core IGM2	SORIN ETI-AB CoreK2 (Co/S)
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/26/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6283-01	0.04	0.057	NOT TESTED	0.43	0.49	0.38	0.235	0.00	0.16
6283-02	0.05	0.057	0.269	0.30	0.53	0.43	0.157	0.00	0.18
6283-03	0.04	0.057	0.192	0.31	0.51	.055	0.196	0.00	0.19
6283-04	0.04	0.057	0.223	0.40	0.51	0.57	0.196	0.00	0.18
6283-05	0.04	0.057	0.246	0.34	0.51	0.41	0.157	0.00	0.18
6283-06	0.04	0.057	0.215	0.51	0.51	0.49	0.118	0.00	0.18
6283-07	0.04	0.075	0.223	0.42	0.51	0.37	0.157	0.00	0.17
6283-08	0.05	0.057	0.223	0.41	0.51	1.00	0.275	0.06	0.16
6283-09	0.04	0.057	0.254	0.53	0.51	17.86	2.333	0.00	0.16
6283-10	0.04	0.113	0.315	0.40	0.51	23.60	4.235	0.00	0.19
6283-11	0.04	0.075	0.977	0.47	0.53	59.67	9.059	0.00	0.18

PANEL MEMBER	ORTHO VITROS Eci ANTI-HBc TOTAL	ORTHO VITROS Eci ANTI-HBc IgM	ORTHO VITROS Eci ANTI-HBs QUANT
	TEST DATE: 8/10/2010	TEST DATE: 8/10/2010	TEST DATE: 8/10/2010
6283-01	3.30	0.01	0.30
6283-02	3.27	0.01	0.50
6283-03	3.45	0.01	0.20
6283-04	3.56	0.01	0.10
6283-05	3.41	0.01	0.90
6283-06	3.47	0.01	0.00
6283-07	3.48	0.01	0.50
6283-08	3.33	0.01	0.00
6283-09	3.48	0.01	0.40
6283-10	3.43	0.01	0.20
6283-11	3.37	0.01	0.30

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com