



**HBV Seroconversion Panel Donor No. 62967**  
**Part Number: HBV6285**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT	BIOMERIEUX	BOEHR.	ORGANON	ORTHO	ORTHO
		Normal Range (10-45u/L)	HBV REAL TIME CP/ML	VIDAS HBsAg	MANNHEIM HBsAg Enzymun-test ES Systems	TEKNIKA Hepanostika HBsAg Uni-Form II	HBsAg ELISA Testsystem 3	VITROS Eci HBsAg
		TEST DATE: 1/28/1998	TEST DATE: UNK	TEST DATE: 4/22/1998	TEST DATE: 4/23/1998	TEST DATE: 4/23/1998	TEST DATE: 3/18/1998	TEST DATE: 11/03/2015
6285-01	11/19/96	2	NOT DETECTED	0.14	0.54	0.43	0.07	0.04
6285-02	11/21/96	2	NOT DETECTED	0.12	0.36	0.35	0.07	0.05
6285-03	11/26/96	4	NOT DETECTED	0.16	0.36	0.35	0.07	0.05
6285-04	11/29/96	0	NOT DETECTED	0.14	0.33	0.32	0.04	0.06
6285-05	12/03/96	0	NOT DETECTED	0.12	0.31	0.28	0.32	0.05
6285-06	12/06/96	7	NOT DETECTED	0.15	0.36	0.27	0.15	0.05
6285-07	12/20/96	9	<52	0.11	0.29	0.24	0.07	0.04
6285-08	12/22/96	10	73	0.15	0.31	0.31	0.15	0.04
6285-09	12/27/96	10	877	0.41	0.38	0.41	0.29	0.08
6285-10	12/29/96	9	1709	0.42	0.54	0.56	0.49	0.24
6285-11	01/03/97	11	5530	2.66	1.56	2.21	4.18	2.60
6285-12	01/05/97	6	8566	4.32	2.43	3.35	7.87	7.65
6285-13	01/10/97	5	83706	28.10	15.02	20.11	45.58	44.10
6285-14	01/12/97	7	200796	41.64	23.89	26.61	67.62	68
6285-15	01/17/97	12	333238	44.06	45.34	MAX.ABS.	MAX.ABS.	30.9
6285-16	01/19/97	7	481675	47.94	57.37	MAX.ABS.	MAX.ABS.	33.6

PANEL MEMBER	ABBOTT	ORTHO	ROCHE	GENETIC	ABBOTT	ABBOTT	BEHRINGWERKE
	HBeAg (Co/S)	VITROS Eci HBeAg	Cobas Core HBsAg II EIA	SYSTEMS HBsAg (2.0)	AxSYM Core-M	AxSYM Core (Co/S)	Enzygnost HBsAg Monoclonal II
		TEST DATE: 11/03/2015	TEST DATE: 4/23/1998	TEST DATE: 2/4/1998	TEST DATE: 4/7/1998	TEST DATE: 4/7/1998	TEST DATE: 5/4/1998
6285-01	0.238	0.32	0.43	0.193	0.04	0.53	0.39
6285-02	0.231	0.33	0.20	0.216	0.05	0.49	0.36
6285-03	0.277	0.33	0.06	0.205	0.04	0.49	0.31
6285-04	0.215	0.33	0.09	0.182	0.04	0.51	0.34
6285-05	0.392	0.33	0.26	0.261	0.04	0.50	0.38
6285-06	0.185	0.31	0.11	0.239	0.04	0.53	0.28
6285-07	0.208	0.28	0.07	0.273	0.04	0.55	0.30
6285-08	0.269	0.26	0.14	0.182	0.04	0.49	0.32
6285-09	0.231	0.28	0.30	0.261	0.04	0.49	0.43
6285-10	0.238	0.28	0.35	0.500	0.04	0.47	0.47
6285-11	0.262	0.32	4.87	2.033	0.04	0.48	1.46
6285-12	0.362	0.42	6.70	2.456	0.04	0.49	2.39
6285-13	0.462	1.73	56.07	13.600	0.04	0.47	13.77
6285-14	0.623	4.12	57.02	16.022	0.05	0.51	21.43
6285-15	2.415	25.1	MAX.ABS.	15.900	0.05	0.53	30.02
6285-16	2.623	34.6	MAX.ABS.	15.889	0.04	0.50	31.72

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HBV Seroconversion Panel Donor No. 62967**  
**Part Number: HBV6285**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT ANTI-HBc TOTAL (Co/S) TEST DATE: UNK	ORTHO VITROS ECI ANTI-HBc TOTAL TEST DATE: 11/03/2015	ORTHO VITROS ECI ANTI-HBe TEST DATE: 11/03/2015	ORTHO VITROS ECI ANTI-HBs QUANT TEST DATE: 11/03/2015	ABBOTT ANTI-HBc IgM (Co/S) TEST DATE: UNK	ORTHO VITROS ECI ANTI HBc IgM TEST DATE: 11/03/2015	ABBOTT PRISM HBsAg TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3 TEST DATE: 8/27/2009
6285-01	0.42	3.57	0.15	0.5	0.163	0.01	0.64	0.46
6285-02	0.42	3.49	0.16	0.3	0.163	0.02	0.73	0.49
6285-03	0.48	3.61	0.15	0.9	0.163	0.01	0.68	0.50
6285-04	0.45	3.55	0.18	1.9	0.163	0.01	0.00	0.51
6285-05	0.52	3.47	0.16	0.9	0.163	0.01	0.75	0.50
6285-06	0.42	3.44	0.16	1.8	0.163	0.01	1.37	0.48
6285-07	0.45	3.52	0.13	1.5	0.163	0.01	1.18	0.51
6285-08	0.39	3.49	0.13	0.3	0.236	0.01	1.00	0.56
6285-09	0.52	3.37	0.15	0.5	0.236	0.01	2.13	0.72
6285-10	0.45	3.46	0.15	0.3	0.200	0.00	1.77	1.07
6285-11	0.39	3.41	0.13	0.3	0.182	0.01	7.87	4.61
6285-12	0.48	3.41	0.15	0.3	0.200	0.01	11.92	8.06
6285-13	0.45	3.51	0.13	1.6	0.218	0.01	72.54	30.53
6285-14	0.42	3.55	0.13	0.5	0.236	0.01	112.58	30.90
6285-15	0.45	3.49	0.07	1.3	0.218	0.01	68.53	31.83
6285-16	0.48	3.29	0.06	0.6	0.218	0.01	77.12	31.71

PANEL MEMBER	BIOTEST ELISA HBsAg TEST DATE: 4/6/1998	RESEARCH USE PCR (copies/ml) TEST DATE: 6/13/1998	ABBOTT AxSYM HBsAg Reagent Pack TEST DATE: 3/19/1998	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/31/1998	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: 4/30/1998	SORIN ETI-AB CoreK2 (Co/S) TEST DATE: 5/7/1998	SORIN (INCSTAR) HBsAg TEST DATE: UNK	SORIN ETI-Core IGMK2 TEST DATE: 5/11/1998
6285-01	0.31	<100	0.48	0.23	0.24	0.16	0.157	0.39
6285-02	0.35	<100	0.52	0.10	0.57	0.17	0.118	0.38
6285-03	0.23	<100	0.48	0.13	0.34	0.19	0.118	0.40
6285-04	0.26	<100	0.50	0.13	0.24	0.20	0.118	0.59
6285-05	0.26	<100	0.49	0.16	0.37	0.19	0.118	0.62
6285-06	0.25	<100	0.48	0.07	0.47	0.19	0.118	0.38
6285-07	0.26	500	0.49	0.13	0.40	0.18	0.118	0.63
6285-08	0.35	600	0.51	0.16	0.34	0.17	0.118	0.45
6285-09	0.45	2600	0.58	0.23	0.44	0.17	0.157	0.61
6285-10	0.58	6600	0.61	0.56	0.40	0.16	0.157	0.66
6285-11	2.14	16000	1.68	4.32	1.55	0.18	0.588	0.45
6285-12	3.28	36000	2.61	6.79	2.39	0.19	0.824	0.61
6285-13	19.11	260000	13.67	37.81	12.47	0.18	4.824	0.77
6285-14	26.15	500000	19.05	61.05	21.57	0.19	6.000	0.79
6285-15	30.83	1000000	27.52	MAX.ABS.	34.82	0.19	3.176	0.85
6285-16	32.76	950000	32.58	MAX.ABS.	39.47	0.17	3.412	0.68

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HBV Seroconversion Panel Donor No. 62967**  
**Part Number: HBV6285**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)