



**HBV Seroconversion Panel Donor No. 63133**  
**Part Number: HBV6286**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 1/30/1998	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	RESEARCH USE PCR (copies/ml)  TEST DATE: 6/13/1998	BIOMERIEUX VIDAS HBsAg  TEST DATE: 4/22/1998	ORGANON TEKNIKA Heparanostika HBsAg Uni-Form II  TEST DATE: 4/23/1998	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: 3/18/1998
6286-01	12/23/96	11	NOT DETECTED	<100	0.13	0.25	0.01
6286-02	01/14/97	7	<52	100	0.13	0.38	0.07
6286-03	01/16/97	1	127	500	0.19	0.33	0.27
6286-04	01/21/97	7	429	1100	0.24	0.44	0.46
6286-05	01/25/97	5	2152	15000	0.53	0.72	1.10
6286-06	01/28/97	9	5791	23000	1.76	1.94	3.43
6286-07	02/01/97	5	24141	130000	2.25	2.50	5.02
6286-08	02/05/97	5	112916	210000	14.69	14.34	30.90
6286-09	02/08/97	5	419522	750000	40.00	29.97	74.31

PANEL MEMBER	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: 2/4/1998	BIOTEST ELISA HBsAg  TEST DATE: 4/6/1998	ABBOTT PRISM HBsAg  TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3  TEST DATE: 8/27/2009	ABBOTT AxSYM HBsAg  TEST DATE: 3/19/1998	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 3/31/1998	SORIN (INCSTAR) HBsAg  TEST DATE: UNK
6286-01	0.244	0.20	0.31	0.45	0.48	0.16	0.157
6286-02	0.200	0.29	0.84	0.50	0.46	0.23	0.078
6286-03	0.211	0.32	0.38	0.52	0.57	0.13	0.118
6286-04	0.311	0.44	0.86	0.75	0.58	0.36	0.157
6286-05	0.511	0.66	3.29	1.46	0.79	0.07	0.235
6286-06	1.244	1.73	6.57	2.92	1.63	0.20	0.510
6286-07	1.644	2.39	8.66	4.56	2.50	6.30	0.627
6286-08	7.556	12.05	42.33	20.93	10.30	32.08	3.490
6286-09	19.311	31.83	115.23	31.77	24.83	MAX.ABS.	9.137

PANEL MEMBER	ABBOTT AxSYM Core-M  TEST DATE: 4/7/1998	ABBOTT AxSYM Core (Co/S)  TEST DATE: 4/7/1998	ABBOTT HBeAg (Co/S)  TEST DATE: UNK	SORIN ETI-Core IGMK2  TEST DATE: 5/11/1998	SORIN ETI-AB CoreK2 (Co/S)  TEST DATE: 5/7/1998	ABBOTT ANTI-HBc TOTAL (Co/S)  TEST DATE: UNK	ABBOTT ANTI-HBc IgM (Co/S)  TEST DATE: UNK
6286-01	0.05	0.62	0.292	0.34	0.21	0.39	0.200
6286-02	0.05	0.63	0.215	0.30	0.19	0.45	0.200
6286-03	0.05	0.60	0.192	0.34	0.18	0.42	0.218
6286-04	0.04	0.61	0.269	0.29	0.18	0.48	0.218
6286-05	0.05	0.61	0.246	0.34	0.19	0.45	0.218
6286-06	0.04	0.60	0.246	0.55	0.21	0.42	0.200
6286-07	0.05	0.61	0.346	0.27	0.20	0.34	0.218
6286-08	0.05	0.62	0.546	0.24	0.20	0.51	0.218
6286-09	0.04	0.72	1.346	0.22	0.22	0.42	0.200

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HBV Seroconversion Panel Donor No. 63133  
Part Number: HBV6286**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS Eci HBsAg  TEST DATE: 10/20/2015	ORTHO VITROS Eci ANTI-HBc TOTAL  TEST DATE: 10/20/2015	ORTHO VITROS Eci HBeAg  TEST DATE: 10/20/2015	ORTHO VITROS Eci ANTI-HBe  TEST DATE: 10/20/2015	ORTHO VITROS Eci ANTI-HBc IgM  TEST DATE: 10/20/2015	ORTHO VITROS Eci ANTI-HBs QUANT  TEST DATE: 10/20/2015
6286-01	0.04	2.82	0.35	0.14	0.03	2
6286-02	0.05	3.11	0.28	0.14	0.02	1.9
6286-03	0.05	3.09	0.27	0.15	0.02	3.1
6286-04	0.08	3.13	0.31	0.12	0.02	2.8
6286-05	0.45	3.12	0.29	0.14	0.02	2.7
6286-06	2.48	3.31	0.33	0.15	0.02	3.3
6286-07	4.28	3.04	0.5	0.14	0.02	2.4
6286-08	32	3.26	1.68	0.13	0.02	2.3
6286-09	102	2.96	5.84	0.12	0.02	2

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)